
PURITAN
BENNETT

Companion® OxiClip™ PC 20

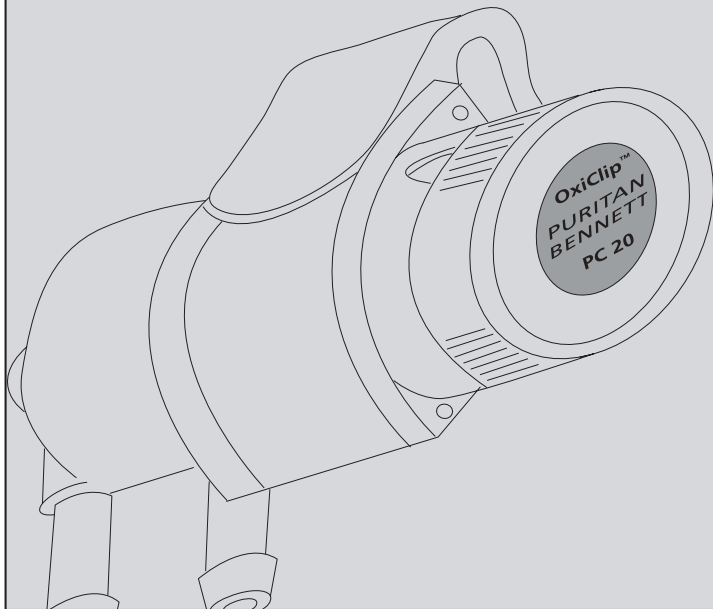
Operating Instructions

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Instrues de utilizaão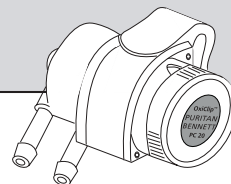

English 3-12
Français 13-22
Español 23-32
Português..... 33-42

Table of Contents

Warnings, Cautions, and Notes	3
Product Description and Features	6
Operating the OxiClip	8
Performance Testing	9
Technical Information	11
Accessories and Options	10
Cleaning and Maintenance	10

CAUTION

Federal (USA) law restricts this device to sale by or on the order of a physician.

Read these operating instructions thoroughly before attempting to use this device.

Warnings, Cautions, and Notes

SAVE THESE INSTRUCTIONS

Read This First

Throughout this manual you will see three kinds of information with emphasis in the text. Carefully read and understand these notices. Each is important and related to the adjacent text.

WARNING

Warnings alert the user to potential serious outcomes (death, injury, or adverse events) to the patient or user.

CAUTION

Cautions alert the user to exercise care necessary for the safe and effective use of the Companion OxiClip PC20.

Indicates points of particular interest or emphasis for more efficient and convenient operation.

WARNING

- **Refer to documentation for information.**

- **Fire intensifying risk** - High concentrations of oxygen can cause rapid burning of other substances. Keep this product away from open flames.

- **Do not smoke near this equipment** - Keep cigarettes or burning tobacco away from area where equipment is operated.

- **Do not oil** - Oxygen is a potentially hazardous substance. Although oxygen is non-flammable, it vigorously supports combustion. Keep oils, grease, and other petroleum-based products away from oxygen. Clean only as directed. Keep this product away from open flames.

- **This product vents a small amount of oxygen during normal operation.** Do not use this product under clothing where oxygen could accumulate and create a fire hazard.

- **The OxiClip is designed for low pressure oxygen sources of 19 to 25 psi (131 to 172 kPa).** The oxygen source pressure for this product should not exceed 30 psi (207 kPa). When used with nominal 20 psi (138 kPa) regulators, make sure they do not exceed the recommended outlet pressure.

WARNING

- **Oxygen supplied from this equipment is for supplemental use and is not intended to be life supporting or life sustaining.** This equipment is not intended for use by patients who would suffer immediate, permanent, or serious health consequences as a result of an interruption in their oxygen supply.

- **Always have the OxiClip serviced at a qualified service facility.** Never attempt to repair or disassemble this equipment.

- **Always keep tubing or oxygen supply line away from path of walking to avoid potential trip or fall.**

- **When using this device with a liquid oxygen source, make sure the source is flow restricted to avoid liquid oxygen from escaping if the OxiClip supply line fails.**

- **Use only with recommended supply line.** Connecting the OxiClip to a wrong gas source could cause inhalation of hazardous substances.


CAUTION

- **Consistent with the recommendations of the medical community on the use of conserving devices, it is recommended that the OxiClip be qualified on patients in the situations it will be used (rest, exercise, sleep).**
 - **Do not use this product with a humidifier. Using a humidifier causes difficulty in triggering the device.**
 - **Always turn off the flow at the oxygen source if the OxiClip is not in use.**
 - **The OxiClip is not intended for use with any mask delivery device.**
 - **Take the following precautions to avoid reduced oxygen or loss of flow to the patient:**
 - **Do not use with a single-lumen cannula or a cannula longer than 7 feet (2.1 meter).**
 - **Patients with nasal obstructions or who are chronic mouth breathers should not use this device.**
 - **Attach the cannula properly.**
 - **Make sure there are no kinks or obstructions in the supply tubing.**
 - **Firmly seat the flow control knob to the desired flow setting.**
 - **Do not set the flow on the oxygen source below the recommended setting.**
-

Product Description and Features

The OxiClip is a pneumatically-powered oxygen conserver designed for low pressure oxygen systems such as a liquid reservoir or an oxygen cylinder equipped with a 20 psi (138 kPa) regulator.

Up to 100 feet (30.5 meter) of oxygen supply tubing can be placed between the source and the OxiClip. The OxiClip is designed to be placed on or near the patient, and a dual-lumen cannula of up to 7 feet (2.1 meter) connects the patient to the device. Placing the OxiClip close to the patient enhances sensitivity and speed of delivery.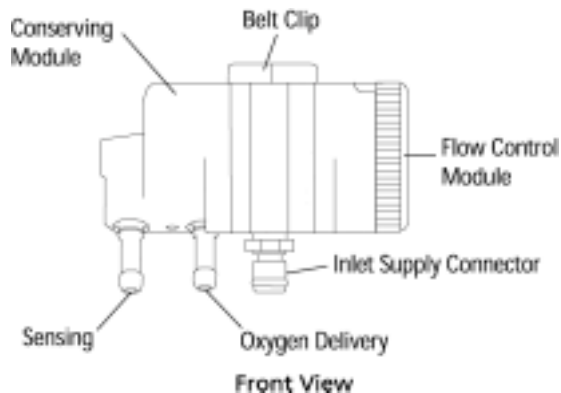

Oxygen from the supply tube enters the OxiClip in the center of the body. Attached to the right side of the oxygen inlet is the flow control module, and attached to the left is the conserving module.


Gas source options:

- Companion stationary or other liquid reservoirs
- Oxygen cylinder with 20 psi (138 kPa) regulator


For the OxiClip to function properly, an adequate supply of oxygen must be supplied by the oxygen source. The length of oxygen supply tubing from the source to the device acts as a reservoir and pressure dampener. Supply pressure at the source must be maintained within OxiClip specifications. Use the supply line shown below to connect the oxygen source to the OxiClip.

See Accessories and Options on page 10 for approved supply lines.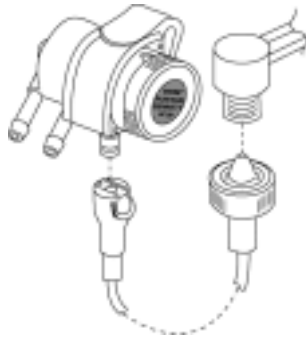

Flow Control Module

The flow control module is a sealed unit on the right side of the OxiClip that controls the flow of oxygen to the patient. Set the flow by turning the control knob to one of 12 positions.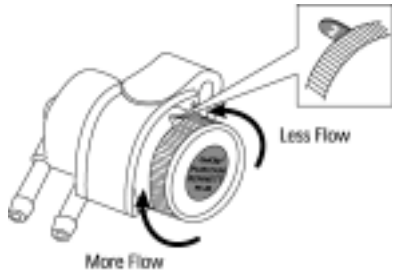


CAUTION

Do not set the flow control between settings. This will result in inaccurate flows. Make sure the flow control is firmly seated on the desired flow setting.

Conserving Module

The conserving module, located on the left side of the OxiClip, features two tubing connectors. The connector nearest the center of the supply inlet provides oxygen flow; the other connector senses inhalations. During normal operation a small amount of oxygen will continuously vent from an opening between the connectors or from around the module.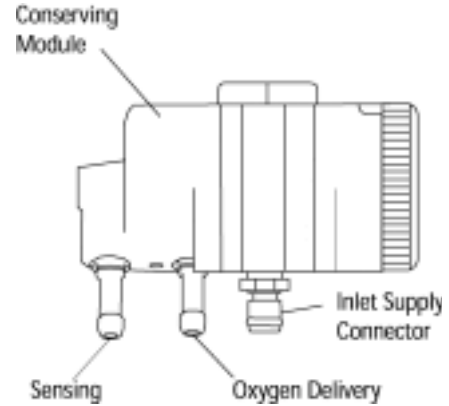


CAUTION


Do not allow fluids or foreign material to enter the vent opening. Failure to follow these instructions can result in malfunction of this device.

Operating the OxiClip

1. Read these operating instructions before attempting to use this device. When in doubt, consult your home care provider or physician.
2. Attach the OxiClip to the oxygen source using supply tubing designed for the OxiClip.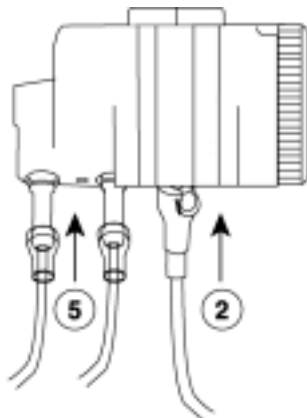

3. Attach the OxiClip securely to the belt or an equally secure location. (See optional mounting attachments.) Make sure that the OxiClip is not under clothing and that it is well ventilated. Place the OxiClip to your right side to improve visibility of the flow control.
4. Set the flowmeter on the oxygen source (Companion Stationary for example) to 6 L/min or more.

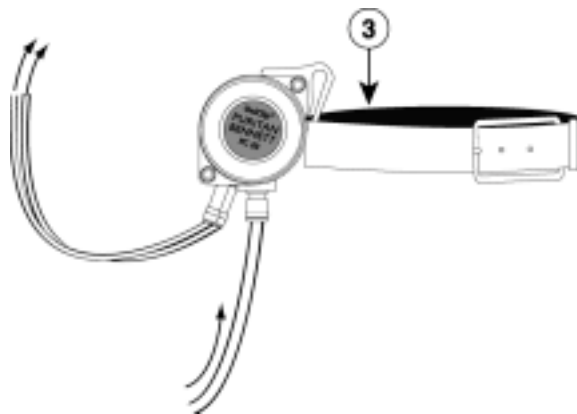5. Attach both ends of the dual-lumen cannula to the two tubing connectors on the conserving module. Either leg of the dual-lumen cannula can be connected to the sensing and oxygen delivery connectors.
6. Set the flow by rotating the flow control knob to the prescribed setting.

CAUTION

If source pressure or source flow is below that required by the OxiClip, the patient may receive less oxygen than desired.

- Place the cannula around your ears and position the prongs in your nose as instructed by your home care provider.

- Breathe in through your nose. You should feel the flow of oxygen as you inhale. Note how the oxygen flow stops as you exhale through your nose. At flows of .75 L/min or less, flow will be continuous. If flow does not stop during exhalation at settings above .75 L/min, contact your home care provider to make sure the unit is operating properly.
- During operation, a normal patient inhalation is sufficient to trigger the device. Once triggered, the OxiClip delivers a 12 mL pulse of oxygen followed by continuous flow at the set flow rate throughout the entire portion of inhalation. When inhalation stops, flow stops.

CAUTION

Use only recommended cannulas (see page 10).

Failure of the patient to trigger a breath results in no flow delivery.

Performance Testing

Qualified service personnel should perform this procedure when a problem is suspected with the OxiClip.

- Visually inspect the OxiClip and supply circuit for loose, damaged, or missing parts.
- Attach the OxiClip to an oxygen source using the recommended supply line and attachments. The system pressure must be between 19 and 25 psi (131 to 172 kPa).
- Attach a dual-lumen cannula to the OxiClip.
- Turn the flow control knob on the Stationary to 6 L/min or more.
- Set the flow on the OxiClip to 2 L/min.
- Place the cannula in place and breathe normally. Make sure the OxiClip triggers on each breath.
- Connect the oxygen delivery connector to a flowmeter using flexible tubing.
- Turn the flow control knob to each setting and verify indicated flow.
- For flows of 1 L/min or more:
 - Place a finger over the vent opening located between the cannula connections, or
 - Attach a flexible tube to the sensing connector and draw a slight vacuum to place the OxiClip into continuous flow.

Technical Information

- Twelve flow settings in L/min are: 0, .25, .50, .75, 1, 1.5, 2, 2.5, 3, 4, 5, 6
At settings of .75 L/min or below, flow is continuous.
- Oxygen savings are nominal 50% at settings of 1 to 6 L/min based on average patient I:E ratio.
 - For a typical COPD patient:
 - Rx 2 L/min, 15-20 BPM, 1:2 to 1:4 I:E = 52% to 60% savings
 - Rx 4 L/min, 15-20 BPM, 1:2 to 1:4 I:E = 55% to 72% savings
 - Rx 6 L/min, 15-20 BPM, 1:2 to 1:4 I:E = 59% to 74% savings
- Oxygen Source Requirements: 19 to 25 psi (131 to 172 kPa), 6 L/min or greater
- Oxygen source connection:
 - Male Quick Connect on the OxiClip
 - DISS O2 nut and nipple on OxiClip supply line.
- Weight: 5oz: (142 g)

Accessories and Options

- B-701421-00 50 Foot (15.2 meter) Supply Line DISS to Female QC Fitting
B-701422-00 14 Foot (4.3 meter) Supply Line DISS to Female QC Fitting
B-701423-00 Nylon Belt
B-701424-00 Cushion/Mattress Mount
B-701425-00 Necklace Mount
B-701432-00 50 Foot (15.2 meter) Supply Line Extension
B-701511-00 3 Foot (.9 meter) Dual-Lumen Nasal Cannula
B-701930-00 7 Foot (2.1 meter) Demand Nasal Cannula
B-701931-00 4 Foot (1.2 meter) Demand Nasal Cannula
6-778057-00 7 Foot (2.1 meter) Dual-Lumen Nasal Cannula
6-778058-00 5 Foot (1.5 meter) Dual-Lumen Nasal Cannula

Cleaning and Maintenance

Do not use alcohol, solvents, polishes, or oily substances on oxygen equipment. If cleaning is necessary, use only warm water or MadaCide (disinfectant). Dampen a cloth in the water or MadaCide and wipe the outside surfaces of the equipment until clean. Do not allow water or MadaCide to enter the vent port, the sensing or delivery connectors, or the oxygen supply inlet connector.

CAUTION

- **Only use water or MadaCide to clean the device; other agents may cause damage to the device.**
 - **NEVER attempt to repair or disassemble this equipment except as described in this manual. You could create a hazardous condition or cause equipment failure. If you have problems, questions, or are unsure if the equipment is operating properly, contact your home care provider.**
-

Replacing Inlet Supply Connector, O-ring, and Tubing Connectors

To replace the inlet supply connector and o-ring, order part numbers B-701746-00 and B-701471-00:

1. Remove old inlet supply connector using 1/2-inch open-end or box-end wrench. If hex portion of connector is damaged and you are unable to use a wrench, remove the connector using a 1/4-inch spiral screw extractor.
2. Fit the new o-ring into the groove at the end opposite the threaded end of the inlet supply connector.
3. Replace the inlet connector in the OxiClip body and tighten with 1/2-inch wrench. Do not overtighten.
4. Check for leaks.

To replace the sensing and delivery tubing connectors, order part number B-701750-00 (order 2 to replace both connectors):

1. Remove old tubing connector using 1/4-inch deepwell socket.
2. Replace with new tubing connector and tighten using socket. Do not overtighten.
3. Check for leaks.

Warranty: Parts and labor are under warranty for one year. Opening the conserving module or flow control module voids all warranties.

Thank you for using Puritan Bennett equipment.

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Companion is a trademark of Puritan-Bennett Corporation.
OxiClip is a trademark of Nellcor Puritan Bennett Inc.
©1999, 2004 Nellcor Puritan Bennett Inc. All rights reserved.
Companion est une marque de commerce de Puritan-Bennett Corp.
OxiClip est une marque de commerce de Nellcor Puritan Bennett Inc.
©1999, 2004 Nellcor Puritan Bennett Inc. Tous droits réservés.