
PROPLUS® 2101KGWA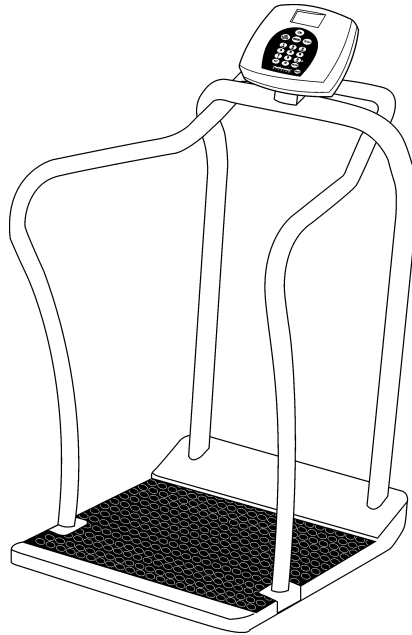

**USER INSTRUCTIONS
INSTRUCCIONES DE USO
MODE D'EMPLOI
INSTRUÇÕES DE USO
BEDIENUNGSANLEITUNG
INSTRUCTIES VOOR DE GEBRUIKER
BRUGSVEJLEDNING
KÄYTTÖOHJEET
ISTRUZIONI PER L'USO
BRUKSANVISNING
POLSKI
SVENSKI
使用说明**

**ISO13485:2003
CERTIFIED**

CE 0459

PELSTAR, LLC 11800 Austin Ave. Alsip, IL 60803

Distributed by:

WelchAllyn®
www.welchallyn.com

Patents D523367, 7550682B2 and D508655 (US); Patents Pending and 0149984/1-8 (EU);
Patents 806658, 879866, ZL200480031711X and 200430004551.2 (PRC) © Pelstar, LLC 2010

PROPLUS[®] 2101KGWA

TABLE OF CONTENTS

ENGLISH.....	2
ESPAÑOL	14
FRANÇAIS	26
PORTUGUÊS	38
DANSK	50
DEUTSCH	62
NEDERLANDS	74
SUOMI	86
ITALIANO	98
NORSK	110
POLSKI	122
SVENSKI	134
简体中文	146

PROPLUS® 2101KGWA

DIGITAL BARIATRIC SCALE WITH HANDRAILS USER INSTRUCTIONS

Thank you for your purchase of this Health o meter® Professional scale.
Please read this manual carefully and keep it for easy reference or training.

TABLE OF CONTENTS

CAUTIONS AND WARNINGS	3
SPECIFICATIONS	3
CERTIFICATIONS / CONNECTIVITY / DISPOSAL / SERVICE	4
ASSEMBLY INSTRUCTIONS	5
SET UP / BATTERIES.....	6
QUICK START INSTRUCTIONS.....	7
MAINTENANCE / CALIBRATION.....	9
TROUBLESHOOTING	10
DETAILED VIEW OF SCALE	11
PARTS FOR MAINTENANCE	12
WARRANTY.....	13

CAUTIONS AND WARNINGS

To prevent injury and damage to your scale, please follow these instructions very carefully.

- Do not transport the scale while the platform is loaded. Unplug the adapter and store both the adapter and cord on the adapter holder before moving the scale. Be sure the scale is tilted enough to assure it is rolling properly on its wheels when it is moved for cleaning or transported from one room to another.
- For accurate weighing the scale must be placed on a flat, stable surface.
- For accurate weighing, verify proper operation according to the procedure described in this manual before each use.
- Do not use in the presence of flammable materials.
- Operating at other voltages and frequencies than specified could damage the equipment.
- If the “LOW BAT” indicator activates, for accurate weighing, replace the batteries or connect the scale to a power source, using the power adapter, as soon as possible.

SPECIFICATIONS

GENERAL

The ProPlus® 2101KGWA Digital Handrail Scale uses highly sophisticated microprocessor technology. Each precision scale is designed to provide accurate, reliable and repeatable weight measurements and features that make the weighing process simple, fast and convenient.

This scale is set up to use motion-sensing weighing technology, to determine the actual weight of a moving patient. The scale may be changed to measure live weight (see the ProPlus® Display Module User Manual for instructions on how to change the scale settings).

Weight is displayed in kilograms. The scale features a wrap-around handrail for patient comfort and safety. It also has a 180° swivel display head for privacy and easy operator reading from either side of the scale. It has two wheels built in for easy mobility. The scale can be operated using its power adapter (included) or by 6-D cell batteries (not included), depending on use—see Connectivity section, below.

2101KGWA SPECIFICATIONS

Capacity and Resolution	2101KGWA: Max 400kg / Min 1kg, e=0.1kg (Class III)	
Power Requirements	120V-240V AC, 50-60Hz - 9V DC adapter, Adapter Model ADPT30 (INCLUDED) or 6 D-cell batteries (not included)	
Environmental	Operating temperatures: 10°C to 35°C (50°F to 95°F) Storage temperatures: 0°C to 50°C (30°F to 125°F) Maximum Humidity: 85% RH	
Physical Dimensions	Platform	Assembled Product Size and Weight
	Length: 56 cm (22") Width: 66 cm (26") Height: 5.7 cm (2-1/4")	Length: 76 cm (30") Width: 70.5 cm (27-3/4") Height: 125 cm (49-2/5") Weight: 32.5 kg (72 lb)

CERTIFICATIONS / CONNECTIVITY / DISPOSAL

CERTIFICATION DESCRIPTIONS

	Scales carrying this symbol meet the Precision Class III standards of the Directive 90/384 EEC
	Products carrying this symbol meet the requirements of: 1. Directive for non-automatic weighing instruments 90/384/EEC 2. Directive governing medical devices 93/42/EEC
	Pelstar, LLC has been officially certified as a manufacture of Health o meter® Professional medical devices to be in compliance with stringent legal requirements. The Pelstar, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.

2101KGWA SCALE CONNECTIVITY INFORMATION

This scale is designed to connect to a computer, printer, monitor or other devices via USB or serial ports, to allow reliable transmission of weight, height and BMI from the scale. This connectivity helps close a source of potential medical errors caused by incorrectly copying and then recording weight information taken. This scale is EMR-EHR compatible. For more information regarding how to connect this scale to devices please see the ProPlus® Display Module User Manual for instructions or request Customer Service through one of the numbers listed below.

2101KGWA SCALE DISPOSAL

This Health o meter® Professional scale must be disposed of properly as electronic waste. Follow the national, regional or local regulations which apply to you for disposal of electronic waste or batteries. Do not dispose of this device in domestic waste.

CUSTOMER SERVICE INFORMATION

For further information or telephone support, please contact Welch Allyn Customer Service at one of the numbers listed below:

- European Call Center - (+353) 46-90-67790
- France - (+33) 1-55-69-58-49
- Germany - (+49) 695-098-5132
- Italy - (+39) 026-968-2425
- Netherlands- (+31) 202-061-360
- Spain - (+34)917-499-357
- Sweden - (+46) 85-853-6551
- United Kingdom - (+44) 207-365-6780
- Latin America - (+1) 305-669-9003
- Middle East - (+49) 7477-927186
- South Africa - (+27) 11-777-7555
- Australia - (+61) 2-9638-3000
- China - (+86) 21-6327-9631

ASSEMBLY INSTRUCTIONS

Before Assembly:

The ProPlus® 2101KGWA Digital Handrail Scale is shipped disassembled in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact Customer Service. The following information details what you will find inside the main carton, as you unpack the parts for assembly.

Remove each assembly from the carton and unwrap packing material carefully, to prevent scratching the unit's parts. Set the carton aside, for storage. To avoid damaging the scale parts when unpacking it, **do not use a box cutter, knife, scissors or any sharp object to open the protective inner packaging.**

Parts List:

- (1) Base (Platform) Assembly
- (1) Center Handrail
- (2) Side Handrails
- (1) Handle
- (1) User information
- (1) Box Containing:
 - (1) Display Assembly
 - (1) Small, Long Box

The Small, Long Box Contains:

- (1) Box Containing the Power Adapter
- (4) Feet
- (1) Rear Cover for Display
- (1) Adapter Holder
- (3) Allen Wrenches—small, medium, large
- (1) Plastic Bag Containing:
 - (1) Front Cover for Display Support
 - (1) Small Rear Cover for Display Support
- Hardware for Assembly (Figure 1)

Tools required: Phillips head screwdrivers (not included), Allen wrenches (included).

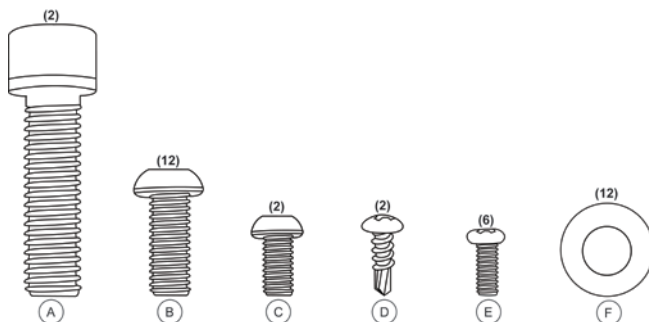
Figure 1

IMPORTANT: Please refer to the maintenance parts list on page 12 for more details about each part.

A detailed view of the scale is illustrated on page 11.

Assembling a 2101KGWA Scale:

It is recommended that the assembly be done by two people.

1. Place scale base on a flat, level, dry floor.
2. Locate and pick up the center handrail, then find the load cell cable, untie it and pull it out the bottom.
3. Position the handrail over the platform and feed the cable through the handrail support in the base, being sure not to pinch the cable.
4. Slide the center handrail over the handrail supports on the base and fasten the handrail to the handrail supports using four (4) M8 1.25 x ¾ screws (B), four (4) M8 washers (F) and the medium Allen wrench (see Figure 1). Lightly tighten the screws. **DO NOT OVERTIGHTEN.** Wait to tighten until Step 6.
5. Attach the left / right side handrails to the center handrail and platform using the two (2) M10 1.5 x 1½ screws (A), eight (8) M8 1.25 x ¾ screws (B) with eight (8) M8 washers (F), and lightly tighten.
6. Starting at the top and working down the sides and base, fully tighten all 14 screws in the handrails.
7. Attach the handle to the back of the center handrail using two (2) medium M6 1.0 x ½ screws (C), and tighten.
8. Pull forward on the side handrails to tip the scale and gently place it face down on the floor.
9. Grasp the load cell cable protruding from the bottom of the base and feed it through the protective metal shield above the wheel in the base of the scale, then plug it into the load cell port in the center of the base.
10. Pull the excess cable through the top of the scale.
11. Thread the 4 feet into the underside of the base. Each foot should be completely hand tightened and then backed out 1-½ full turns. This allows for proper load cell operation for accuracy of the scale.

ASSEMBLY INSTRUCTIONS (CONTINUED)

Assembling a 2101KGWA Scale (continued):

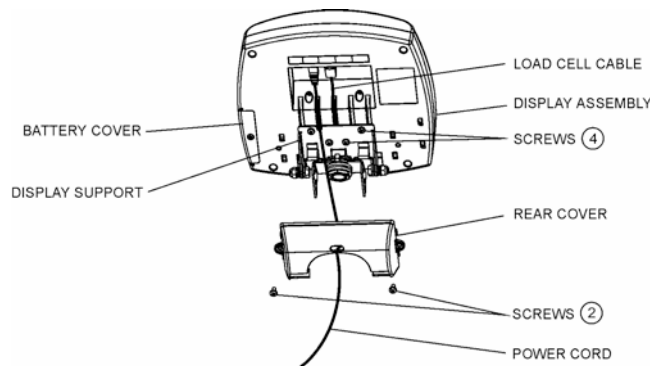
Figure 2

12. Carefully tilt the scale back to an upright position on the floor. DO NOT DROP THE SCALE. Adjust the feet, to level the scale. Be certain that the feet are NOT completely tightened into the base (see Step 11, above).
13. Rotate the display support 90° then position the Display Assembly close to the Display Support. Plug the load cell cable connector into the load cell port on the rear of the Display then insert and secure the cable into the center channel in the display. See Figure 2, above, for an illustration. **Note:** Do not attach the power cord at this point. Wait until Step 19.
14. Slide the display assembly onto the metal display support insuring that the two metal tabs go into the two slots in the rear of the display. **Note: The load cell cable should now be secured in the channel between the display assembly and the display support.**
15. Insert four (4) Phillips pan head M4 screws (E) into the metal display support, and tighten (see Figure 1).
16. Attach the Adapter Holder to the center rail using two (2) #8 x ½ (D) sheet metal screws, and tighten.
17. Connect the power adapter to the power port in the display, then insert and secure the cable into the exposed portion of the second channel from the left in the back of the display (see Figure 2, above). Hold the cord in place through the retaining slot on the rear cover and align the large rear cover tabs with the slots on the rear of the display assembly. Apply light pressure to snap the cover into place and secure it with two (2) Phillips pan head M4 screws (E). **Do not plug the scale into a power source until the scale is fully assembled.**
18. Rotate the display to align it with the center handrail then attach the front and small rear display support covers, applying light pressure to snap these covers into place.
19. Secure the power cord to the inside of the center handrail, using the 4 plastic cable clips, arranged to hold the cord out of the way. The adhesive backing on the clips will peel off then the clips can be fastened to the scale by applying light pressure.
20. The scale is now assembled. Plug the scale adapter into a power source and / or insert batteries.

SET UP

ProPlus® 2101KGWA scale is shipped with the Menu Lock function enabled. The Menu Lock function disables certain buttons on the scale's keypad, preventing unauthorized users from making changes to your scale settings. The Menu Lock can be bypassed temporarily to allow you to customize your scale with the functionality you prefer. Please refer to the ProPlus® Display Module User Manual for instructions on how to bypass the Menu Lock.

Preparing the Scale for Use

1. Remove protective plastic films covering the keypad and display.
2. Place batteries in the battery holder (see "Batteries" information in Figure 3, below).
3. Plug the scale's power adapter into the power source.
4. Be sure all objects are removed from the scale's platform, and then press the **ON/OFF** or **I/O** button to turn the scale on. The display will show "Health o meter ProPlus" and then "0.0KG". **Note:** To adjust the display backlight and/or contrast, please refer to the ProPlus® Display Module User Manual.
Note: The scale will always default to the settings and units last used.
5. When you disconnect the scale's power adapter from the power source, the scale will switch to battery power, if batteries are installed. **Note:** If the set up procedure failed, refer to the troubleshooting instructions. If the problem is not corrected, refer to qualified service personnel.

SET UP / BATTERIES

Replacing / Inserting Batteries

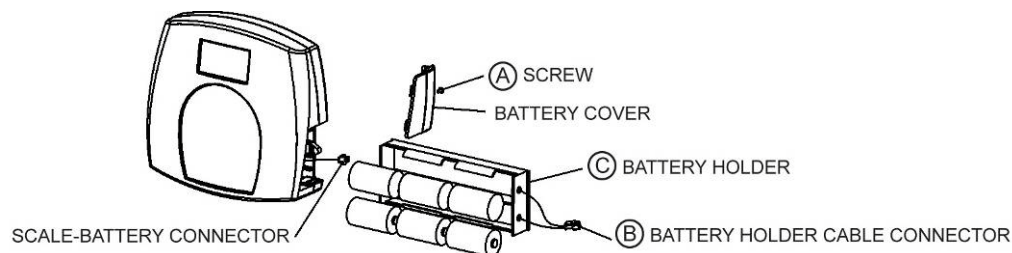
Figure 3 - Batteries

1. Unplug the scale.
2. Remove the battery cover from the display assembly—refer to Figure 3 (A) above.
3. Disconnect battery holder cable connector from the scale-battery connector (B).
4. Carefully remove the battery holder (C) by sliding it out of the display assembly.
5. Replace the batteries with new ones. We recommend EVEREADY Energizer® e2™ batteries.
6. Carefully slide the battery holder (C) into the display assembly.
7. Plug the battery holder cable connector to the scale-battery connector (B).
8. Attach the battery cover to the display assembly and install the screw (A).

Warning: If the scale will not be used for some time, remove batteries to avoid a safety hazard. Disposal of batteries should be according to the national, regional or local regulations which apply to you for their disposal.

QUICK START INSTRUCTIONS

NOTE: This scale will always default to the settings and units last used when it is turned on.

WEIGHING A PATIENT

1. Make sure there is no object on the weighing platform.
2. Press the **ON/OFF** or **I/O** button to turn the scale on.
3. Wait until "0.0Kg" and "ZERO" on the left side of the display appear.
4. Ask the patient to step on the scale. The scale will calculate the patient's weight but the display will not show a weight until the stable weight of the person is determined. Depending on the movement of the patient on the scale, it may take several seconds for the scale to lock onto the weight.
5. Once a weight has been displayed, while the patient is still on the scale, we recommend performing a second weighing by pressing the **REWEIGH** button.
6. After the weight measurement has been taken and is recorded ask the patient to step off the scale.
7. Press the **ON/OFF** or **I/O** button to turn the scale off.

CALCULATING BMI

Body Mass Index is another name for Quetelet Index, a formula that is a convenient and reliable indicator of obesity. **NOTE:** The scale will not calculate BMI for a patient that is 12 kg or less. If a height is not entered for the patient within 30 seconds of pressing the BMI button, the scale will return to the normal weighing mode.

1. Complete steps 1 to 4 above for "Weighing A Patient".
2. After obtaining the patient's weight on the scale, press the BMI (2) button while the scale is on.
3. The scale will prompt you to enter the patient's height. Use the keypad to enter the height.

NOTE: When weighing in kilograms, the height must be entered in 0.1 centimeter increments.

QUICK START INSTRUCTIONS (CONTINUED)

CALCULATING BMI (CONTINUED)

4. Once height is keyed in, press **ENTER**.
5. The display will show the patient's **BMI** (Body Mass Index).
6. Press **EXIT** to return to the normal weighing mode of the scale.

TARE FUNCTION

When using the scale, the weight of an object such as shoes accompanying the patient can be subtracted from the total weight to determine the weight of the patient alone. The Tare Function automatically performs this subtraction. ProPlus® scales allow tare weight to be entered manually by using the keypad or automatically, as outlined in this section.

AUTOMATIC (PUSH BUTTON TARE)

1. With all weight off the scale, press the **ON/OFF** or **I/O** button to turn on the scale.
2. Wait until "0.0Kg" and "ZERO" appear on the left side of the display.
3. Put the weight to be tared on the scale. The display will show a value for that weight.
4. Press the **TARE (9) button**. The scale will zero out and the word "TARE" will appear in the upper right side of the display to indicate there is a tare value for this patient.
5. Remove the weight from the scale. The display will show a negative weight equal to the weight of the shoes or other item, once that item is removed from the scale.
6. Place the patient and the tared object on the scale. The scale will automatically deduct the tared weight, as entered, from the gross weight of the patient and tared object.
7. The weight of the patient will appear on the scale.
8. The tared value is stored in memory until changed, cleared or the scale is turned off.
9. To remove the Tare, press the **TARE (9) button** again. The word "TARE" will disappear from the display and the tare value will be deleted from memory. The scale will resume normal operation.

MANUAL (KEYBOARD OR KEYPAD TARE)

1. Make sure there is no object on the weighing platform.
2. Press the **ON/OFF** or **I/O** button to turn on the scale.
3. Wait until "0.0Kg" and "ZERO" appear on the left side of the display.
4. Press the **TARE (9) button**. The user will be prompted to enter the tare value.
5. Use the keypad to enter the weight of the shoes or other object and then press **ENTER**.
6. The value entered will show as a negative value on the display.
7. Place the patient and the tared object on the scale. The scale will automatically deduct the tared weight, as entered, from the gross weight of the patient and tared object.
8. The weight of the patient will appear on the scale.
9. The tared value is stored in memory until changed, cleared or the scale is turned off.
10. To remove the Tare, press the **TARE (9) button** again or **EXIT**. The word "TARE" will disappear from the display and the tare value will be deleted from memory. The scale will resume normal operation.

REMOVING THE TARE VALUE

Press the **TARE (9) button** again to delete a tare value from memory. The word "TARE" will disappear from the display and the tare value will be deleted from memory. You can also press the **EXIT** button. The scale will resume normal operation.

Additional, complete instructions regarding all of the display and keypad functions and menus can be found in the ProPlus® Display Module User Manual included with this scale.

MAINTENANCE

The following pages provide instructions for maintenance, cleaning, troubleshooting and operator replaceable parts for the ProPlus® 2101KGWA. Maintenance operations other than those described in this manual should be performed by qualified service personnel.

MAINTENANCE

Before first use and after periods of non-use, check the scale for proper operation and function. If the scale does not operate correctly, refer to qualified service personnel.

1. Check overall appearance of the total scale for any obvious damage, wear and tear.
2. Inspect the power adapter for cord cracking or fraying or for broken or bent prongs.

CLEANING

Proper care and cleaning is essential to ensure a long life of accurate and effective operation.

Disconnect the scale from the adapter power source.

1. Clean all external surfaces with a clean damp cloth or tissue. Mild antimicrobial or antibacterial soap and water solution may be used. Dry with a clean soft cloth.
2. Do not immerse the scale into cleaning or other liquid solution.
3. Do not use Isopropyl Alcohol or other solutions to clean the display surface.
4. Do not use abrasive cleaners.

2101KGWA CALIBRATION

This scale has been factory calibrated and does not require calibration prior to use. For maintenance, the scale can be calibrated. Calibration of your ProPlus® scale is performed using kilos (kg)


Operator Action	Display
1. Make sure the display is turned off, then press and hold the HOLD/RELEASE button while pressing the ON/OFF or I/O button.	"Enter load weight: "200.0KG"
2. Using the ▲▼ keys and/or the numeric keypad, set the calibration load to at least half the capacity of the scale (for the best results and greater accuracy, use the full capacity of the scale).	"Enter load weight: "200.0KG"
3. Make sure there is nothing on the scale, then press the ENTER button.	Zero calibration: Please clear the scale
4. Please wait 1-3 seconds, without touching or shaking the scale, to allow the Zero Calibration to finish.	Zero calibration: Please wait
5. No action required by operator.	Weight calibration: Put: XXX.X
6. Load the scale with the required weight and press the ENTER button. Please wait 1-3 seconds, without touching or shaking the scale, to allow the Calibration to finish.	Weight calibration Please wait
7. If calibration is successful, the display will show "Factor OK!". If the calibration process failed, it will show "Factor Fail!".	Factor Update... "Factor OK!" or "Factor Fail!"
8. If successful, the display will begin to reboot.	Rebooting... Please wait
9. The display will return to the start up message.	Health o meter ProPlus
10. Please wait until the scale resumes normal operation and then remove the weight from the scale.	Health o meter ProPlus

TROUBLESHOOTING

Refer to the following instructions to check and correct any failure before contacting Customer Service.

SYMPTOM	POSSIBLE CAUSE	CORRECTIVE ACTION
Scale does not turn on	1. Dead Battery 2. Faulty electrical outlet 3. Bad power supply	1. Replace batteries 2. Use a different outlet 3. Replace adapter
Questionable weight or the scale does not zero	1. External object interfering with the scale 2. The display did not show "0.0" before weighing 3. Scale is not placed on a level floor 4. Scale is out of calibration 5. Improper Tare	1. Remove interfering object from the scale 2. Ask the patient to get off the scale, zero the scale and begin weighing process again 3. Place the scale on a level floor and begin weighing process again 4. Check weight with known weight value 5. Place the item to be tared on the scale. Press REWEIGH. Once the weight of the item is displayed, press TARE. Place the patient and the tared item back on the scale. Press REWEIGH again
Weighing is performed but the display shows "weigh" and "reweigh" every few seconds; the weighing process takes too long and no weight is displayed.	The patient is not standing still	Ask the patient to stand still or you can change to live weight setting
The display shows "Overload" message	The load on the scale exceeds the capacity (400 kg)	Remove the excess weight and use the scale according to its limits
The display shows "LOW BAT" message	The batteries are empty	Replace batteries according to instructions
The display shows "Load Cell Error" message	There is a problem with one or more load cells or the load cell cable is disconnected.	Check load cell cable connection at the display and platform assembly ports. If the problem is not corrected, refer to qualified service personnel to replace the defective load cell

DETAILED VIEW OF 2101KGWA SCALE

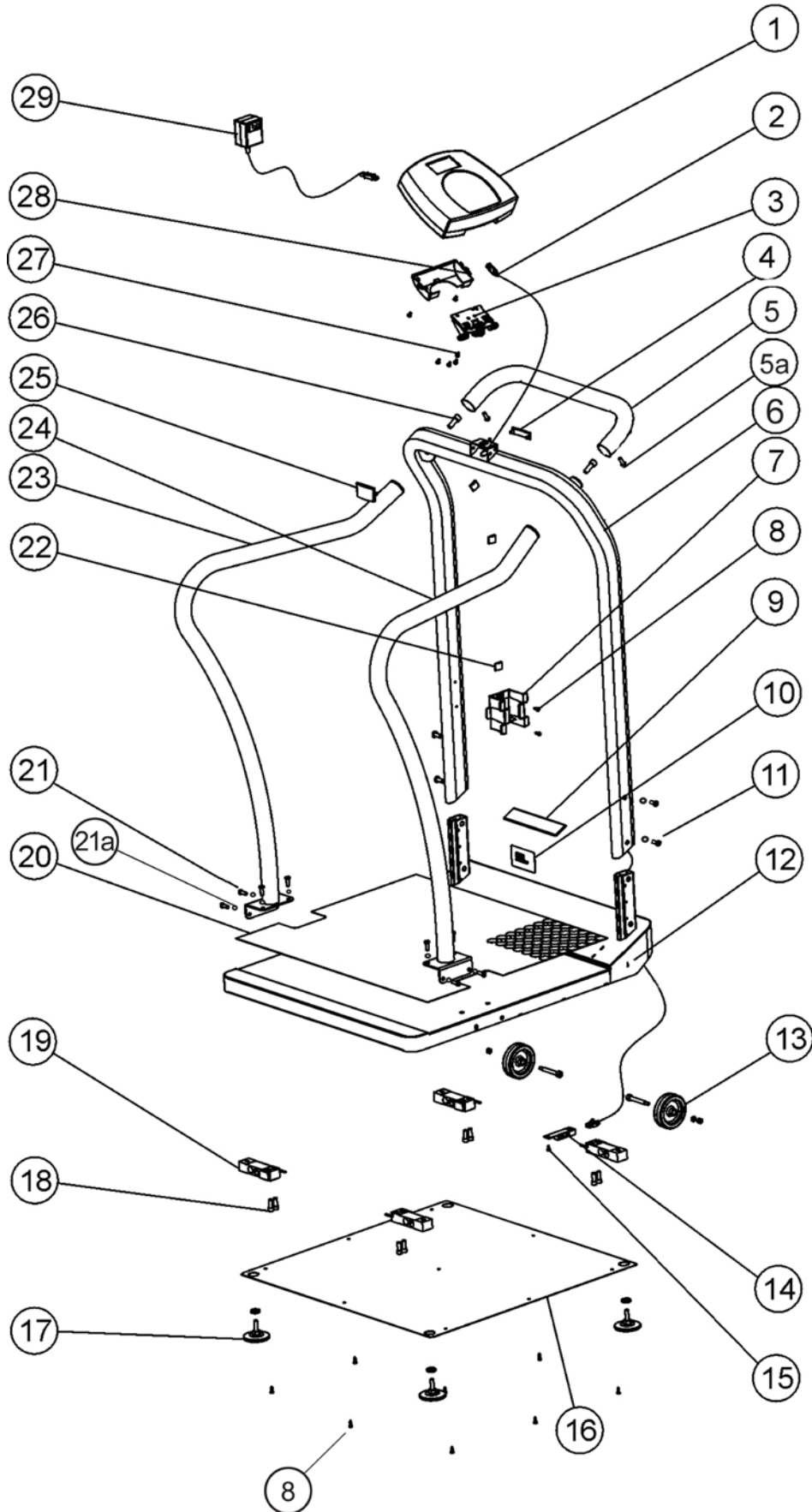2101KGWA PARTS FOR MAINTENANCE

Field Replaceable Parts			
Key No.*	Part No.	Description	Qty.
1	B1109401-0	DISPLAY ASSEMBLY	1
2	B2033701-0	MAIN BOARD TO JUNCTION BOX LOAD CELL CABLE	1
4	B3822601-0	SMALL REAR COVER FOR DISPLAY SUPPORT	1
7	B3817001-0	ADAPTER HOLDER	1
14	B2138901-0	CONNNECTOR BOARD	1
17	B2266701-0	FOOT ASSEMBLY	4
19	B1304202-0	LOAD CELL	4
25	B3822501-0	FRONT COVER FOR DISPLAY SUPPORT	1
28	B3822701-0	REAR COVER	1
29	ADPT30	POWER ADAPTER 120V-240V	1
		INCLUDES ADAPTER, DISPLAY ASSY, 4 LOAD CELLS, CONNECTOR BOARD, 4 FOOT ASSEMBLY	1
Below listed items are for reference only (refer to detailed view on page 11)			
3		DISPLAY SUPPORT	1
5		HANDLE	1
5a		SOCKET HEAD CAP SCREW M6 1.0 x ½	2
6		HANDRAIL	1
8		SHEET METAL SCREW #8 x ½	11
9		HEALTH O METER LABEL	1
10		MODEL LABEL	1
11		SOCKET HEAD CAP SCREW M8 1.25 x ¾	4
12		BASE ASSEMBLY (PLATFORM ASSEMBLY)	1
13		WHEEL ASSEMBLY	2
15		PHILLIPS PAN HEAD SCREW #6 x ¼	1
16		BOTTOM COVER	1
18		SOCKET HEAD CAP SCREW NC5/16-18 x ¾	8
20		MAT	1
21		SOCKET HEAD CAP SCREW M8 1.25 x ¾	12
21a		M8 WASHERS	12
22		PLASTIC CABLE CLIP WITH ADHESIVE BACK	4
23		LEFT SIDE HANDRAIL	1
24		RIGHT SIDE HANDRAIL	1
26		SOCKET HEAD SCREW M10 1.5 x 1½	2
27		PHILLIPS PAN HEAD SCREW M4	6

* See page 11 for a detailed view of parts by number.

WARRANTY

3- YEAR LIMITED WARRANTY

What does the Warranty Cover?

Health o meter® Professional ProPlus® scales are warranted from date of purchase against defects of materials or in workmanship for a period of three (3) years. If you have an equipment problem that you cannot resolve, and the Customer Service provider determines that a defect of material or in workmanship exists, the customer's sole remedy will be repair or replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Authorized dealers selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call the Customer Service Center nearest you for assistance--please see page 4 of this manual. **Before returning a product, you must obtain authorization from Customer Service.**

Customer Service personnel will give you a Service Notification number. Please note this number on the outside of your shipping box. Returns without a Service Notification number will not be accepted for delivery. **Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send it with shipping and insurance prepaid to the address you are given by Customer Service.** Write, "Attention SN# _____", and the shipping address, on the shipping label, following Customer Service instructions.

PRODUCT LIFE: 10 YEARS / 100,000 CYCLES

QNET BV
HOMMERTEWEG 286
6436 AM AMSTENRADE
THE NETHERLANDS

PELSTAR, LLC

11800 South Austin Avenue - B • Alsip, IL 60803 • USA
www.homscales.com

ProPlus® is a registered trademark of Pelstar, LLC. Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license. Health o meter® Professional products are manufactured, designed and owned by Pelstar, LLC and are distributed by Welch Allyn outside the US and Canada. We reserve the right to improve, enhance or modify Health o meter® Professional product features or specifications without notice.

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