Manufacturer:
Cochlear Bone Anchored Solutions AB Konstruktionsvägen 14, SE - 435 33 Mölnlycke, Sweden Tel: +46 31 792 44 00 Fax: +46 31 792 46 95

Regional offices:
Cochlear Ltd (ABN 96 002 618 073) 1 University Avenue, Macquarie University NSW 2109, Australia Tel: +61 2 9428 6555 Fax: +61 2 9428 6352
Cochlear Americas 13059 E Peakview Avenue, Centennial, CO 80111, USA Tel: +1 303 790 9010 Fax: +1 303 792 9025
Cochlear AG EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland Tel: +41 61 205 0404 Fax: +41 61 205 0405

Local offices:
Cochlear Deutschland GmbH & Co. KG Karl-Wiechert-Allee 76A, D-30625 Hannover, Germany Tel: +49 511 542 770 Fax: +49 511 542 7770
Cochlear Europe Ltd 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey KT15 2HJ, United Kingdom Tel: +44 1932 87 1500 Fax: +44 1932 87 1526
Nihon Cochlear Co Ltd Ochanomizu-Motomachi Bldg, 2-3-7 Hongo, Bunkyo-Ku, Tokyo 113-0033, Japan Tel: +81 3 3817 0241 Fax: +81 3 3817 0245
Cochlear (HK) Limited Unit 1810, Hopewell Centre, 183 Queens Road East, Wan Chai, Hong Kong SAR Tel: +852 2530 5773 Fax: +852 2530 5183
Cochlear Medical Device (Beijing) Co LtdUnit 2208-2212, Tower B, No.9 Gemdale Building, No.91 Jianguo Road, Chaoyang District, Beijing, PR China 100022
P.R. CHINA Tel: +86 10 5909 7800 Fax: (+86 10) 5909 7900
Cochlear Limited (Singapore Branch) 6 Sin Ming Road, #01-16 Sin Ming Plaza Tower 2, Singapore 575585 Tel: +65 6553 3814 Fax: +65 6451 4105
Cochlear Korea Ltd 1st floor, Cheongwon building, 828-5, Yuksam dong, Kangnam gu, Seoul, Korea Tel: +82 2 533 4663 Fax: +82 2 533 8408
Cochlear Benelux NV Schalienhoevedreef 20 1, B - 2800 Mechelen, Belgium Tel: +32 1579 5511 Fax: +32 1579 5500
Cochlear Medical Device Company India (P) Ltd Platina Building, Ground Floor, Plot No. C-59, G-Block, BKC, Bandra (E), Mumbai – 400 0051, India
Tel: +91 22 6112 1111 Fax: +91 22 6112 1100
Cochlear Italia S.r.l Via Larga n°33, 40138 Bologna, Italia Tel: +39 051 601 53 11 Fax: +39 051 39 20 62
Cochlear France S.A.S. Route de l'Orme aux Merisiers, Z.I. Les Algorithmes - Bât. Homère, 91190 Saint Aubin, France Tel: +33 811 111 993 Fax: +33 160 196 499
Cochlear Nordic AB Konstruktionsvägen 14, SE - 435 33 Mölnlycke, Sweden Tel: +46 31 335 14 61 Fax: +46 31 335 14 60
Cochlear Tıbbi Cihazlar ve Sağlık Hizmetleri Ltd. Sti. Cubuklu Mah. Bogazici Cad., Bogazici Plaza No: 6/1, Kavacik,TR - 34805 Beykoz-Istanbul, Turkey
Tel: +90 216 538 5900 Fax: +90 216 538 5919
Cochlear Canada Inc 2500-120 Adelaide Street West, Toronto, ON M5H 1T1 Canada Tel: +1 416 972 5082 Fax: +1 416 972 5083

www.cochlear.com

Cochlear™


Baha Divino®

User manual part A

- 3
GB
- 9
ES
- 15
EL
- 21
PT
- 28
HE

English Español Ελληνικά Português עברית **ZONE 2**

1

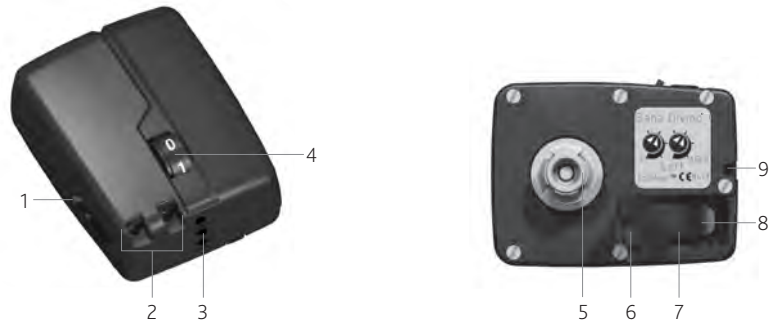2

3

4

5

6

7

8

Welcome

Congratulations on your Cochlear™ Baha Divino® sound processor. You are now ready to use Cochlear's advanced bone conduction sound processor.

This manual is full of tips and advice on how to best use and care for your Baha sound processor. By reading this manual and then keeping it handy for future reference, you'll ensure that you get the most benefit out of your Baha sound processor.

Key to device

See figure **1**

1. Program selector
2. Microphone
3. Direct audio input
4. Volume control
5. Plastic snap connector
6. Battery door
7. Battery compartment
8. Serial number
9. Attachment point for the safety line

Note on figures: The figures included on the front cover correspond to information specific to this model of sound processor. Please reference the appropriate figure when reading. The images shown are not to scale.

Introduction

The Baha Divino® sound processor uses direct bone conduction to transmit sounds to the cochlea (inner ear). It works by combining a sound processor and a small titanium implant that is placed in the skull behind the ear. The skull bone integrates with the titanium implant through a process called osseointegration. This allows sound to be conducted via the skull bone directly to the cochlea, which improves hearing performance.

The Divino comes in both left and right hand versions (indicated on the sound processor label) to allow optimal positioning of the directional microphone. It is essential to use the correct version for the corresponding ear.

To ensure optimal performance, your hearing care professional will fit the processor to suit your needs based on your connection configuration. Be sure to discuss any questions or concerns that you may have regarding your hearing or use of this system with your hearing care professional. Be sure to follow all applicable instructions in this manual based on your specific connection configuration.

Warranty

The warranty does not cover defects or damage arising from, associated with, or related to the use of this product with any non-Cochlear processing unit and/or any non-Cochlear implant. See "Cochlear Baha Global Limited Warranty card" for more details.

Contacting customer service

We strive to provide the best possible products and services to our clients. Your views and experiences of our products and services are important to us. If you have any comments you would like to share, please contact us.

Customer Service - Cochlear Americas

13059 E. Peakview Avenue, Centennial,
CO 80111, USA
Toll free (North America) 1800 523 5798
Tel: +1 303 790 9010 Fax: +1 303 792 9025
Email: customer@cochlear.com

Customer Service - Cochlear Europe

6 Dashwood Lang Road, Bourne Business Park,
Addlestone, Surrey KT15 2HJ, UK
Tel: +44 1932 87 1500 Fax: +44 1932 87 1526
Email: info@cochlear.co.uk

Customer Service - Cochlear Asia Pacific

1 University Avenue, Macquarie University, NSW
2109, Australia
Toll free (Australia) 1800 620 929
Toll free (New Zealand) 0800 444 819
Tel: +61 2 9428 6555 Fax: +61 2 9428 6352 or
Toll free 1800 005 215
Email: customerservice@cochlear.com.au

Key to symbols

Throughout the manual you will see small symbols. Please refer to the list below for explanations:

	"Consult instructions for use" or "Operating instructions"
	"Caution" or "Caution, consult accompanying documents"
	Audible signal
	CE-mark
	Temperature limitation
	Interference risk
	Batch Code
	Serial Number
	Refer to instructions/booklet

Getting started

On/Off

See figure **2**

Off ► When the volume control is in the 0 position it is off. You will feel a "click" as it goes into off mode.

On ► For the sound processor to be on, the volume control must be set to a position higher than 0. You will feel a click as the sound processor moves from off mode to on mode.

Volume control ► If you turn the volume control upwards the volume increases, and if you turn it downwards it will decrease.

Program selector

See figure **3**

Use the program selector to get the right setting.

Program 1 ► Program one uses the omnidirectional microphone so that the sound processor picks up sound regardless of the direction it comes from. This program is most suitable for quieter surroundings.

Program 2 ► In this program setting the directional microphone is activated, emphasising sound from the front and reducing sound received from the side and behind. This program is recommended for noisy surroundings as it improves speech understanding in noise. For best performance in Program 2 the sound processor should be positioned vertically with the microphone at the bottom. See figure **4**

Note: A sound processor with the directional microphone switched on reduces background noise such as the sound of a vehicle approaching from behind. Warning signals may also be suppressed.

Tone and AGCo controls

The tone and AGCo controls are to be adjusted by audiologists only. When fitting the Divino, the audiologist will set the tone and AGCo controls to suit your needs by increasing or decreasing the low frequency output and/or compression threshold. The AGCo ensures a wearer will not be startled by sudden, loud noises and limits distortion, improving sound quality in loud environments.

Telephones and mobiles

See figure **5**

When using telephones, including mobiles, with your processor, you hold the receiver in the normal manner, but close to the microphone inlet of the sound processor instead of close to your ear. It is important that neither you, nor the phone, touches the sound processor as this may cause feedback ("whistling").

Maintenance of your processor

The sound processor can be worn during waking hours apart from when you take a shower, or swim. Keep in mind that the sound processor is an electrical device that should be used with due care and attention. When engaging in physical activities, use common sense. If the activity is of such a nature that the sound processor risks getting damaged (for example contact sports) – remove it! If you choose to keep your sound processor on during physical activity we recommend securing it with the safety line. See figure **6**

When not using your processor

When not in use ensure that the sound processor is switched off and stored carefully where it will not gather dust and dirt. If you will not be using the sound processor for any length of time, remove the battery.

Attention: For children who have been fitted with a Baha sound processor it is extremely important that the parent or caregiver of the child provides all necessary assistance and support in cleaning and maintenance, as detailed above.

Batteries

Zinc-air batteries type 13 should be used. Please note that these batteries are not rechargeable.

Battery replacement

See figure 

Open the battery compartment by releasing the catch on the door, which allows it to swing outwards. Turn the sound processor over and, if the battery is stuck, shake the device gently or pull the battery out. Place the new battery in with the + mark facing upwards and close the cover. If you are unable to slide the cover back easily, the battery may be incorrectly inserted. Please check that the + mark is facing upwards.

 **Caution:** The Baha sound processor contains small parts that may be a risk for small children. Batteries can be harmful if swallowed. Be sure to keep your batteries out of reach of small children or people of diminished mental capacity. In the event a battery is accidentally swallowed, seek medical attention at the nearest emergency centre immediately.

Battery life

Battery life begins decreasing as soon as the plastic strip is removed exposing it to air, even if the battery is not in use.

To maximise battery life, switch off the sound processor when not in use. Battery life varies depending on volume setting, the sound

environment and the program setting. Battery life will decrease significantly when using the Baha sound processor with a connected FM receiver if the FM receiver uses the sound processor as its power source. Using powerful batteries is particularly important when high current consumption is common. The batteries included with the sound processor reflect our latest battery recommendation. Battery function can be related to a number of problems including: no sound, intermittent sound and crackling/buzzing. Thus, when encountering problems one of the first things one can do is to try changing the battery.

Note: Always follow local regulations for battery disposal.

General advice

- A sound processor will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic conditions.
- Infrequent use of a sound processor does not permit a user to attain full benefit from it.
- The use of a sound processor is only part of a hearing rehabilitation and may need to be supplemented by auditory training and instructions in lip-reading.
- The sound processor is suitable for use in electromagnetic environments with mains power of typical commercial or hospital quality, and power frequency magnetic fields of typical commercial or hospital levels. It is intended for use in an electromagnetic environment in which radiated RF (radio frequency) disturbances are controlled.
- Interference may occur in the vicinity of equipment the symbol  to the right.

Accessories

Several accessories are available to enhance your sound experience in different contexts. To ensure the smooth operation of your Baha, only use Cochlear original accessories or accessories approved by Cochlear. See figure 8

Direct audio input and microphone settings

The sound processor has a three-pin DAI (Direct Audio Input) port designed for connecting external equipment. This can be used in program 1, for safety reasons you will still be able to hear sounds from your surrounding environment. In program 2, the DAI is not activated so connected accessories will be muted.

Program setting	1	2
Omni directional microphone	on	off
Directional microphone	off	on
DAI	on	off

Attention: The marking on the plug must match up with the corresponding marking on the sound processor.


Safety instructions when connecting external equipment

Only connection cables and accessories that are supplied or approved by Cochlear should be used. Non-standard connection cables can cause damage to the sound processor and injury to you. If in doubt, consult your audiologist. Cochlear is not liable for any damages arising as a result from this or any other form of incorrect use of the product or its parts.

 **Caution:** It is extremely dangerous and potentially fatal to connect any equipment that is plugged into a 110 and/or 220 V power supply without using the audio adapter.

Warnings and precautions

- The sound processor is a digital, electrical, medical device designed for specific use. As such, due care and attention must be exercised by the user at all times.
- The sound processor is not waterproof! Never wear it in the bath or shower!
- Do not expose the sound processor to extreme temperatures etc. For example do not wear it in the sauna. The sound processor is designed to operate within the temperature ranges +5°C (+41°F) to +40°C (+104°F). In particular, battery performance deteriorates in temperatures under +5°C. The device should not be subject, at any time, to temperatures below -20°C (-4°F) or above +50°C (+122°F).
- This product is not suitable for use in the presence of flammable materials.
- If you are to undergo an MRI (Magnetic Resonance Imaging) procedure, refer to the MRI Reference Card included in the document pack.
- Dispose of electronic items in accordance with your local regulations.
- Do not dispose of in fire.

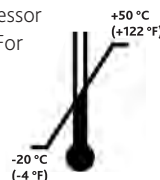Troubleshooter

If you have encountered any problems with your Baha sound processor:

- Check that plastic housing is not touching anything (such as a hat, eyeglasses or similar items).
- Check the volume control.
- Check the program setting.
- Try changing the battery.

If the problem persists, contact your audiologist to get it fixed. Do not try fixing the sound processor yourself.

Service & repairs

If there is any fault with your Baha sound processor, contact your ear clinic or hearing aid centre and they will help you to get it serviced.

Telephone number of hearing centre: _____

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