## Power Sources

(Portable and Direct Plug-In)

Operating Instructions .....1

## Sources d'alimentation

(portative et directe enfichable)

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## Stromquellen

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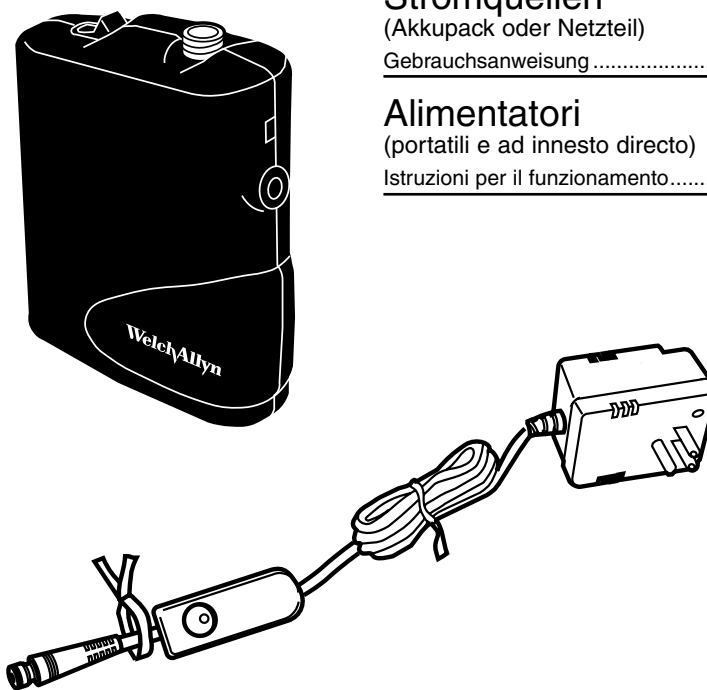
## Alimentatori

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# WelchAllyn® LumiView®

CE

Portable Binocular Microscope  
Microscope binoculaire portatif  
Microscopio binocular portátil  
Tragbares Binokular-Mikroskop  
Microscopio binoculare portatile

---

---

Thank you for purchasing the Welch Allyn LumiView™ Power Source. To ensure accurate and reliable performance of your new instrument, please follow the operating and maintenance instructions found in this manual.

U.S. and International Patents Pending.

**IMPORTANT:** The material outlined in this manual should be reviewed and understood prior to operation of the equipment.

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The CE mark on this device indicates it has been tested to and conforms with the provisions noted within the 93/42/EEC Medical Device Directive.

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**Authorized European Representative Address:**

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European Regulatory Manager  
Welch Allyn, Ltd.,  
Kells Road, Navan,  
County Meath,  
Republic of Ireland  
Tel. 353 46 28122  
Fax 353 46 28536

## Symbols

---

**WARNING:** Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

**CAUTION:** Indicates a potentially hazardous situation which, if not avoided, may result in minor injury or property damage. It also alerts against unsafe practices.

**NOTE:** Provides supplemental information to the text and indicates a potentially hazardous situation, which, if not avoided, may result in property damage. Additionally, it highlights important information on the use of this equipment.

## Standards Compliance

---

Double Insulation

Internally Powered

Listed Product Underwriters Laboratories, Standard UL544

Listed Product Canadian Standards Association, Standard CAN/CSA C22.2 No. 125

Listed Product ETL Laboratories

Listed Standards UL544, CAN/CSA C22.2 No. 125, IEC 601-1, IEC 601-1-2

# Contents

---

|                                                      |   |
|------------------------------------------------------|---|
| CE Approval.....                                     | 2 |
| Symbols .....                                        | 3 |
| Standards Compliance .....                           | 3 |
| Power Sources (Portable and Direct Plug-In) .....    | 5 |
| Portable .....                                       | 5 |
| Features.....                                        | 5 |
| Specifications.....                                  | 5 |
| Connecting Instrument to Portable Power Source ..... | 6 |
| Recharging the Portable Power Source .....           | 6 |
| Accessing Spare Lamp .....                           | 6 |
| Battery Replacement.....                             | 7 |
| Direct Plug-In Transformer .....                     | 7 |
| Product and Accessory Ordering Information .....     | 8 |
| Approvals .....                                      | 8 |
| Service and Warranty .....                           | 9 |

## Power Sources (Portable and Direct Plug-In)

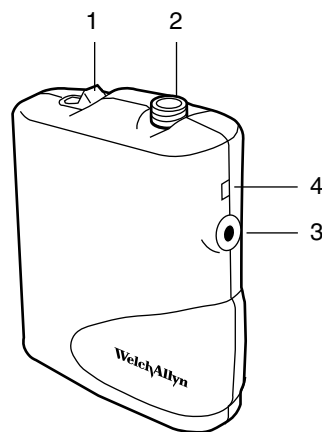
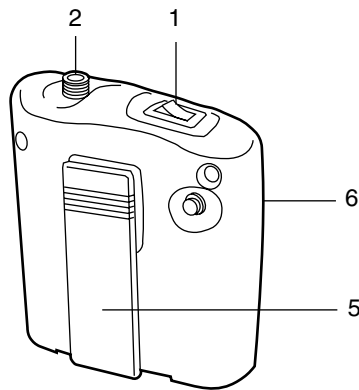
---

### Portable Power Source (Welch Allyn Model No. 75260)

This power source is designed for use in both the office and the hospital and is ideal for carrying to satellite offices. Power On time is 90 minutes after a full charge of 10 hours.

#### Features:

1. On/Off Switch
2. Instrument Connector
3. Charging Transformer Port

4. Charging Indicator
5. Belt Clip
6. Spare Lamp Receptacle


#### Specifications:
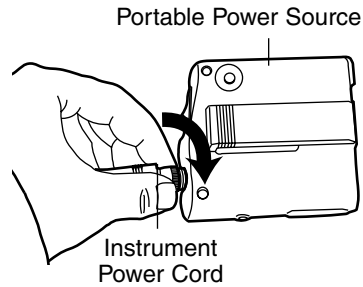
|                  |                                                                   |
|------------------|-------------------------------------------------------------------|
| Battery Type     | Welch Allyn #72250 (nickel-cadmium)                               |
| Voltage/Capacity | 3.6 VDC/1.8 Ahr                                                   |
| Charger          | Welch Allyn #74180<br>(for international orders, specify country) |

---


Connecting Instrument to Portable Power Source:

- Plug instrument power cord into the power cord connector port on top of the power source.
- Push in until firmly in place.
- Gently screw the external connector sleeve clockwise to secure (optional).


Recharging the Portable Power Source:

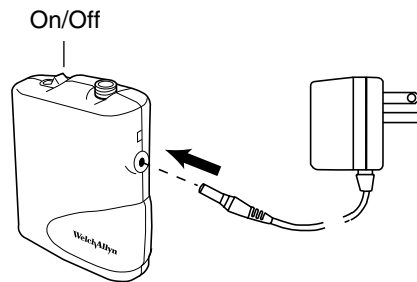- Turn portable power source off.
- Disconnect instrument from power source.
- Plug charging transformer into portable power source.
- Plug charging transformer into wall socket. LED light indicates unit is charging.
- Disconnect charging transformer from power source when fully charged, after 10-12 hours.

Accessing Spare Lamp

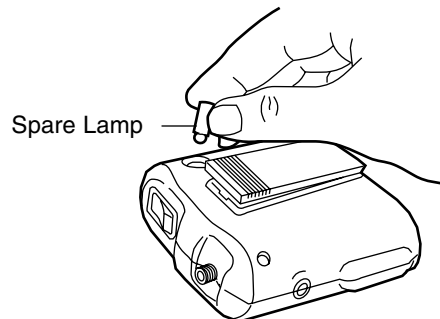- Lift the spare lamp out of its receptacle and use.


---

## Battery Replacement

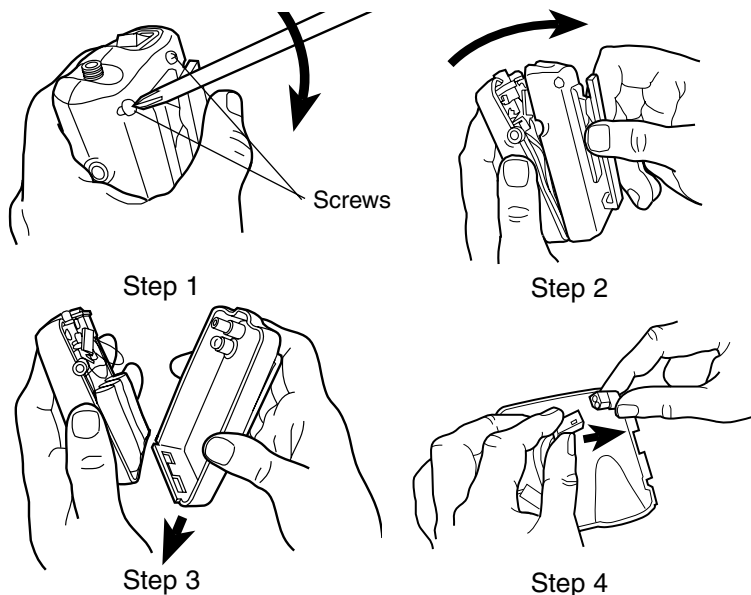
- Be sure power is off.
- Remove both screws from back of power source with Phillips head screwdriver.
- Grasp the belt clip, gently pull back and down on the housing. (Steps 2 and 3)
- Lift battery out and unplug from unit.
- Replace with new battery and reverse above steps.

NOTE: Power source housing screws must be securely tightened in order to properly hold electrical components in place.

## Direct Plug-In Transformer

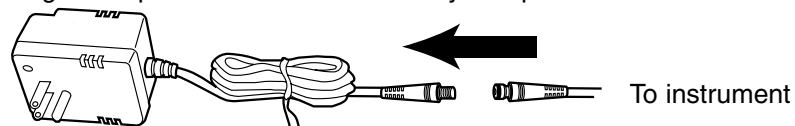- Plug Power cord from the binocular otoscope into the Direct Plug-In Power cord.
- Push in until firmly in place.
- Gently screw the external connector sleeve clockwise to secure (optional).
- The instrument is now ready for use.
- Plug direct power source into a nearby receptacle.

## Product and Accessory Ordering Information

---

| Product No. | Description                                                                    |
|-------------|--------------------------------------------------------------------------------|
|             | Power Sources                                                                  |
| No. 75260   | Portable Power Source w/ charger                                               |
| No. 75265   | International Portable Power Source w/Charger<br>(specify country and voltage) |
| No. 74180   | Charger only for No. 75260                                                     |
| No. 74185   | International Charger only for No. 75265<br>(specify country and voltage)      |
| No. 75210   | Direct Plug-In Power Source                                                    |
| No. 75215   | International Direct Plug-In Power Source<br>(specify country and voltage)     |

Note: The power source options listed above operate with all instrument configurations.

|           | Accessories                              |
|-----------|------------------------------------------|
| No. 08500 | Replacement lamp for device (all models) |

 Note: Only accessories and power sources indicated in this manual are to be used with the Portable Binocular Microscope and Flat Surface Magnifier.

## Approvals

---

UL 544, CAN/CSA C22.2 No.125

IEC 601-1, IEC 601-1-2, CE

## Service and Warranty

---

### Service

For customers in North America, please return instruments requiring service to a Welch Allyn Technical Service Department listed below or to an authorized Welch Allyn distributor.

### Technical Service Department

Welch Allyn, Inc.

4341 State Street Road

Skaneateles Falls, NY 13153-0220 U.S.A.

Telephone: 800-535-6663 (in U.S.A. only) or (315) 685-4560

Fax: 315-685-4653

For customers outside of North America, return your unit to a local, authorized Welch Allyn distributor or to your nearest Welch Allyn service center.

### Warranty

Welch Allyn, Inc. warrants the Portable Binocular Microscope and Flat Surface Magnifier to be free of original defects in material and workmanship and to perform in accordance with manufacturer's specifications for a period of one year from the date of purchase.

If this instrument or any component thereof is found to be defective or at variance with the manufacturer's specifications during the warranty period, Welch Allyn will repair or replace the instrument or component(s) at no cost to the purchaser.

This warranty only applies to instruments purchased new from Welch Allyn or its authorized distributors or representatives. The purchaser must return the instrument directly to Welch Allyn or an authorized distributor or representative and bear the costs of shipping.

This warranty does not cover breakage or failure due to tampering, misuse, neglect, accidents, modification or shipping, and is void if the instrument is not used in accordance with manufacturer's recommendations or if repaired or serviced by other than Welch Allyn or a Welch Allyn authorized representative.

No other express or implied warranty is given.

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Skaneateles Falls, NY 13153-0220, U.S.A.