
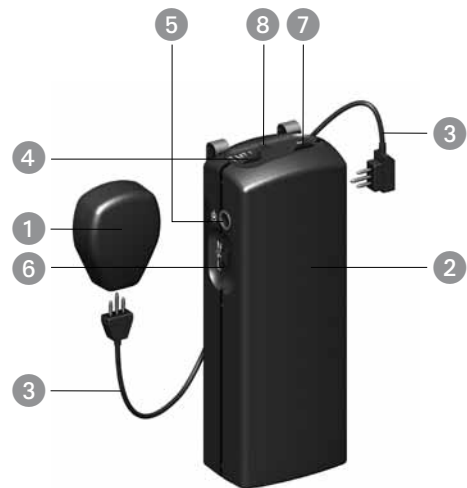
Baha[®] Cordelle II

User Manual

Instructions

English	4
Français	26
Español	48
Português	70

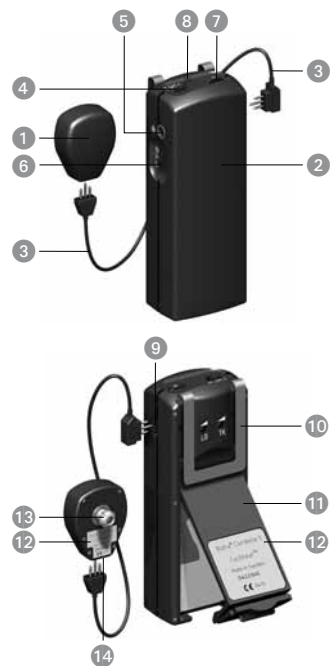- ① Head worn unit
- ② Body worn unit
- ③ Cord
- ④ Program switch (M/MT/T)
- ⑤ Direct audio input
- ⑥ Tone switch
- ⑦ Volume control, on/off
- ⑧ Microphone
- ⑨ Electrical output
- ⑩ Clip
- ⑪ Battery compartment
- ⑫ Serial number
- ⑬ Plastic snap connector
- ⑭ Electrical input

Baha® Cordelle II

The Baha® Cordelle II is a bone conduction sound processor. The Cordelle II is the most powerful Baha sound processor and consists of a body worn unit connected to a head worn unit. The Baha® bone conduction system works by combining a sound processor with an abutment and a small titanium implant placed in the skull behind the ear. The system is based on a process of osseointegration through which living bone tissue integrates with titanium. Thus, the titanium implant becomes one with the bone allowing sound to be conducted via the skull bone directly to the cochlea and improving your hearing.

Please read this manual carefully to learn how to use and maintain your sound processor. Be sure to discuss any questions or concerns that you may have regarding hearing, or use of this system, with your hearing care professional.

We strive to provide the best possible products and services to our clients. Your views and experiences of our products and services are important to us. If you have any comments you would like to share, please contact us:

Address:

Customer Service - Cochlear Americas

13059 E. Peakview Avenue, Centennial, CO 80111, USA
Toll free (North America) 1800 523 5798
Tel: +1 303 790 9010 Fax: +1 303 792 9025
Email: customer@cochlear.com

Customer Service - Cochlear Europe

9 Weybridge Business Park, Addlestone Road, Addlestone, Surrey, KT152UF, UK
Tel: +44 (0) 1932 87 1500 Fax: +44 (0) 1932 87 1526
Email: info@cochlear.co.uk

Customer Service - Cochlear Asia Pacific

14 Mars Road, Lane Cove, NSW 2066, Australia
Toll free (Australia) 1800 620 929
Toll free (New Zealand) 0800 444 819
Tel: +61 2 9428 5202 Fax: +61 2 9428 6352 or Toll free 1800 005 215
Email: customerservice@cochlear.com.au

Getting started

Connecting and disconnecting the unit

A plastic snap connector is mounted to the head worn unit. It is designed to snap onto the abutment and hold the head worn unit securely in place. The body worn unit is equipped with a clip so you can attach the unit to your clothes. The audio cord is connected by attaching the angulated plug to the body worn unit and the straight plug to the head worn unit (see pictures on the inside of the cover). The three-pin plug is reversible.

Connecting ► ① Hold the head worn unit so that the plastic snap connector is accessible and move any hair away from the abutment. Push the device carefully in with a tilting motion and it will attach to the abutment.

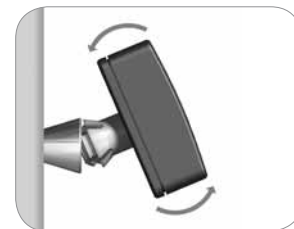
Disconnecting ► ② Tilting the head worn unit slightly in any direction will release it from the abutment.

Warning! Do not push the head worn unit straight in and do not pull it straight out. This puts pressure on the abutment and may cause discomfort.

①

②

On/Off and Volume control

Off ► When the volume control is in the 0 position it is off. You will feel a “click” as it goes into off mode.

On ► For the sound processor to be on, the volume control must be set to a position higher than 0. You will feel a click as the sound processor moves from off mode to on mode.

Volume control ► If you turn the volume control towards the M/T switch the volume increases, and if you turn it away from the M/T switch it will decrease.

Tone switch

The tone switch controls the frequency response. The switch can be set in three different positions.

N = Normal. This gives the widest frequency response.

H = High frequency emphasis. This setting reduces low frequency sounds, which often can improve speech understanding.

L = Low frequency emphasis. This position is often useful during the first period after fitting as it allows you to gradually get used to high frequency sounds you have not heard in a long time.

The tone switch is effective for all inputs (microphone, telecoil and accessories connected via the direct audio input).

Program switch

The body worn unit is equipped with a program switch for the telecoil function. The telecoil improves sound quality and speech understanding when using telephones in homes where there are personal loop systems installed and in other buildings with loop facilities.

There are three programs available. “M” is for listening with the microphone only. In program “M/T” both the microphone and telecoil are active. In program “T” only the telecoil is active.

Trim controls

There are two trim controls on the back of the body worn unit. These are to be adjusted by audiologists only. When fitting the Cordelle, the audiologist will set these trimmers to suit your needs.

Telephones and mobiles

Hold the telephone or mobile “upside down” close to the microphone (using the M program) or close to the telecoil (if using the MT or T program) as seen to the left. The telecoil is placed vertically, close to the electrical input. Please note the magnetic field from many modern phones is not strong enough to be picked up by the telecoil. Some phones can be equipped with a neck-loop to reinforce the magnetic field. It is important that neither you, nor the phone, touches the sound processor as this may cause feedback (“whistling”).

Care and maintenance

Cleaning the abutment area

When the last dressing has been removed and the abutment is exposed hygiene continues to be very important.

Daily care

On a daily basis the area around the abutment should be cleaned to avoid debris build-up. This build-up around the base of the abutment is not a scab and must be removed. Cleaning this area is most easily done when you bath or take a shower as plenty of warm water and soap on the area will help to soften any crust that may have developed around the base of the abutment. The soft brush, which was provided with your sound processor, should be used for cleaning.

During the first weeks, after the dressing is removed, cleaning with the soft brush may feel a little rough. If so, use a strip of baby wipe until your skin becomes less sensitive and you can use the brush comfortably. The soft brush can be used with any mild soap or shampoo and plenty of warm water.

When you brush, aim the bristles at the side of the abutment rather than at the skin. Your focus should be to remove crust and debris around the base of the abutment rather than brushing your skin. Your skin is delicate and should not be treated too harshly.

Remember, it is important to keep your soft brush clean too!

After cleaning, gently dry the area around the abutment with a clean towel or tissue. Do not allow your hair to wrap itself around the abutment. If you use a hairdryer do not keep hot air on the abutment for too long.

Weekly care

If you use an abutment cover it will help keep the inside of the abutment clean. However, the inside of the abutment should still be cleaned on a weekly basis. The soft brush can also be used for cleaning inside the abutment. Again, this is best done in conjunction with a bath or shower when soap and warm water can soften dirt making it easy to remove.

Skin debris or any other dirt in the abutment can prevent a smooth fitting of the abutment to your abutment cover and head worn unit. This could potentially lead to damage of your head worn unit.

You will have routine check-ups, initially every 4-6 months and later every 6-12 months.

However, should you experience any persistent irritation, soreness or any other signs of inflammation around the abutment, **always contact your doctor.**

Maintenance of your Baha® Cordelle II

Do not get the sound processor wet.

The sound processor can be worn during waking hours apart from when you take a bath, shower, or swim.

Keep in mind that the sound processor consists of two delicate electrical devices that should be used with due care and attention. When engaging in physical activities, use common sense. If the activity is of such a nature that the sound processor risks getting damaged (for example contact sports) – remove it!

When not using your Baha® Cordelle II

When not in use ensure that the sound processor is switched off and stored carefully where the head worn unit and body worn unit will not gather dust and dirt. If you will not be using the sound processor for any length of time, remove the battery from the body worn unit. To maintain a good snap connection to the abutment, we recommend that you use an abutment cover when not wearing your head worn unit.

Cleaning your Baha sound processor

Ensure that your snap coupling is always visibly clean. Use a brush for cleaning purposes. (A regular toothbrush with hard bristles is suitable). Do not use water or other liquids when cleaning your Baha.

Keep the plastic housings clean by regularly wiping with a soft, dry cloth.

Attention: For children who have been fitted with a Baha sound processor it is extremely important that the parent or caregiver of the child provides all necessary assistance and support in cleaning and maintenance, as detailed above.

Batteries

The Cordelle II is powered by a 9V battery. Three rechargeable batteries are included with your Cordelle II and these should last for many years if handled with appropriate care. However, non-rechargeable batteries can also be used.

Battery replacement

Open the battery compartment by pressing the cover at the bottom of the body worn unit and lift it up. Turn the body worn unit over and the battery will fall out. If it gets stuck you can shake the device carefully. Place the new battery with the + mark facing the + mark in the battery compartment. Push the battery in and close the cover.

Warning! The Baha sound processor contains small parts that may be a risk for small children. Batteries can be harmful if swallowed. Be sure to keep your batteries out of reach of small children, people of diminished mental capacity and pet animals. In the event a battery is accidentally swallowed, seek medical attention immediately at the nearest emergency centre.

Battery life

To maximise battery life, switch off the sound processor when not in use. Battery life varies depending on volume setting, the sound environment and the program setting. Battery life will decrease significantly when using the Baha sound processor with a connected FM receiver if the FM receiver uses the sound processor as its power source. Using powerful batteries is particularly important when high current consumption is common. Battery function can be related to a number of problems including: no sound, intermittent sound and crackling/buzzing. Thus, when encountering problems one of the first things one can do is to try recharging or changing the battery.

Rechargeable batteries

A fully charged battery will normally last for 1-2 days but this can vary considerably depending on settings and the sound environment. Do not forget to switch off the body worn unit when you are not using it to save the battery.

Due to self-discharge, battery capacity always decreases over time. A fully charged battery will have about 80% of its capacity left after one month (60% after 6 months). After long storage, several charge/discharge cycles may be necessary to restore full capacity.

Batteries have to be charged for 2-4 hours before use. A suitable battery charger can be purchased from Cochlear or a local electronics store. We recommend "intelligent" fast chargers for NiMH batteries.

Non-rechargeable batteries

Non-rechargeable batteries usually have a higher capacity and last longer. Batteries from different manufacturers tend to vary in size. If possible test the battery to make sure it fits and will function properly prior to purchase.

Warning! Non-rechargeable batteries cannot be recharged!

Note: Always follow local regulations for battery disposal.

General advice

- A sound processor will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic conditions.
- Infrequent use of a sound processor does not permit a user to attain full benefit from it.
- The use of a sound processor is only part of hearing rehabilitation and may need to be supplemented by auditory training and instructions in lip-reading.
- The Baha® Cordelle II is suitable for use in electromagnetic environments with mains power of typical commercial or hospital quality, and power frequency magnetic fields of typical commercial or hospital levels. It is intended for use in an electromagnetic environment in which radiated RF (radio frequency) disturbances are controlled.
- Interference may occur in the vicinity of equipment with the symbol to the left.
- For advice on “do’s and don’ts” for your Baha sound processor see the section on Warnings and Precautions (p. 22).

Accessories

Several accessories are available to enhance your sound experience in different contexts. To ensure the smooth operation of your Baha, only use Cochlear original accessories or accessories approved by Cochlear.

Phonak FM-receiver

Enables the use of most Phonak FM transmitters. For more information and orders contact your local Phonak distributor www.phonak.com.

Direct audio input and microphone settings

The body worn unit is equipped with a direct audio input designed for connecting external equipment. The M/T switch automatically switches off the telecoil, if external equipment is connected, as well as regulating which sound sources are accessed (see table on page 20).

Depending on the signal level, the volume control may have to be adjusted to achieve a normal listening level. If sound is distorted at a normal listening level, the electrical signal into the Cordelle II may be too high. If possible reduce the volume at the signal source so that the volume control on the Cordelle II can be left in a normal position.

Sound sources accessed

M/T switch position	Direct audio input NOT connected	Direct audio input connected
M	Microphone only	Microphone only
M/T	Microphone and telecoil	Microphone and direct audio input
T	Telecoil only	Direct audio input only

Safety instructions when connecting external equipment

Only connection cables and accessories that are supplied or approved by Cochlear should be used. Non-standard connection cables can cause damage to the sound processor and injury to you. If in doubt, consult your audiologist. Cochlear is not liable for any damages arising as a result of this, or any other form of incorrect use of the product or its parts.

Warning! It is extremely dangerous and potentially fatal to connect any equipment that is plugged into a 110 and/or 220 V power supply without using the audio adapter.

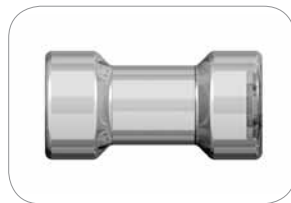
Abutment cover

An abutment cover is included in the sound processor kit. The cover helps conceal and protect the abutment when the sound processor is not in place. This improves aesthetics and prevents dirt from collecting in the abutment, which helps maintain a good snap connection between the abutment and sound processor. Simply press the cover gently onto the abutment.


Share the Experience

The test rod provided can be used by others so that they can listen to your Baha sound processor themselves. Snap the unit onto the test rod and hold the rod against the skull bone behind the ear while keeping the ears plugged (take care not to allow the sound processor to touch anything except the test rod as this may cause feedback).

Warnings and precautions

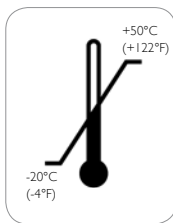- The sound processor is a digital, electrical, medical device designed for specific use. As such, due care and attention must be exercised by the user at all times.
- Neither the body worn unit nor the head worn unit are waterproof! Never wear them in the bath or shower! If the body worn unit gets wet by accident open the battery door immediately and take out the battery. Leave the battery door open until it is dry.
- Do not expose the sound processor to extreme temperatures. For example do not wear it in the sauna. The sound processor is designed to operate within the temperature ranges +5°C (+41°F) to +40°C (+104°F). In particular, battery performance deteriorates in temperatures under +5°C. The device should not be subject, at any time, to temperatures below -20°C (-4°F) or above +50°C (+122°F).
- This product is not suitable for use in the presence of flammable materials.
- If you are to undergo an MRI (Magnetic Resonance Imaging) procedure, the head worn unit should be removed beforehand. The implant and abutment remain in place.
- Portable and mobile RF (radio frequency) communications equipment can affect the performance of your Baha.
- Dispose of electronic items in accordance with your local regulations.
- Do not dispose of in fire.

Troubleshooter

If you encounter any problems with your Baha sound processor:

- Check that the head worn unit is not touching anything (such as a hat, eyeglasses or similar items).
- Check the volume control.
- Check the program setting.
- Try changing the battery.

If the problem persists, contact your audiologist to get it mended.

Do not try fixing the sound processor yourself.

Service, repairs & warranty

Service and repairs

If there is any fault with your Baha sound processor, contact your ear clinic or hearing aid centre and they will help you to get it serviced.

Telephone number of hearing centre: _____

Contact person: _____

Telephone number of ear clinic: _____

Contact person: _____

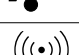
Warranty


The warranty does not cover defects or damage arising from, associated with, or related to the use of this Product with any non-Cochlear processing unit and/or any non-Cochlear implant. See "Cochlear Baha Global Limited Warranty card" for more details.

DISCLAIMER: Cochlear reserves the right to make any necessary changes and improvements to the product.

Key to symbols

Throughout the manual you will see small symbols. Please refer to the list below for explanations:

Symbol	Explanation	Symbol	Explanation
	"Consult instructions for use" or "Consult operating instructions"		Manufacturer
	"Caution, consult accompanying documents" or "Attention, see instructions for use"		Date of manufacture
	Audible signal		Batch Code
	CE-mark		Serial Number
	Temperature limitation		Caution. Federal law (USA) restricts this device to sale by or on the order of a licensed medical practitioner
	Interference risk		

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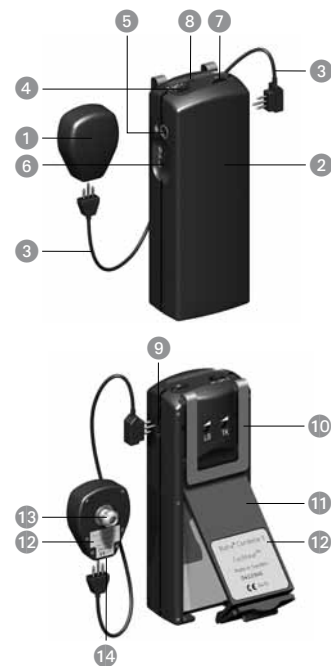Instructions for use in the USA

Caution: Federal law (USA) restricts this device to sale by or on the order of a medical practitioner.

Warning to hearing aid dispensers

A prospective user of Baha® must always have a medical evaluation by a licensed physician (preferably an ear specialist) and have an implant placed in the skull bone before a Baha sound processor can be dispensed.

Important notice for prospective users of a Baha® sound processor

- A prospective user of Baha must always have a thorough medical evaluation by a licensed physician (preferably a physician who specialises in diseases of the ear, otolaryngologist, otologist, or otorhinolaryngologist) and have an implant placed in the skull bone before a Baha sound processor is purchased.
- The purpose of the medical evaluation is to ensure that all medically treatable conditions that may affect hearing are identified and treated before the sound processor is purchased. Following the medical evaluation, the physician will give you a written statement declaring that your hearing loss has been medically evaluated and that you may be considered a candidate for Baha. The physician will refer you to an audiologist or a hearing aid dispenser, as appropriate, for an audiological evaluation.
- Federal law restricts the sale of Baha sound processors to those individuals who have obtained a medical evaluation from a licensed physician. Federal law permits a fully informed adult to sign a waiver statement declining the medical evaluation for religious or personal beliefs that preclude consultation with a physician. The exercise of such a waiver is not in your best health interest and its use is strongly discouraged.

A word of caution about batteries

Batteries can be harmful if swallowed. Be sure to always keep your batteries out of reach of small children or people of diminished capacity. In the event a battery is accidentally swallowed, seek prompt medical attention at the nearest emergency centre, or call the **National Button Battery Hotline collect at (202) 625 - 3333**

Cochlear™

Manufacturer:

Cochlear Bone Anchored Solutions AB Konstruktionsvägen 14, SE-435 33 Mölnlycke, Sweden Tel: +46 31 792 44 00 Fax: +46 31 792 46 95

Regional offices:

Cochlear Americas 13059 E. Peakview Avenue, Centennial, CO 80111, USA Tel: +1 303 790 9010 Fax: +1 303 792 9025

Cochlear AG European Headquarters, Peter Merian-Weg 4, CH-4052 Basel, Switzerland Tel: +41 61 205 0404 Fax: +41 61 205 0405

Cochlear Ltd (ABN 96 002 618 073) 14 Mars Road, Lane Cove NSW 2066, Australia Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

Local offices:

Cochlear Benelux NV Schaliënhoevedreef 20 i, B-2800 Mechelen, Belgium Tel: +32 15 79 55 77 Fax: +32 15 79 55 70

Cochlear Canada Inc 2500-120 Adelaide Street West, Toronto, ON M5H 1T1, Canada Tel: +1 416 972 5082 Fax: +1 416 972 5083

Cochlear (HK) Ltd Beijing Representative Office Unit 2208-2212, Tower B, No.9 Germdale Building, No.91 Jianguo Road, Chaoyang District, Beijing, PR China 100022 P.R. CHINA Tel: +86 10 5909 7800 Fax: +86 10 5909 7900

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Cochlear Europe Ltd 9 Weybridge Business Park, Addlestone Road, Addlestone, Surrey KT15 2UF, United Kingdom Tel: +44 1932 87 1500 Fax: +44 1932 87 1526

www.cochlear.com

Baha, Baha Divino, Baha Intenso and Vistafix are trademarks of Cochlear Bone Anchored Solutions. Cochlear and the elliptical logo are trademarks of Cochlear Limited. © Cochlear Bone Anchored Solutions 2010. All rights reserved.
JUL10. Printed in Sweden.

E81157B

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Cochlear Nordic AB Konstruktionsvägen 14, SE-435 33 Mölnlycke, Sweden Tel: +46 31 335 14 61 Fax: +46 31 335 14 60