OPERATOR'S MANUAL - ENGLISH
MANUAL DE INSTRUCCIONES -
DOMESTIC SPANISH

PLATINUM 5
HFII

OXYGEN CONCENTRATORS STANDARD,
WITH/SENSO₂[®] AND HOMEFILL™ II

CONCENTRADORES DE OXÍGENO
STANDARD, CON/SENSO₂[®] Y HOMEFILL™ II

PART ONE

CONCENTRATOR INFORMATION

Invacare 5 *with*/SensO₂
Model IRC5LXO2

Invacare 5 *con*/SensO₂
Modelo IRC5LXO2

Invacare 5
Model IRC5LX

Invacare 5
Modelo IRC5LX

DEALER: THIS MANUAL MUST BE GIVEN TO THE END USER.

USER: BEFORE USING THIS CONCENTRATOR, READ THIS MANUAL AND SAVE FOR FUTURE REFERENCE

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Yes, you can.®

SPECIAL NOTES

WARNING/CAUTION notices used in this manual apply to hazards or unsafe practices which could result in personal injury or property damage.

NOTICE

THE INFORMATION CONTAINED IN THIS DOCUMENT IS SUBJECT TO CHANGE WITHOUT NOTICE.

WARNING

DO NOT OPERATE THIS EQUIPMENT WITHOUT FIRST READING AND UNDERSTANDING THIS MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS, AND INSTRUCTIONS, CONTACT INVACARE TECHNICAL SERVICES BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE SERIOUS INJURY OR PROPERTY DAMAGE MAY RESULT.

SAVE THESE INSTRUCTIONS

DANGER

Users MUST NOT SMOKE while using this device. Keep all matches, lighted cigarettes or other sources of ignition out of the room in which this product is located. **NO SMOKING** signs should be prominently displayed. Textiles and other materials that normally would not burn are easily ignited and burn with great intensity in oxygen enriched air. Failure to observe this warning can result in severe fire, property damage and cause physical injury or **DEATH**.

**CONCENTRATOR
TABLE OF CONTENTS
PART ONE**

SPECIAL NOTES	2
SAFETY SUMMARY	4
FEATURES	7
SPECIFICATIONS	8
HANDLING	11
Unpacking	11
Inspection	11
Storage	11
INTRODUCTION	12
OPERATING INSTRUCTIONS	12
Select a Location	12
Recommended End-User Guidelines for Optimum Performance	12
Plug in Power Cord	13
Connect Humidifier (if so prescribed)	13
Power Switch	15
Flowrate	15
Low Flow alarm	16
SensO ₂ [®] Oxygen Purity Indicator (if the unit features the O ₂ Sensor)	16
Initial Startup of the Concentrator	16
Explanation of Indicator Lights	17
Indicators (if the unit does not feature the O ₂ Sensor)	17
CONCENTRATOR MAINTENANCE	18
Cleaning the Cabinet Filters	18
Cabinet	19
Humidifier	19
TROUBLESHOOTING GUIDE	19
PREVENTIVE MAINTENANCE RECORD	21
WARRANTY	22
SPANISH OWNER'S MANUAL (MANUAL DEL USARIO)	24
CONCENTRATOR USE WITH HOMEFILL II	PART TWO

SAFETY SUMMARY

CAUTION

“Caution: Federal law restricts this device to sale or rental by or on order of a physician, or any other practitioner licensed by the law of the State in which he/she practices to use or order the use of this device.”

Risk of electric shock. **DO NOT** disassemble. Refer servicing to qualified service personnel.

READ THE FOLLOWING INFORMATION BEFORE OPERATING THIS PRODUCT.

WARNING

The use of oxygen therapy requires that special care be taken to reduce the risk of fire. Any materials that will burn in air, and some that will not, are easily ignited and burn rapidly in high concentrations of oxygen. For safety concerns, it is necessary that all sources of ignition be kept away from the product and preferably out of the room in which it is being used. **NO SMOKING** signs should be prominently displayed.

A spontaneous and violent ignition may occur if oil, grease or greasy substances come in contact with oxygen under pressure. These substances **MUST** be kept away from the oxygen concentrator, tubing and connections, and all other oxygen equipment. **DO NOT** use any lubricants unless recommended by Invacare.

For optimum performance, Invacare recommends that each concentrator be on and running for a minimum of 30 minutes at a time. Shorter periods of operation may reduce maximum product life.

If the concentrator has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water, call Qualified Service Personnel for examination and repair.

Keep the cord away from **HEATED** or **HOT** surfaces.

Do not move or relocate concentrator by pulling on the cord.

NEVER drop or insert any object into any opening.

WARNING (CONTINUED)

NEVER block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air opening may be blocked. Keep the openings free from lint, hair and the like. Fill humidifier with water to the level shown by the manufacturer. **DO NOT** overfill.

Keep unit at least 3 inches away from walls, draperies, furniture, and the like.

Invacare recommends that **Crush-Proof** oxygen tubing (supplied by Invacare) be used with this product and **NOT** exceed 50 ft. (15.2m) in length.

MAINTENANCE

The Invacare Oxygen Concentrator was specifically designed to minimize routine preventive maintenance at intervals of once per year. Only professionals of the healthcare field or persons fully conversant with this process such as authorized or factory trained personnel should perform preventive maintenance or performance adjustments on the oxygen concentrator.

RADIO FREQUENCY INTERFERENCE

Most electronic equipment is influenced by Radio Frequency Interference (RFI). **CAUTION** should be exercised with regard to the use of portable communications equipment in the area around such equipment.

TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURY TO PERSONS.

Avoid using while bathing. If continuous usage is required by the physician's prescription:

The concentrator must be located in another room at least seven (7) feet from the bath.

DO NOT come in contact with the concentrator while wet.

DO NOT place or store product where it can drop into water or other liquid.

WARNING (CONTINUED)

DO NOT reach for product that has fallen into water.
UNPLUG IMMEDIATELY.

A product should **NEVER** be left unattended when plugged in.

This device is to be used only in accordance with the prescription of a physician and this **Owners Manual**. If at any time the patient or attendant conclude that the patient is receiving an insufficient amount of oxygen, the supplier and/or physician should be contacted immediately. No adjustments should be made to the flowrate unless prescribed by a physician.

Close supervision is necessary when this product is used near children or physically-challenged individuals.

Use this product for only intended use as described in this manual.

DO NOT use parts, accessories or adapters other than those authorized by Invacare.

DO NOT connect the concentrator in parallel or series with other oxygen concentrators or oxygen therapy devices.

Use of some administration accessories or certain humidifiers not specified for use with oxygen concentrator may impair the performance.

In certain circumstances oxygen therapy can be hazardous. Invacare recommends that you seek medical advice before using this machine.

This equipment is to be used as an oxygen supplement and is not considered life supporting or life sustaining.


Avoid creation of any spark near medical oxygen equipment. This includes sparks from static electricity created by any type of friction.

POLARIZED PLUG INSTRUCTION


As a safety feature, this appliance has a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. **DO NOT ATTEMPT TO DEFEAT THIS SAFETY FEATURE.**

FEATURES

FRONT VIEW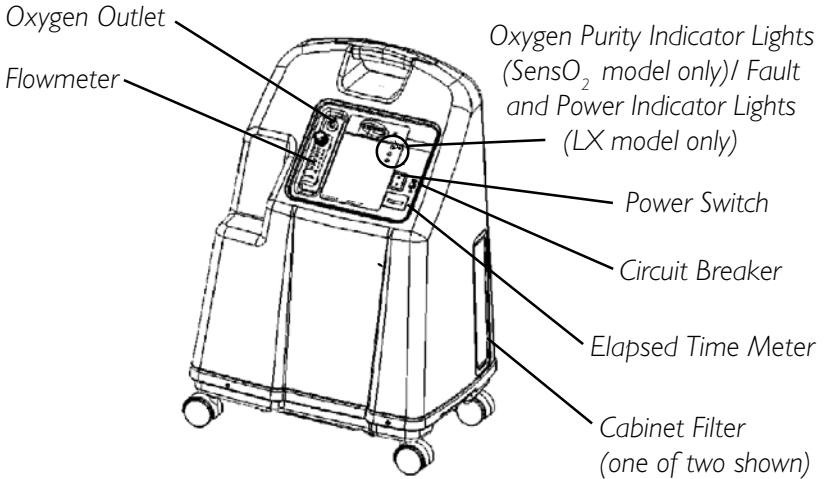

REAR VIEW

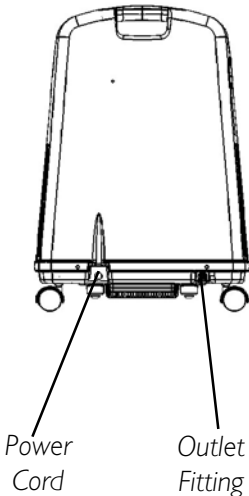ACCESSORIES (NOT SHOWN):

Remote Humidifier/Flowmeter
Accessory - IRCRF5

PreciseR_x[™] Pediatric Humidifier/
Flowmeter Accessory - IRCPF16

Venture[®] Homefill II home oxygen
compressor - IOH200

NOTE: This outlet fitting is to be used only for filling oxygen cylinders with the Venture HomeFill II home oxygen compressor. The outlet fitting does not affect concentrator performance. Refer to the HomeFill II owner's manual, part number 1100873 or Part Two of this manual, for connection and operating instructions. When not in use, the plug provided with the concentrator should be inserted into the outlet fitting. For more information about the HomeFill II, contact your Invacare dealer.

SPECIFICATIONS

SPECIFICATIONS

Electrical Requirements:	120 VAC \pm 10% (132 VAC/108 VAC), 60 Hz
Rated Current Input:	4.3 A
Sound Level:	50 dB Average
Altitude:	Up to 6,000 ft. (1828 meters) above sea level without degradation of concentration levels. From 6,000 ft. (1828 meters) to 13,129 ft. (4000 meters) below 90% efficiency.
*Oxygen Output Concentration Levels:	ALL 5LXO2/5LX MODELS 95.6% to 87% at 1/2 to 5 L/min. * Stated concentration levels achieved after initial warm-up period (approximately 30 minutes).
Maximum Outlet Pressure:	5 \pm 0.5 psi. (34.5 \pm 3.45 kPa.)

5LXO₂ INDICATORS:

Label Symbol	O ₂ Purity	Indicator Lights
	SYSTEM FAILURE Below 73%	RED - Continuous Audible Alarm Sieve-GARD™ Compressor Shutdown
	73% to 85%	YELLOW
O ₂	SYSTEM OKAY Over 85%	GREEN

5LX INDICATORS:

Label Symbol	Status	Indicator Lights
	SYSTEM FAILURE	RED
I/O	SYSTEM OKAY	GREEN

SPECIFICATIONS (CONTINUED)

Flow Range:	1/2 - 5 L/min. For flowrates less than 1 L/min., we recommend the use of the Invacare Pediatric Flowmeter IRCPFI6 Accessory.
Low Flow Alarm:	0 L/min. to 0.5 L/min., Rapid Audible Alarm Beeping (No Accessories Connected)
Average Power Consumption:	400W
Pressure Relief Mechanism Operational at:	35 psi \pm 3.5 psi (24.1 kPa \pm 24.1 kPa)
Change in maximum recommended flow when back pressure of 7kPa is applied:	0.7 L/min.
Filters:	Cabinet (2), Outlet HEPA and Compressor Inlet.
Safety System:	Current overload or line surge shutdown. High temperature compressor shutdown. High Pressure Alarm w/compressor shutdown. Low Pressure Alarm w/compressor shutdown. Battery Free Power Loss Alarm. SensO ₂ Oxygen System (SensO₂ Model) Low Flow Alarm.
Width:	18 3/8" +/- 3/8" (46.7 cm \pm 1 cm)
Height:	26 3/8" +/- 3/8" (67.0 cm \pm 1 cm)
Depth:	14 3/8" +/- 3/8" (36.5 cm \pm 1 cm)
Weight:	51 lbs. +/- 1 lb (23.2 kg +/- 0.5 kg)
Shipping Weight:	58 lbs. +/- 1 lb (26.3 kg +/- 0.5 kg)

SPECIFICATIONS

SPECIFICATIONS (CONTINUED)

Operating Ambient Temperature:	50°F - 95°F (10°C - 35°C) at 20-60% relative humidity
Exhaust:	Less than Ambient + 35°F (+19°C).
Oxygen Output:	Less than Ambient + 4°F (+2°C).
Cabinet:	Impact Resistant flame-retardant plastic cabinet that conforms to UL 94-V0.
Regulatory Listing:	DOUBLE INSULATED PRODUCT ETL certified complying with UL 1097 ETL certified to CSA C22.2 No. 68
Electrical:	No extension cords.
Placement:	No closer than 3-inches from any wall, furniture, draperies, or similar surfaces.
Tubing:	Crush-Proof Tubing - 7 ft. (2.1m) cannula with a maximum 50 ft. (15.2 m) extension (DO NOT PINCH) .
Relative Humidity:	20 to 60%
Time of Operation:	Up to 24 hours per day.
Recommended Storage and Shipping Temperature:	-20°F to 150°F (-29°C to 65°C), at 15-95% relative humidity

HANDLING

The concentrator should always be kept in the upright position to prevent cabinet damage while being transported. The shipping container has been designed to assure maximum protection of the concentrator. If the concentrator is to be reshipped by common carrier, additional cartons are available from Invacare.

UNPACKING

1. Check for any obvious damage to the carton or its contents. If damage is evident, notify the carrier, or your local dealer.
2. Remove all loose packing from the carton.
3. Carefully remove all the components from the carton.

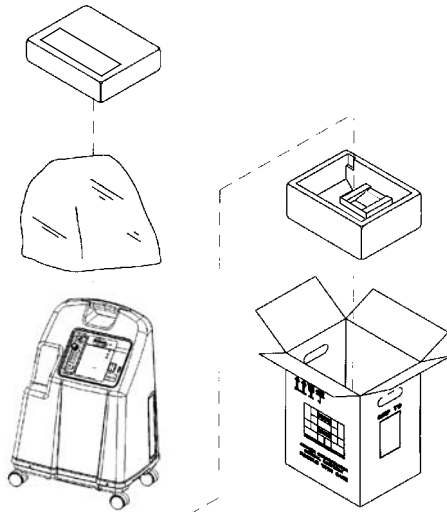
FIGURE I - PACKAGING

NOTE: Unless the oxygen concentrator is to be used immediately, retain containers and packing materials for storage until use of the concentrator is required.

INSPECTION

1. Examine exterior of the oxygen concentrator for nicks, dents, scratches or other damages. Inspect all components.

STORAGE

1. Store the repackaged oxygen concentrator in a dry area.
2. **DO NOT** place other objects on top of the repackaged concentrator.

INTRODUCTION

Your oxygen concentrator is intended for individual use in the home. It is an electronically operated device that separates oxygen from room air. It provides high concentration of oxygen directly to you through a nasal cannula. Clinical studies have documented that oxygen concentrators are therapeutically equivalent to other types of oxygen delivery systems.

Your provider will show you how to use your oxygen concentrator. He/She should be contacted with any questions or problems regarding your oxygen concentrator. This owner's manual will tell you about your concentrator and will serve as a reference as you use your concentrator.

OPERATING INSTRUCTIONS

SELECT A LOCATION

You may select a room in your house where using your oxygen concentrator would be most convenient. Your concentrator can be easily rolled from room to room on its casters.

Your oxygen concentrator will perform best when operated under the following conditions. Usage in environments other than those described may result in the need for increased equipment maintenance. The air intake of the unit should be located in a well ventilated area to avoid airborne pollutants and/or fumes.

RECOMMENDED END-USER GUIDELINES FOR OPTIMUM PERFORMANCE:

Temperature:	50°F - 95°F (10°C - 35°C)
Electrical:	No extension cords.
Placement:	No closer than 3-inches from the wall, furniture, draperies, or similar surfaces.
Tubing and Cannula:	7 ft. cannula with a maximum 50 ft. of Crush-Proof Tubing (DO NOT PINCH).
Environment:	Smoke and soot-free. No confined spaces (Example: NO closets).
Relative Humidity:	20 to 60%
Time of Operation:	Up to 24 hours per day.
Flow Rate	From 1/2 L/min. to 5 L/min. For flow rates less than 1 L/min., use the Pediatric Flowmeter Accessory.
Minimum Operating Time:	30 Minutes

OPERATING INSTRUCTIONS

NOTE: Ensure that your concentrator is at least (3) three-inches away from walls, draperies or furniture to assure sufficient air flow. Avoid deep pile carpets and heaters, radiators or hot air registers.

PLUG IN POWER CORD

1. Plug in power cord to an electrical outlet.

CONNECT HUMIDIFIER (IF SO PRESCRIBED)

1. Fill humidifier with distilled water to the level indicated by the manufacturer. (Recommended humidifiers: Invacare 3260, 003-40, or 006-40).

WARNING

DO NOT overfill humidifier.

Replace the humidifier cap and securely tighten.

DO NOT reverse the oxygen input and output connections.

Water from the humidifier bottle WILL travel through the cannula back to the patient.

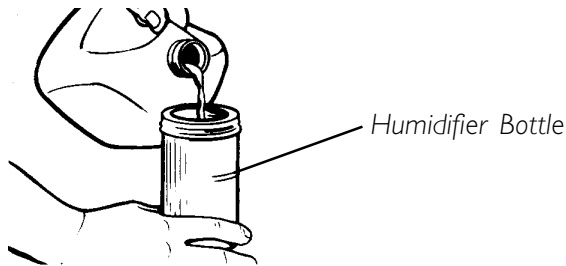
FIGURE 2 - FILLING THE HUMIDIFIER

2. Place the humidifier in the humidifier compartment.


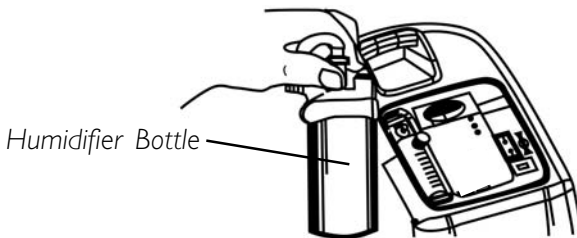
FIGURE 3 - HUMIDIFIER COMPARTMENT


OPERATING INSTRUCTIONS

3. Remove the Filter Access Door located on the side of the concentrator. Insert a flathead screwdriver in the plate groove on the top edge of the Filter Access Door and gently pry the Filter Access Door open (**FIGURE 4, DETAIL "A"**).
4. The humidifier adapter is next to the inlet filter. Pull up and remove the humidifier bottle adapter (**FIGURE 4, DETAIL "B"**).
5. Attach it to the humidifier by turning the wing nut on the humidifier bottle counterclockwise until it is securely fastened (**FIGURE 4, DETAIL "C"**).

DETAIL "A"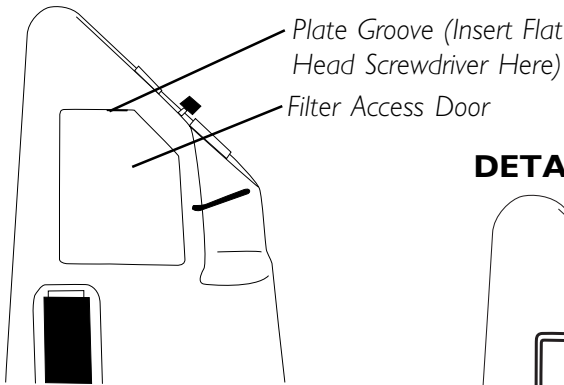

DETAIL "B"

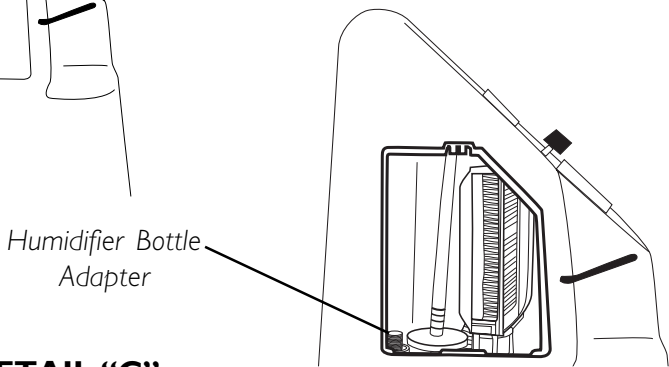DETAIL "C"

FIGURE 4 - ATTACHING THE HUMIDIFIER BOTTLE ADAPTER

OPERATING INSTRUCTIONS

6. Attach oxygen tubing from the humidifier bottle to oxygen outlet connector on the oxygen concentrator.

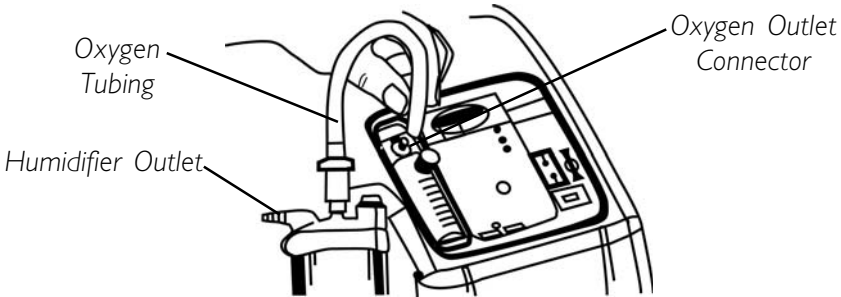
FIGURE 5 - ATTACHING THE OXYGEN TUBING

7. Attach the cannula/patient supply tubing to the humidifier bottle outlet.
8. After assembly, ensure that oxygen is flowing through the cannula.

WARNING

For optimum performance, Invacare recommends that concentrator be operated a minimum of 30 minutes during any use. Shorter periods of operation may adversely affect the life of the product.

POWER SWITCH

1. Press power switch to the **ON** position. All the panel lights and the audible alarm will come on for one second, indicating that the unit is functioning properly. After one second, the only green **SYSTEM OK/POWER** light will stay on.

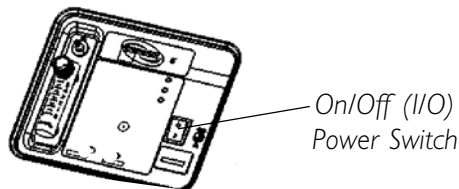
FIGURE 6 - POWER SWITCH

FLOWRATE

1. Turn the flowrate knob to the setting prescribed by your physician or therapist.


OPERATING INSTRUCTIONS

WARNING

DO NOT change the L/min. setting on the flowmeter unless a change has been prescribed by your physician or therapist.

NOTE: To properly read the flowmeter, locate the prescribed flowrate line on the flowmeter. Next, turn the flow knob until the ball rises to the line. Now, center the ball on the L/min. line prescribed (FIGURE 7).

2. If the flowrate on the flowmeter ever falls below 0.5L/min. for more than about one minute, the **LOW FLOW ALARM** will be triggered. This is a rapid beeping of the audible alarm. Check your tubing or accessories for blocked or kinked tubing or a defective humidifier bottle. After rated flow is restored, more than 0.5 L/min., the LOW FLOW audible alarm will go off.

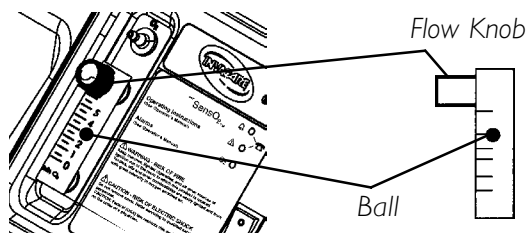**FIGURE 7 - ADJUSTING THE FLOWRATE-
PLATINUM 5 WITH SENSO₂ SHOWN**

NOTE: The use of some accessories such as the pediatric flowstand and the Homefill II compressor will deactivate the Low Flow Alarm.

SENSO₂ OXYGEN PURITY INDICATOR (IF YOUR UNIT FEATURES THE O₂ SENSOR FEATURE)

This feature monitors the purity of oxygen generated by the oxygen concentrator. If purity falls below factory preset standards, indicator lights on the control panel will illuminate.

INITIAL STARTUP OF THE CONCENTRATOR

NOTE: Concentrator may be used during the initial start warm-up time (approximately 30 min.) while waiting for the O₂ purity to reach maximum.

When the unit is turned on, the green light will come on (SYSTEM OK / O₂ greater than 85%). After 5 minutes, the oxygen sensor will be operating normally and will control the indicator lights depending on oxygen concentration values. The explanation of the indicator light functions are as follows.

EXPLANATION OF OXYGEN PURITY INDICATOR LIGHTS (FIGURE 8):

GREEN LIGHT (O₂) - Normal Operation

YELLOW LIGHT (⚠) - Call Supplier **IMMEDIATELY**. You may continue to use the concentrator unless instructed otherwise by your supplier. Be certain that **BACKUP OXYGEN** is nearby.

RED LIGHT (🔔) - Total unit shutdown. Switch **IMMEDIATELY** to **BACKUP OXYGEN** supply. Call Supplier **IMMEDIATELY**.

GREEN LIGHT - with **YELLOW LIGHT FLASHING** - Call Supplier **IMMEDIATELY**. Oxygen sensor malfunctioning; you may continue to use the concentrator.

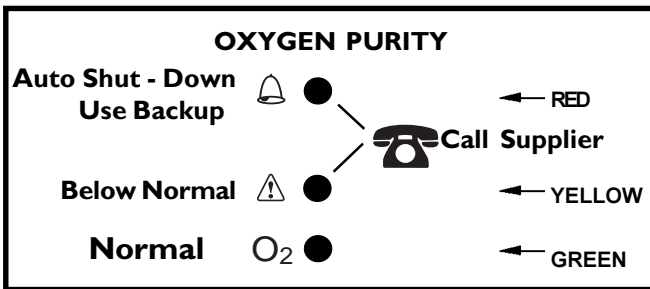
FIGURE 8 - O₂ DISPLAY CONTROL PANEL

INDICATORS- IF YOUR UNIT DOES NOT FEATURE THE O₂ SENSOR

LX MODELS INDICATOR LIGHT EXPLANATION (FIGURE 9):

RED LIGHT (🔔) - Total Unit Shut-Down. Switch **IMMEDIATELY** to a **BACK-UP OXYGEN** supply. Call Supplier **IMMEDIATELY**.

GREEN LIGHT (I/O) - On/Off. System okay.

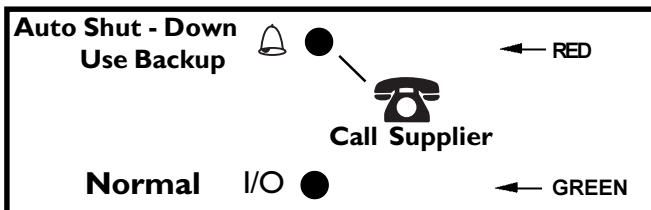
FIGURE 9 - DISPLAY CONTROL PANEL

MAINTENANCE

CONCENTRATOR MAINTENANCE

NOTE: The Invacare Concentrators are specifically designed to minimize routine preventive maintenance at intervals of once per year. In places with high dust or soot levels, maintenance may need to be performed more often. The following must be performed at a minimum of one (1) year in service to assure years of additional reliability. Only authorized or factory-trained personnel should perform preventive maintenance on the concentrator. Power should be disconnected before beginning preventive maintenance on the Invacare concentrator.

CLEANING THE CABINET FILTERS

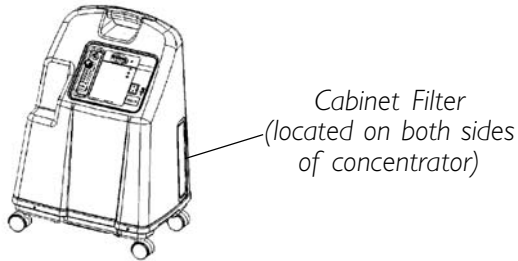
FIGURE 1 - CABINET FILTER LOCATION

NOTE: There are two (2) cabinet filters. One (1) is located on each side of the cabinet (FIGURE 1).

1. Remove each filter and **CLEAN AT LEAST ONCE A WEEK** depending on environmental conditions (**FIGURE 2**).

NOTE: Environmental conditions that may require more frequent cleaning of the filters include but are not limited to: high dust, smoking, air pollutants, etc.

2. Clean the cabinet filters with a vacuum cleaner or wash in warm soapy water and rinse thoroughly (**FIGURE 2**).
3. Dry the filters thoroughly before reinstallation.

CAUTION

DO NOT operate the concentrator without the filters installed.

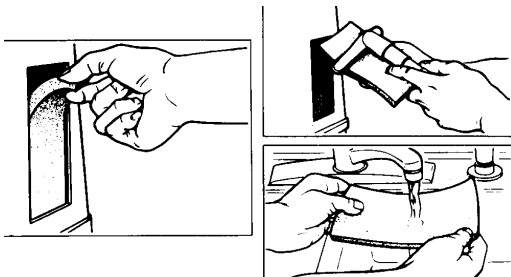
FIGURE 2 - CLEANING THE CABINET FILTERS

CABINET

WARNING

UNPLUG the concentrator when cleaning. To avoid electrical shock, **DO NOT** remove cabinet.

1. Clean the cabinet with a mild household cleaner and non-abrasive cloth or sponge.

HUMIDIFIER

NOTE: To clean the oxygen humidifier, follow the instructions provided by the manufacturer. If none are provided, follow these steps:

1. Clean the humidifier every day.
2. Wash it in soapy water and rinse it with a solution of ten (10) parts water and one (1) part vinegar.
3. Rinse thoroughly with hot water and refill with distilled water to the level shown on the humidifier.

WARNING

DO NOT overfill Humidifier Bottle.

CONCENTRATOR TROUBLESHOOTING GUIDE

SYMPTOM:	PROBABLE CAUSE:	SOLUTION:
<p>Alarm: SHORT BEEPS, LONG PAUSE</p> <p>Concentrator NOT operating, power switch ON.</p> <p>Beep.....Beep.....</p>	<p>Main Power Loss:</p> <ol style="list-style-type: none"> 1. Power cord NOT plugged In. 2. No power at outlet. 3. Tripped circuit breaker. 	<ol style="list-style-type: none"> 1. Insert plug into outlet. 2. Inspect house circuit breakers or fuses. If problem recurs, use a different outlet. 3. Call Service Provider.
<p>Alarm: CONTINUOUS</p> <p>Concentrator NOT operating, Power Switch ON.</p> <p>Beeeeeeep....</p>	<p>System Failure:</p> <ol style="list-style-type: none"> 1. Unit overheating due to blocked air intake. 2. Insufficient power at outlet. 3. Internal repairs required 	<ol style="list-style-type: none"> 1a. Remove and clean cabinet filters. 1b. Move oxygen concentrator at least three (3) inches away from walls, draperies or furniture. 2. Do not use extension cords. Move to another electrical outlet or circuit. 3. Call Service Provider.

TROUBLESHOOTING

SYMPTOM:	PROBABLE CAUSE:	SOLUTION:
<p>YELLOW or RED LIGHT Illuminated.</p> <p>Alarm: CONTINUOUS On red light only.</p> <p>* Only applies to SensO₂ models.</p>	<ol style="list-style-type: none"> 1. Low oxygen purity.* 2. Kinked or blocked tubing, cannula or humidifier.* 3. Flowmeter set at 1/2 L/min. or less.* 4. Unit overheating due to blocked air intake. 5. Insufficient power at outlet. 6. Internal repairs required. 	<ol style="list-style-type: none"> 1. Clean or Replace filters. 2. Inspect for kinks or blockages. Correct, clean or replace item. Once corrected, turn power OFF for 60 seconds and then turn power back ON. 3. Check flowmeter is set to 1.0 L/min. or more. Refer to Recommended End-User Guidelines in the Operating Instructions of this manual. 4.a. Remove and clean cabinet filters. 4b. Move concentrator at least 3- inches from walls, draperies, and furniture. 5. Do not use extension cords. Move to another electrical outlet or circuit. 6. Call Service Provider.
<p>GREEN LIGHT with YELLOW Light Flashing.</p>	<ol style="list-style-type: none"> 1. Internal repairs required. 	<ol style="list-style-type: none"> 1. Call Service Provider.
<p>Alarm: RAPID</p> <p>Beep....Beep... Beep...Beep</p>	<p>LOW FLOW ALARM</p> <ol style="list-style-type: none"> 1a. Kinked or blocked tubing, cannula or humidifier. 1b. Flowmeter set at 0.5 L/min. or less. 	<ol style="list-style-type: none"> 1a. Inspect for kinks or blockages. Correct, clean or replace item. Once corrected, turn power OFF for 60 seconds and then turn power back ON. 1b. Reset flowmeter to prescribed flowrate. If prescribed flowrate is below 0.5 L/min., use the Pediatric Flowmeter IRCPF16.

**PREVENTIVE MAINTENANCE RECORD
INVACARE PLATINUM OXYGEN CONCENTRATOR**

MODEL NO. IRC _____ SERIAL NO. _____

Yes, you can:

ON EACH INSPECTION

Record Date of Service																				
Record Elapsed Hours On Hour Meter																				
Clean Cabinet Filter(s)																				
Check Prescribed L/min. Flow Rate																				
EVERY 90 DAYS																				
Check Oxygen Concentration (UNITS WITHOUT SensO ₂)																				

ANNUALLY OR DURING PREVENTIVE MAINTENANCE SCHEDULE

Clean/Replace Cabinet Filter(s)																				
Check Outlet HEPA Filter**																				
Check Compressor Inlet Filter**																				
Check Oxygen Concentration (SensO ₂ MODELS)																				
Check Power Loss Alarm																				
AS REQUIRED																				
Clean Heat Exchanger																				
Replace Exhaust Muffler																				
Rebuild Top End of Compressor																				

*Refer to Preventive Maintenance section of Service Manual.

LIMITED WARRANTY

NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to original purchaser of our products. This warranty gives you specific legal rights and you may also have other legal rights which may vary from state to state.

Invacare Corporation warrants its Platinum Oxygen Concentrator to be free from defects in materials and workmanship from date of purchase for a period of five (5) years

The 4-way valve body carries a lifetime warranty from date of purchase. **DO NOT OPEN OR ATTEMPT TO SERVICE VALVE BODY.** This will void any and all warranty applicable to the valve body. If within such warranty period any such product shall be proven to Invacare Corporation's satisfaction to be defective, such product shall be repaired or replaced, at Invacare Corporation's option. This warranty only applies to the labor for repairs performed by the Invacare Service Department or authorized Invacare Authorized Service Centers. It does not apply to the labor performed by the purchaser or user. This warranty does not include normal wear and tear or shipping charges. Invacare Corporation's sole obligation and your exclusive remedy under this warranty shall be limited to such repair or replacement. Routine maintenance items, such as filters, are excluded from this warranty.

For warranty service, please contact Invacare Corporation's service department at the toll free number on the back cover.

Upon receiving notice of an alleged defect in a product, Invacare Corporation will issue a serialized return authorization. It shall then be the responsibility of the purchaser to return the entire unit or remove, at purchaser's cost, the defective component part(s) identified, pack the component part(s) in a manner to avoid shipping damage and to ship the component part(s) to either Invacare Corporation's plant or service center as specified by Invacare Corporation in advance. Defective component part(s) must be returned for warranty inspection using the serial number as identification within thirty (30) days of return authorization date. Do not return products to our factory without prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, SOOT OR SMOKE-FILLED ENVIRONMENTS, OR OTHER THAN NORMAL APPLICATION, USE OR SERVICE, OR TO PRODUCTS MODIFIED WITHOUT INVACARE CORPORATION'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS) OR TO PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE CORPORATION OR TO PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE CORPORATION'S CONTROL.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Some states do not allow the exclusion or limitation of incidental or consequential damage, or limitation of how long an implied warranty lasts, so the above exclusion and limitation may not apply to you.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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Canada

570 Matheson Blvd E Unit 8
Mississauga Ontario
L4Z 4G4 Canada
800-668-5324

Invacare, SensO₂, PreciseR_x and
Venture are registered trademarks
of Invacare Corporation.
Homefill, Sieve-Gard, and Yes, you can.
are trademarks of Invacare
Corporation.
© 2003 Invacare Corporation
Part No. 1118353 Rev A 01/03
Asm. No. 1106664

OPERATOR'S MANUAL - ENGLISH
MANUAL DE INSTRUCCIONES -
DOMESTIC SPANISH

PLATINUM 5
HFII

OXYGEN CONCENTRATORS STANDARD,
WITH /SENSO₂[®] AND HOMEFILL™ II
CONCENTRADORES DE OXÍGENO
STANDARD, CON /SENSO₂[®] Y HOMEFILL™ II

PART TWO

HOMEFILL II
CONNECTION AND USE WITH
CONCENTRATOR

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USER: BEFORE USING THIS CONCENTRATOR WITH HOMEFILL II, READ THIS MANUAL AND SAVE FOR FUTURE REFERENCE

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**CONCENTRATOR AND
HOMEFILL II SET-UP AND OPERATION**
TABLE OF CONTENTS
PART TWO

SPECIAL NOTES	2
HOMEFILL INFORMATION	4
PROCEDURE 1	
GENERAL GUIDELINES	5
General Warnings	5
Handling Warnings	6
PROCEDURE 2	
INITIAL SETUP	7
Compressor Initial Setup Checklist	7
Choosing a Location	7
Transporting the Compressor	8
Installing the Power Cord	9
Connecting the Compressor to the Concentrator	9
PROCEDURE 3	
COMPRESSOR OPERATION	11
Compressor Operation Checklist	11
Cylinder Prefill Inspection	11
Connecting the Cylinder to the Compressor	14
Turning the Compressor On	16
Removing the Cylinder	18
Setting the Flow Knob to the Prescribed Setting	19
Concentrator Warranty	20
SPANISH OWNER'S MANUAL (MANUAL DEL USARIO)	22
CONCENTRATOR INFORMATION	PART ONE

HOMEFILL II USE, MAINTENANCE AND CARE

NOTICE

**REFER TO COMPRESSOR OWNERS MANUAL
1100873 FOR DETAILED INFORMATION ON ALL
WARNINGS, USE, CARE AND MAINTENANCE OF
THE HOMEFILL II COMPRESSOR.**

HOMEFILL INFORMATION

Homefill II is an accessory device to the oxygen concentrator that enables the patient to refill custom portable oxygen cylinders for personal use.

WARNING

DO NOT INSTALL, ASSEMBLE OR OPERATE THE HOMEFILL II COMPRESSOR WITHOUT FIRST READING AND UNDERSTANDING THE COMPRESSOR OWNER'S MANUAL, THE CONCENTRATOR OWNER'S MANUAL, AND THE OXYGEN CYLINDER WARNING LABEL. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS AND INSTRUCTIONS, CONTACT A DEALER OR TECHNICAL PERSONNEL IF APPLICABLE BEFORE ATTEMPTING TO INSTALL THIS EQUIPMENT - OTHERWISE, INJURY OR DAMAGE MAY OCCUR.


Check all parts for shipping damage. In case of damage, DO NOT use. Contact Invacare for further instruction.

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NOTICE

THE VENTURE HOMEFILL II COMPRESSOR IS FOR USE WITH SPECIALLY DESIGNED INVACARE 5 LITER HOMEFILL II COMPATIBLE CONCENTRATORS SET AT 3 L/MIN OR LESS ONLY. TO DETERMINE IF YOUR INVACARE 5 LITER CONCENTRATOR IS HOMEFILL II COMPATIBLE, EXAMINE THE LOWER RIGHT CORNER OF THE REAR OF THE CONCENTRATOR FOR AN OUTLET FITTING, AS SHOWN BELOW.

REAR VIEW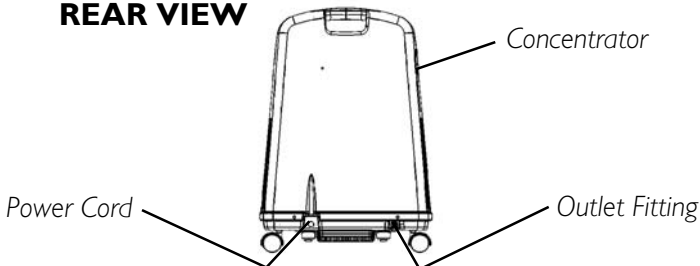

This Procedure includes the following:

General Warnings

Handling Warnings

NOTICE

REFER TO COMPRESSOR OWNERS MANUAL I100873 FOR DETAILED INFORMATION ON ALL WARNINGS, USE, CARE AND MAINTENANCE OF THE HOMEFILL II COMPRESSOR.

In order to ensure the safe installation, assembly and operation of the **VENTURE HOMEFILL II COMPRESSOR** these instructions **MUST** be followed:

GENERAL WARNINGS

“Caution: Federal law restricts this device to sale by or on order of a physician, or any other practitioner licensed by the law of the State in which he/she practices to use or order the use of this device.”

The use of this device is limited to the oxygen patient. Cylinders **MUST** be used **ONLY** by the oxygen patient and are not to be distributed to any other individual for any purpose.

DO NOT use parts, accessories or adapters other than those authorized by Invacare or warranty will be voided.

The maximum patient flow while filling cylinders with the IOH200 is 3 L/min. Patients with a higher perscribed flow should not transfill while receiving supplemental oxygen treatment from the Platinum 5 Concentrator.

NEVER block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air opening may be blocked. Keep the openings free from lint, hair and the like.

DO NOT disassemble. There are minimal serviceable parts on the Venture HomeFill II Compressor. If any service is required, contact Invacare technical service.

DO NOT allow oil from your hands or other source to come into contact with the regulator or cylinder valve connection. These solutions may be flammable and cause injury.

GENERAL WARNINGS (CONTINUED)

The HomeFill II is equipped with a High Pressure Relief Valve to ensure the user's safety. When activated, this safety feature will make an extremely loud noise. If this noise occurs, turn the unit OFF and contact your Home Health Care Provider or Service Representative.

Before moving or repositioning either the compressor or concentrator, always disconnect the AC power cords and the interconnect hose between the compressor and concentrator. Failure to do so may result in damage to the unit or personal injury.

Children should always be supervised around the HomeFill unit. When used with the HomeFill table or similar elevated support surfaces, a child could pull over any such table causing the unit to fall. Failure to observe this warning may result in personal injury or product damage.

HANDLING WARNINGS

Use extreme care when handling and filling an oxygen cylinder. Full oxygen cylinders are under pressure and can become a projectile if dropped or mishandled.

NEVER transport a compressor with a cylinder connected to or stored on the compressor, otherwise injury or damage can occur.

DO NOT store filled Oxygen bottles near a furnace, hot water tank, in the trunk of a car or other high temperature area. Storage in areas such as this can result in bursting of the bottle or fire.

This Procedure includes the following:

Compressor Initial Setup Checklist

Choosing a Location

Transporting the Compressor

Installing the Power Cord

Connecting the Compressor to the Concentrator

NOTE: DEALERS ONLY - Refer to DEALER INFORMATION in owner's manual part number, 1100873, BEFORE performing this procedure.

COMPRESSOR INITIAL SETUP CHECKLIST

Before operating the HomeFill II Compressor for the first time, complete the following checklist:

- Choose a location for the compressor.
- Transport the compressor to the desired location.
- Connect the power cord to the compressor.
- Connect the compressor to the concentrator.

CHOOSING A LOCATION

The compressor should be at least three (3) inches away from walls, draperies or furniture to ensure sufficient air flow.

Avoid placing the compressor next to heaters, radiators or hot air registers.

DO NOT use in a closet.

The compressor should sit on a hard flat stable surface, such as a table.

TRANSPORTING THE COMPRESSOR (FIGURE I)

WARNING

NEVER transport a compressor with a cylinder connected to or stored on the compressor, otherwise injury or damage can occur.

CAUTION

When transporting the compressor, be careful not to bump the compressor or the connectors into obstacles. **Otherwise** damage to the compressor may occur.

There are two (2) methods to safely transport the compressor:

- A. ONE HAND - Grasp the handle at the bottom rear of the compressor.
- B. TWO HANDS - Grasp the hand grips on the sides of the compressor.

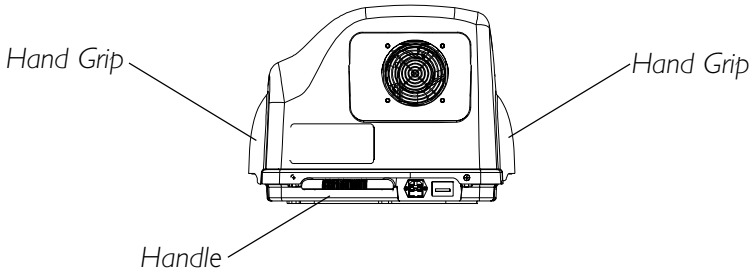
FIGURE I - TRANSPORTING THE COMPRESSOR

INSTALLING THE POWER CORD (FIGURE 2)

1. Ensure the compressor is on a hard flat stable surface, such as a table.
2. Plug the power cord into the electric connector in the back of the compressor.
3. Plug the opposite end of the power cord into the wall outlet.

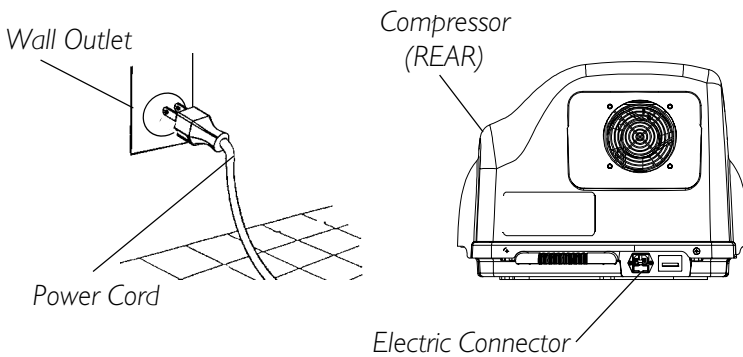
FIGURE 2 - INSTALLING THE POWER CORD

CONNECTING THE COMPRESSOR TO THE CONCENTRATOR (FIGURE 3)

1. Insert one (1) end of the interconnect hose into the outlet fitting on the back of the concentrator.
2. Insert the opposite end of the interconnect hose into the inlet fitting on the side of the compressor.

NOTE: The metal tabs on the concentrator outlet fitting and the compressor inlet fitting will pop out with an audible “click” when the end of the interconnect hose is properly installed.

3. Loop any excess interconnect hose and secure to the back of the concentrator with the fastening straps.

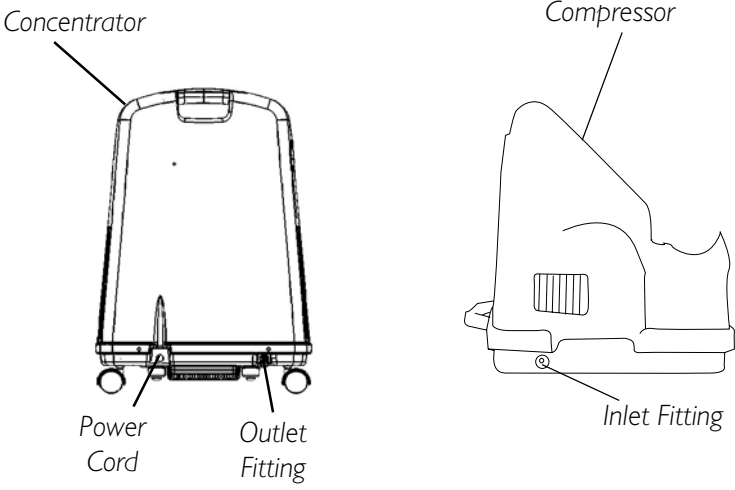
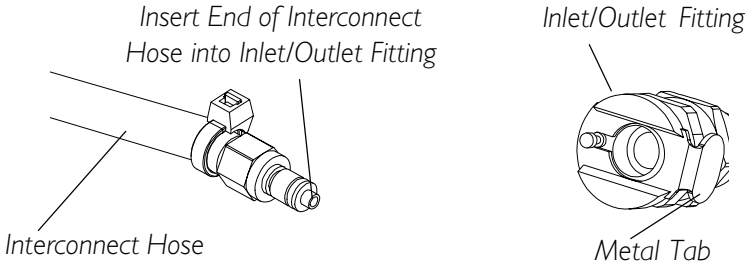
FIGURE 3 - CONNECTING THE COMPRESSOR TO THE CONCENTRATOR

This Procedure includes the following:

Compressor Operation Checklist

Cylinder Prefill Inspection

Connecting the Cylinder to the Compressor

Turning the Compressor On

Removing the Cylinder

Setting the Flow Knob to the Prescribed Setting

COMPRESSOR OPERATION CHECKLIST

Each time the HomeFill II Compressor is used to fill a cylinder, complete the following checklist:

NOTE: All steps mentioned below are in this procedure, unless otherwise noted.

- Ensure the concentrator has been on for at least thirty (30) minutes. Refer to the concentrator owner’s manual, Part One.
- Perform the prefill inspection on the cylinder.
- Connect the cylinder to the compressor.
- Turn the compressor on.
- Examine the indicator lights on the control panel.
- Remove the full cylinder.
- Turn the compressor off.
- If filling another cylinder, repeat this checklist.

**CYLINDER PREFILL INSPECTION
(FIGURE 1)**

WARNING

ONLY use cylinders that have the Invacare Venture HomeFill II connection and the label shown in **FIGURE 1**.

All cylinders must be inspected before attempting to fill: otherwise, injury or damage may occur.

THE HYDROSTATIC TESTING DATE

Aluminum cylinders must undergo testing every five (5) years.

NOTE: DO NOT fill cylinders that have not been tested in the past five (5) years. Contact your dealer for replacement.

EXTERNAL EXAMINATION

1. Examine the outside of the cylinder for the following conditions, and replace the cylinder if they exist:
 - A. Dents or dings
 - B. Arc Burns
 - C. Oil or Grease
 - D. Any other signs of damage that might cause a cylinder to be unacceptable or unsafe for use.
2. Examine the cylinder for evidence of fire or thermal damage. Evidence includes charring or blistering of the paint, or other protective coating or heat sensitive indicator. If fire or thermal damage is found, replace the cylinder.
3. Inspect the Invacare valve for the following, and replace if found:
 - A. Debris, oil or grease
 - B. Noticeable signs of damage
 - C. Signs of corrosion inside the valve
 - D. Signs of excessive heat or fire damage

THIS CYLINDER IS TO BE USED WITH THE INVACARE VENTURE HOMEFILL SYSTEM. THIS CYLINDER CONTAINS 93±3% OXYGEN PRODUCED BY AN OXYGEN CONCENTRATOR USING THE PRESSURE SWING ADSORPTION PROCESS.

OXYGEN CYLINDERS FILLED BY THE VENTURE HOMEFILL SHALL BE USED FOR PERSONAL USE ONLY. "NOT TO BE FILLED FOR RESALE OR USE BY PROFESSIONAL USERS"

WARNING: HIGH PRESSURE OXIDIZING GAS VIGOROUSLY ACCELERATES COMBUSTION.

CAUTION: For medical applications. Use only as directed by a licensed practitioner. Uninterrupted use of high concentrations of oxygen over a long duration, without monitoring its effect on oxygen content of arterial blood, may be harmful.

CAUTION: DO NOT HANDLE CYLINDER OR USE CONTENTS UNTIL YOU ARE TRAINED TO USE CYLINDER AND CONTENTS, INCLUDING EMERGENCY PROCEDURES.

CAUTION: IMPROPER FILLING OR ABUSE OF THIS CYLINDER, OR FAILURE TO HEED THIS CAUTION MAY CAUSE SERIOUS INJURY OR LOSS OF LIFE. DO NOT ALTER OR MODIFY CYLINDER OR RELATED COMPONENTS. KEEP OUT OF THE REACH OF CHILDREN. Secure cylinder during storage and use. No smoking in cylinder area. Open valve slowly. Close valve after each use and when empty. Use in accordance with Venture HomeFill Operator's Manual. Cylinder temperature should not exceed 125°F (52°C). Keep away from heat, flame and spark. Do not drop.

"Use No Oil or Grease". KEEP CYLINDER, CYLINDER VALVE, ALL EQUIPMENT AND CONNECTIONS FREE OF OIL AND GREASE TO AVOID VIOLENT IGNITION.

TRANSFILLING OF THIS GAS IS PERFORMED BY VENTURE HOMEFILL ONLY. Cylinders that have been refinished and/or exposed to elevated temperatures must be hydrostatically tested before filling. Do not use caustic paint strippers. Aluminum cylinders subject to the action of fire or heated to temperatures in excess of 350°F must be withdrawn from service and condemned by trained personnel. Valve and safety relief devices must be removed and replaced by trained and authorized personnel. Do not alter or change valve assembly.

DO NOT REMOVE THIS PRODUCT LABEL
Not for Commercial Resale

Invacare Corporation
699 Aero Lane
Sanford, FL 32771
1-800-832-4707 P/N: 1082951

FIGURE I - CYLINDER PREFILL INSPECTION

CONNECTING THE CYLINDER TO THE COMPRESSOR (FIGURE 3)

WARNING

DO NOT modify **ANY** connections on the HomeFill II compressor. **NEVER** use tools of any kind to connect or disconnect the cylinder and the compressor. **Otherwise, severe injury and/or damage may occur.**

NOTE: Refer to the GENERAL WARNINGS and the HANDLING WARNINGS in PROCEDURE 1 of this manual.

1. Examine the cylinder gauge.
 - If the cylinder pressure is **greater** than 1500 psig., **DO NOT** attempt to top off this cylinder. Select another cylinder to fill.
 - If the cylinder pressure is **less** than 1500 psig., proceed to STEP 2.
2. Set the regulator flow knob on the cylinder to “0”.
3. Remove the connector fillport cover (if present).

WARNING


DO NOT drop oxygen cylinders. **Use two (2) hands** when handling/transporting oxygen cylinders. **Otherwise, injury or damage may occur.**

4. Position the cylinder in the compressor cradle.
5. Align the cylinder fillport with the connector fillport.
6. Grasp the connector fillport just below the ring.

NOTE: Your hand should rest against the bottom of the ring during this procedure.

7. Push **UP** on the connector fillport ring while pushing **DOWN** on the cylinder to insert the cylinder fillport into the connector fillport.

NOTE: The cylinder is properly connected to the compressor when an audible “click” is heard.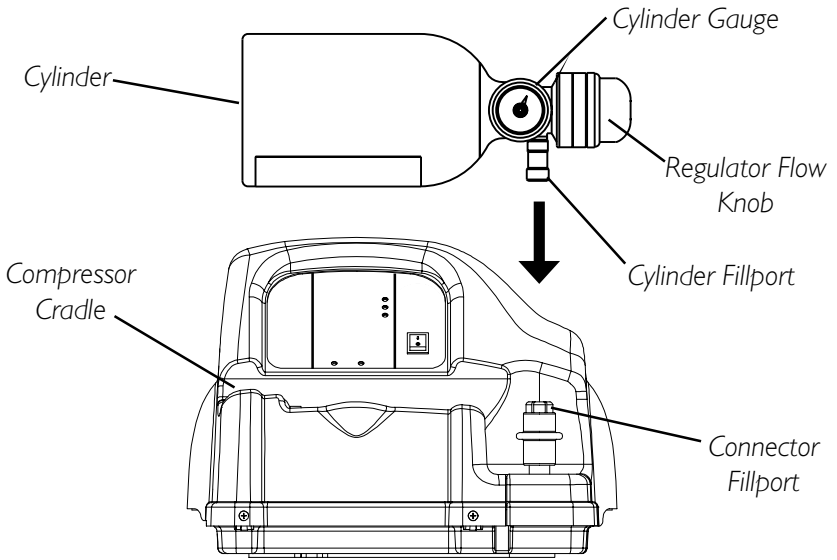

NOTE: Connector fillport cover not shown for clarity.

FIGURE 3 - CONNECTING THE CYLINDER TO THE COMPRESSOR

TURNING THE COMPRESSOR ON (FIGURE 3)

NOTE: Refer to the GENERAL WARNINGS and the HANDLING WARNINGS in the SAFETY SUMMARY of this manual.

1. Make sure the concentrator is on. Refer to the concentrator owner's manual, Part One.

CAUTION

The concentrator flow rate to the patient MUST be at 3 L/min. or less. Otherwise the oxygen level to the compressor will be <90% and the compressor will not fill the cylinder.

2. Set the concentrator flow rate to the patient to 3 L/min. or less. Refer to the concentrator owner's manual, Part One.
3. Make sure the cylinder is connected to the compressor. Refer to CONNECTING THE CYLINDER TO THE COMPRESSOR in this section of the manual.
4. Push the power switch on the control panel to the ON position.
5. Examine the control panel. The following sequence of events should occur:
 - A. **0-3 MINUTES -**
The WAIT (YELLOW) light will be lit while the compressor warms up.
 - B. **AFTER 3 MINUTES -**
The FILLING (GREEN) light will be on while the cylinder is filling.

The FULL (GREEN) light will be on when the cylinder is finished filling. Proceed to STEP 6.

NOTE: The O2 BELOW NORMAL (YELLOW) light may come on if the oxygen level from the concentrator has not yet reached or drops below 90%. The compressor will begin or resume filling when the oxygen level reaches 90%. If light DOES NOT go off within 10 minutes, refer to the TROUBLESHOOTING section of the Homefill II Owner's Manual (Part No. 1100873).

TURNING THE COMPRESSOR ON (CONTINUED)

NOTE: If the ATTENTION (red) light and audible alarm are on. Refer to the TROUBLESHOOTING section of the Homefill II owner's manual (part no. 1100873).

- 6. Remove the full cylinder. Refer to REMOVING THE CYLINDER in this instruction sheet.

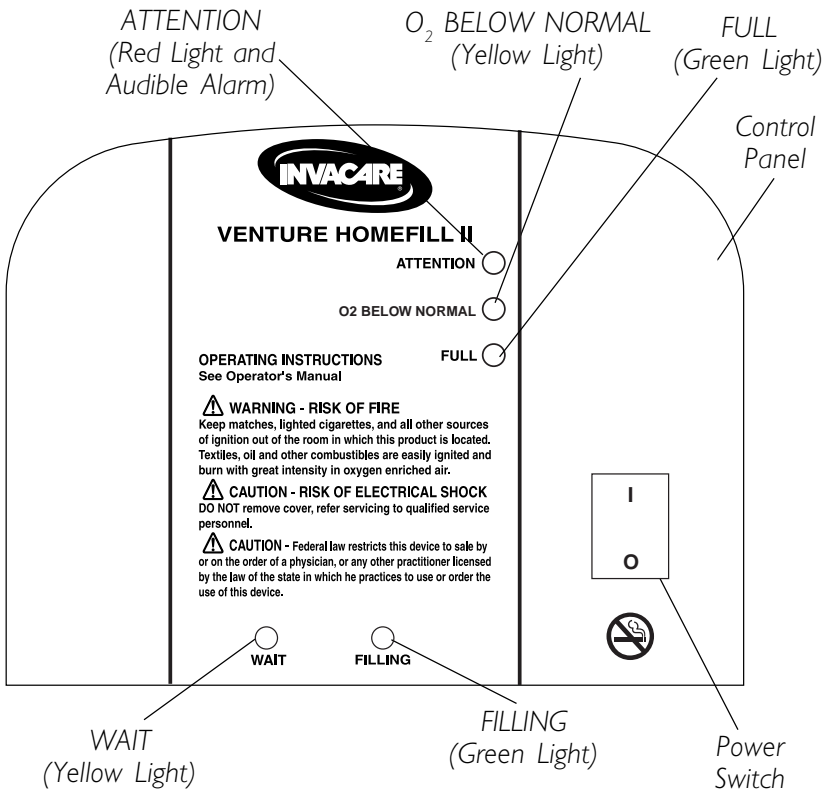
FIGURE 3 - TURNING THE COMPRESSOR ON


REMOVING THE CYLINDER (FIGURE 4)**WARNING**

DO NOT modify **ANY** connections on the HomeFill II compressor. **NEVER** use tools of any kind to connect or disconnect the cylinder and the compressor. Otherwise, severe injury and/or damage may occur.

NOTE: Refer to the GENERAL WARNINGS and the HANDLING WARNINGS in PROCEDURE 1 of this manual.

1. Press the compressor power switch to the OFF position.
2. Grasp the outer ring of the fillport connector and PULL DOWN.
3. Lift up on the cylinder to remove from the fillport connector.
4. Perform one (1) of the following steps:
 - A. If desired, fill another cylinder. Refer to the COMPRESSOR OPERATION CHECKLIST in this instruction sheet.
 - B. Place the fillport connector cover onto the fillport connector.

NOTE: The fillport connector cover should be used whenever cylinders are not being filled.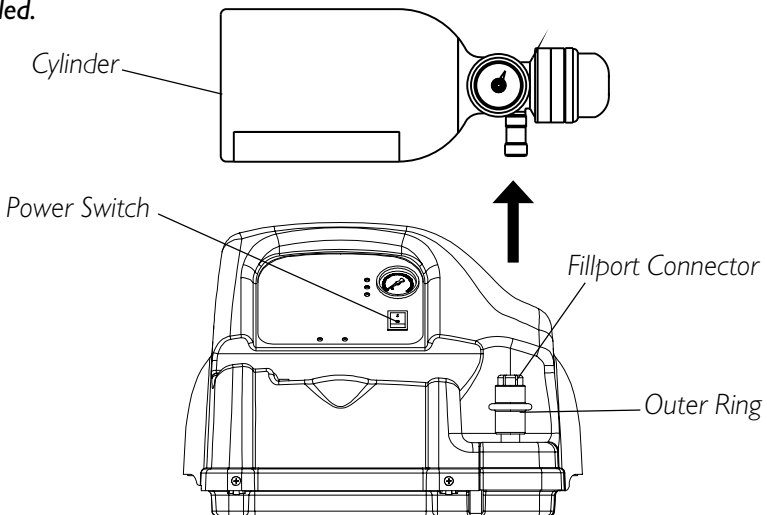

NOTE: Fillport connector cover not shown for clarity.

FIGURE 4 - REMOVING THE CYLINDER

SETTING THE FLOW KNOB TO THE PRESCRIBED SETTING (FIGURE 5)

1. Attach the nasal cannula to the outlet fitting of the cylinder.
2. Turn the flow knob to the L/min. setting prescribed by your physician or therapist.

WARNING

As changing the L/min. setting on the flow knob will affect the dose of oxygen delivered, **DO NOT** readjust the L/min. setting unless directed by your physician or therapist.

3. To turn the flow of oxygen OFF, turn the flow knob counter-clockwise to the "0" (OFF) position.

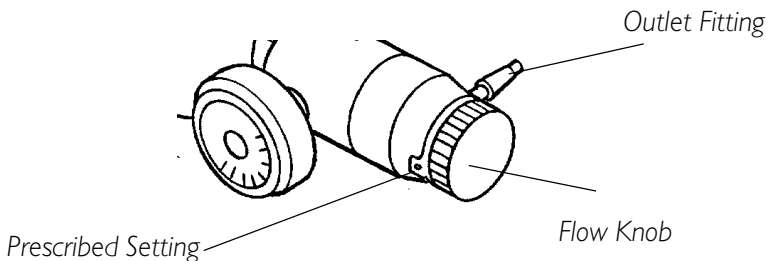
FIGURE 5 - SETTING THE FLOW KNOB TO THE PRESCRIBED SETTING

NOTICE

REFER TO COMPRESSOR OWNERS MANUAL I100873 FOR DETAILED INFORMATION ON ALL WARNINGS, USE, CARE AND MAINTENANCE OF THE HOMEFILL II COMPRESSOR.

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