THE OPUS³⁶⁰ NASAL PILLOWS MASK HAS BEEN VALIDATED TO WITHSTAND 20 CYCLES OF THE ABOVE PARAMETERS. DO NOT USE IF DAMAGED.
 Note: Failure to follow the cleaning instructions may compromise the performance and safety of the mask and shorten its life.
CONTACT YOUR LOCAL HEALTHCARE PROVIDER FOR ADDITIONAL INFORMATION.

HEADGEAR ASSEMBLY
Note: The Headgear Assembly does not need to be disassembled for cleaning.
 1. Lay out the Headgear and Buckles as shown.
 2. Thread the Top Straps (H) through the Top Buckle (K) and fasten back on themselves. Repeat with the Back Straps (L) and Back Buckle (L).
 3. Fold the Side Straps (J) back on themselves.

TECHNICAL SPECIFICATIONS

EXHAUST FLOW
 The Opus³⁶⁰ Nasal Pillows Mask has an air exhaust system that vents the expelled CO₂ through a group of fine holes in the front of the Elbow. It is important that these holes are not blocked by any object. This controlled leak ensures exhaled CO₂ is expelled from the mask.

| PRESSURE (cmH ₂ O) | 3 | 5 | 7 | 9 | 11 | 13 | 15 | 17 | 19 | 21 | 23 | 25 |
|-------------------------------|----|----|----|----|----|----|----|----|----|----|----|----|
| Flow (L/min) | 17 | 22 | 26 | 30 | 33 | 36 | 39 | 42 | 45 | 48 | 50 | 52 |

RESISTANCE TO FLOW

| | MEDIUM |
|--|------------------------|
| Pressure drop through mask @ 50 L/min | 1.2 cmH ₂ O |
| Pressure drop through mask @ 100 L/min | 5.4 cmH ₂ O |

MASK DEAD SPACE
 22cc

OXYGEN/PRESSURE PORT CONNECTOR
If pressure readings and/or additional oxygen are required, an Oxygen/Pressure Port Connector is available (Code 900HC452).

- Connect the large end of the Connector (T) to the Male Swivel (C) on the mask and the small end of the Connector to the Main Breathing Tube (R).
- Open the Port Cap(s) (P) and place the Pressure and/or Oxygen Tube (S) firmly over the Port(s) (Q).

(The two Ports are identical; either Port may be used for oxygen or pressure measurement. Only remove the Port Cap from Ports that are being used.)

- Note:** At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient's breathing pattern, mask selection, and the leak rate.
- Warning:** If oxygen is used with CPAP/Bi-Level machine, the oxygen flow must be turned off when the CPAP/Bi-Level machine is not operating.
Explanation of Warning: When the CPAP/Bi-Level device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP/Bi-Level machine enclosure. Oxygen accumulated in the CPAP/Bi-Level machine enclosure will create a risk of fire.
- Warning:** Do not smoke while using or in the vicinity of someone using oxygen with this mask.
- Warning:** Do not use Polyvinyl Chloride (PVC) Pressure and/or Oxygen Tubing. Use Silicone Tubing instead.
Explanation of Warning: Contact with PVC can cause premature cracking of ports.

OPUS³⁶⁰ NASAL PILLOWS MASK MULTI-PATIENT CLEANING INSTRUCTIONS
 Your Opus³⁶⁰ Nasal Pillows Mask is provided assembled and ready for patient use. To reuse between patients please follow the guidelines below.

CLEANING / DE-SOILING

- Before disinfecting, disassemble all components from the Mask Frame (the Headgear Assembly, Flexitube Assembly, Silicone Pillows and Male Swivel). Clean all surfaces of the mask and components thoroughly with a mild detergent solution and a soft-bristled brush. Ensure all visible soil is removed. Rinse thoroughly under running potable water to remove detergent solution.

FAILURE TO PROPERLY CLEAN THIS DEVICE MAY RESULT IN INADEQUATE DISINFECTION.

HIGH LEVEL DISINFECTION PARAMETERS

FLEXITUBE ASSEMBLY

THERMAL DISINFECTION
 Soak parts in hot-water bath for:
 30 minutes at 70°C (158°F)
 or
 30 minutes at 75°C (167°F)

OR

CIDEX® OPA
 Follow manufacturer's instructions for high level disinfection. Immerse for 30 minutes at 20°C (68°F). Rinse thoroughly in potable water.

OR

STERRAD® 100S
 Follow manufacturer's instructions for Sterrad® 100S Sterilization system.

HEADGEAR

Wash in warm soapy water and leave to dry.

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FISHER & PAYKEL HEALTHCARE LTD