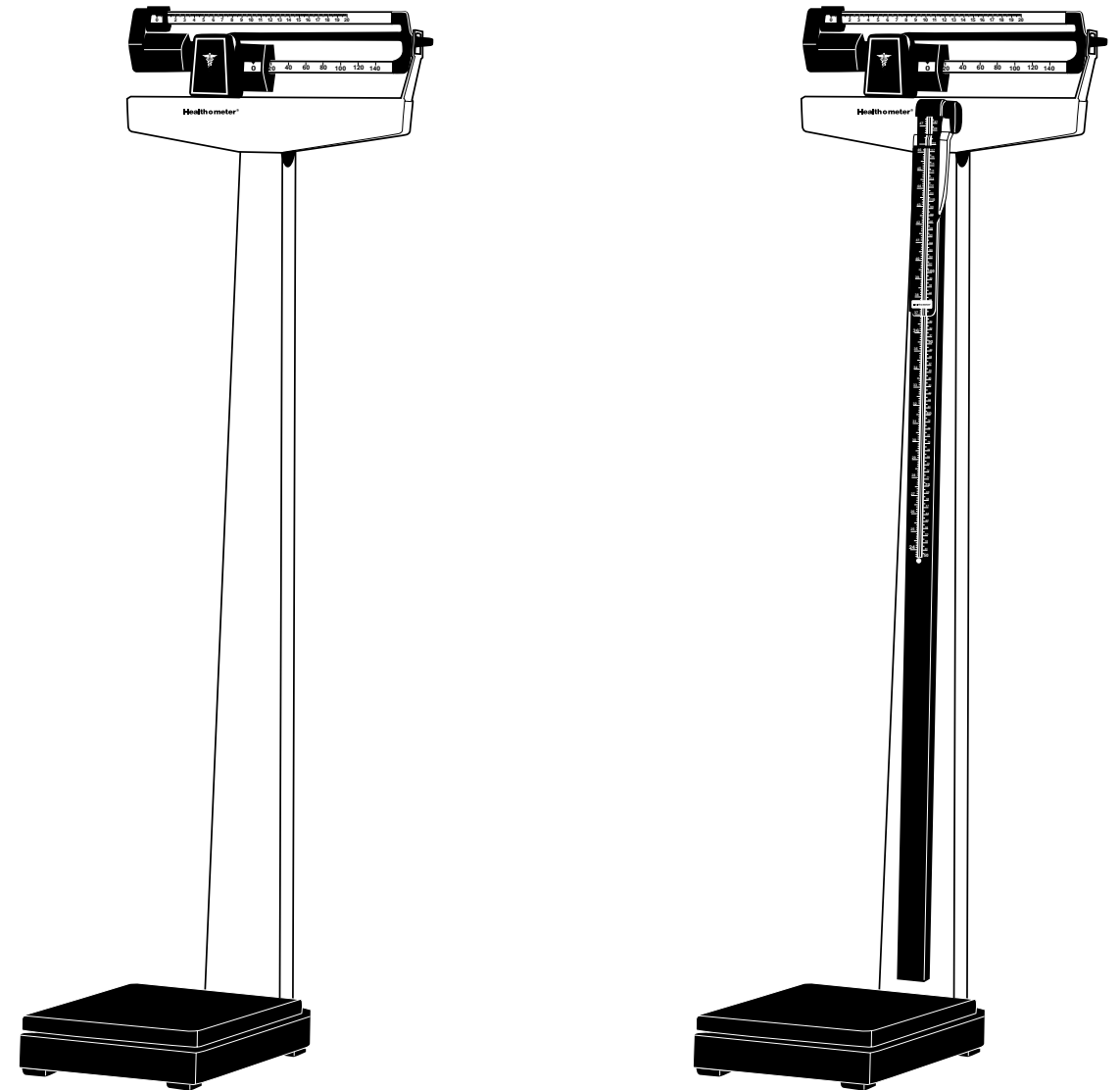
Health o meter®

ASSEMBLY & OPERATING INSTRUCTIONS
INSTRUCCIONES DE INSTALACIÓN Y OPERACIÓN
INSTRUCTIONS DE MONTAGE ET MODE D'EMPLOI

PHYSICIAN BEAM SCALE

MODELS: 400KL, 402KL, 402LB, 402LBCERT, 402KLWH, 402LBWH

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MODÈLES: 400KL, 402KL, 402LB, 402LBCERT, 402KLWH, 402LBWH

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7400 West 100th Place, Bridgeview IL 60455 • 1-800-815-6615

www.healthometermedical.com

Health o meter® is a registered trademark of Sunbeam Products Inc., Boca Raton, FL 33431

Assembly & Operating Instructions

Your scale is a precision instrument and should be handled accordingly. Remove all parts carefully to prevent scratching the enamel finish. In assembling the parts, do not use force since the scale has been assembled at our plant and all parts will fit properly if instructions are followed.

Carton Contents – 1 Pillar/Beam, 1 Base, counterweight(s) (one 100 lb & one 40 kg for 402KL; one 100 lb or 402LB) 4 Pillar Nuts and Wrench

Assembly Instructions

1 Insert studs situated at lower end of pillar into the four holes provided in the base (See Fig 1)

2 Tip scale at angle (See Fig 2) resting top of pillar on any convenient support and, using the wrench provided, secure pillar to base with the four nuts (included).

3 Pull the Steel Rod toward the Base by grasping the Finger Pull Loop on the end.

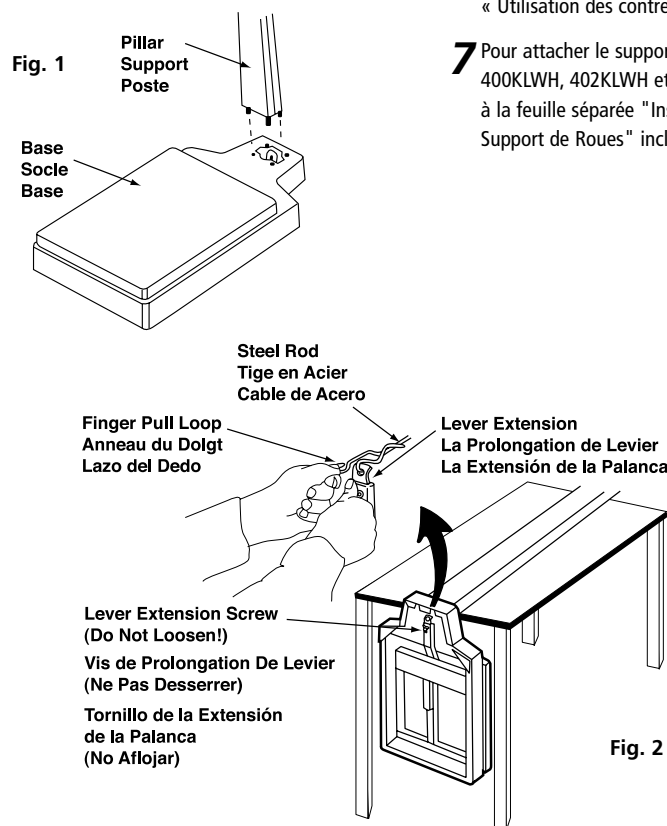
4 Hook the Steel Rod into the Lever Extension by pushing the Lever Extension toward the beam with your other hand. While holding in this position the hook on the Steel Rod is easily attached to the hole at end of lever extension. Release the Finger Pull Loop and the scale is ready to use.

NOTE: The proper position of the multi-lever assembly in order to attach the steel rod to the base is show in Figure 3. This assembly must be free floating in the pillar head. This can be observed through the opening between the Pillar Head and the Pillar.

5 Stand the scale upright.

6 (Models 402KL and 402LB only) To increase your scale's capacity by using the included counterweight(s), please refer to section "Using the Counterweights" below.

7 For instructions on attaching the wheel bracket to models 400KLWH, 402KLWH and 402LBWH, please refer to the separate sheet "Wheel Bracket Assembly Instructions" included in the package.

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LIMITED WARRANTY

What does the Warranty Cover?

Pelstar LLC scales are warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If scale fails to function properly, return the product, freight prepaid and properly packed to Pelstar. If manufacturer determines that a defect of material or in workmanship exists, customers' sole remedy will be repair or replacement of scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instruction, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service

Make sure you keep your sales receipt or document showing proof of purchase.

Call 1 (800) 638-3722 or 1 (708) 598-9100 to receive a return authorization number. Attach proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar LLC
Attention R/A# _____
Repair Department
7400 W. 100th Place
Bridgeview, IL 60455

If your scale is not covered by warranty, or has been damaged, an estimate of repair costs or replacement costs will be provided to you for approval prior to servicing or replacing.

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(models 402KL and 402LB only)

For 100 lb counterweight only:

If the weight of the patient is between 350 lbs and 400 lbs

- 1 Place the large (lower) poise weight in the 250 lb notch (this will weigh 350 lbs with the addition of the 100 lb counterweight).
- 2 Hang pound counterweight in notch on lower poise bar between 100 lbs and 150 lbs.
- 3 Move the small poise weight along the upper poise bar until the pointer is centered in the trig loop.
- 4 To arrive at the patient's total weight, add 350 lbs to the weight registered by the small poise weight. Example: for a patient weighing 378 lbs, the large poise weight will be on 250 lbs, and the small poise weight on the upper bar will be on 28 lbs.

If the weight of the patient is over 400 lbs

- 1 Place the large (lower) poise weight in the 300 lb notch (this will weigh 400 lbs with the addition of the 100 lb counterweight).
- 2 Hang pound counterweight in notch on lower poise bar between 100 lbs and 150 lbs.
- 3 Move the small poise weight along the upper poise bar until the pointer is centered in the trig loop.
- 4 To arrive at the patient's total weight, add 400 lbs to the weight registered by the small poise weight.

For 40 kg counterweight (402KL only):

If the weight of the patient is between 160 kg and 180 kg

- 1 Place the large (lower) poise weight in the 120 kg notch (this will weigh 160 kg with the addition of the 40 kg counterweight).
- 2 Hang kilogram counterweight in notch on lower poise bar between 60 kg and 80 kg.
- 3 Move the small poise weight along the upper poise bar until the pointer is centered in the trig loop.
- 4 To arrive at the patient's total weight, add 160 kg to the weight registered by the small poise weight. Example: for a patient weighing 172 kg, the large poise weight will be on 120 kg, and the small poise weight on the upper bar will be on 12 kg.

If the weight of the patient is over 180 kg

- 1 Place the large (lower) poise weight in the 140 kg notch (this will weigh 180 kg with the addition of the 40 kg counterweight).
- 2 Hang kilogram counterweight in notch on lower poise bar between 60 kg and 80 kg.
- 3 Move the small poise weight along the upper poise bar until the pointer is centered in the trig loop.
- 4 To arrive at the patient's total weight, add 180 kg to the weight registered by the small poise weight.

Scale Accessories:	Accessory	Part #
	Height rod	55150
	Wheels	55000
	Counterweights	55070

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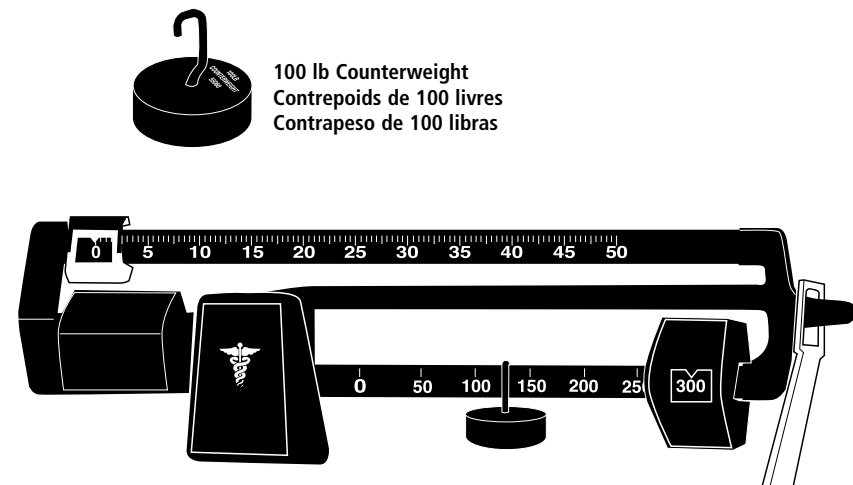
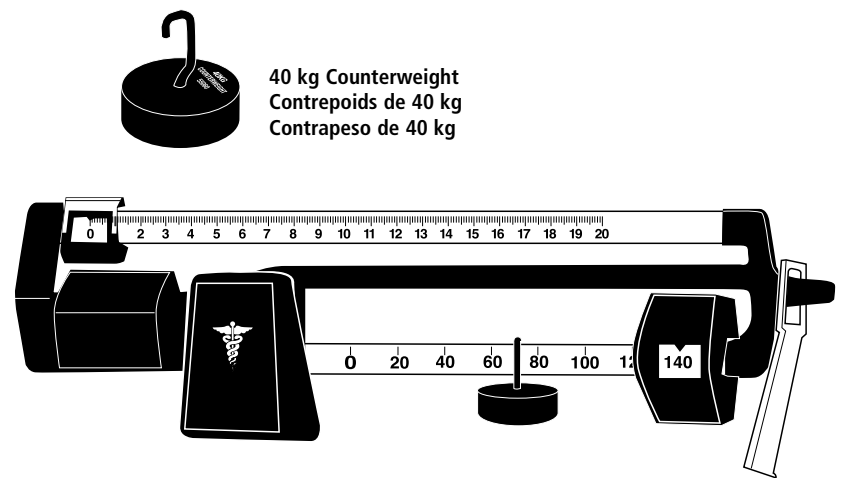
Models 400/402 Series

INCORRECT WEIGHT

The accuracy of these scales when calibrated at the factory is within plus or minus 1/4 pound. This means that two properly calibrated scales can vary 1/4 pound from each other and still be calibrated correctly. For the most accurate weight, use the scale on a hard level surface and stand in the center of the platform with the weight distributed as evenly as possible. If the error seems excessive, check the following:

1 Zero Balance of Beam

The beam must be balanced so the pointer comes to rest in the center of the trig-loop when both poises are set at zero. Make certain that the large poise is firmly seated in the notch and the small poise is firmly against the shoulder of the beam. Adjust the balance by turning the screw at the left end of the beam.

2 Beam Does Not Move Freely

Check if the beam pointer is touching the side of the trig-loop in its range of travel. Contact Technical Support at (800) 638-3722 for instructions on realigning the beam.

3 Platform Rocks Excessively or Touches Base at Any Corner

When you push down at any corner of the platform, you should not feel any significant rocking. If you do, please contact Technical Support at (800) 638-3722 for instructions on leveling the platform.

4 Beam Does Not Move at All during Weighing Process

This may be caused by the poises being set to a higher weight than the person's actual weight. Reset the weights to a lower weight. An improperly connected Steelyard Rod (the rod that runs inside the pillar) or a disconnected rod which can occur if the scale is transported may also be the cause. See Figure 2.

5 Scale Out of Calibration

This can happen if an adjustment is jolted out of place during transit or by wear in the notches in the beam after a long period of use. Lifting the large poise over the notches during movement will preserve the accuracy of the scale and lengthen the life of the beam. The accuracy should be determined with accurate test weights or the scale should be compared to a scale known to be accurate, such as a commercial scale with a Weights & Measures sticker affixed. The scale can be recalibrated by loosening the lock screw at the end of the long lever in the bottom of the base with a 5/16" hexagon socket wrench and moving it outward (longer) in the elongated slot to decrease the indicated weight or inward (shorter) to increase the indicated weight. One sixteenth inch movement of the screw will change the indicated weight by approximately 1/4 pound per 100 pounds.

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Operating Instructions

Zero Balance Adjustment

The scale should be checked periodically, particularly when moved, by following the instructions below :

1 Make certain scale is on a hard, level surface. Step on and off scale a few times in order to align all internal parts.

2 Place both poise weights on zero (see Fig 3). The beam pointer should float gently up and down and not touch the top or bottom of the trig-loop.

NOTE: If the beam pointer is not moving equally centered in the trig-loop opening, turn balance ball screw (See figure 3) to correct.

Conversion (lb to kg/ kg to lb)

(Models 400KL and 402KL only- Figure 4)

1 Rotate both the upper and lower poise bars with a gentle twist of the wrist (Figure 4) to the selected unit of measure.

2 Check poise bars to make sure both match in color. For easy identification, pound poise bars have chrome graduations on a black background; kilogram graduations have black graduations on a chrome background.

Weighing Instructions

1 Set the large and small poises to indicate your approximate weight.

2 Step on the scale. Continue to adjust the small poise until the pointer is centered.

NOTE: It is not necessary for you to wait for the pointer to stop moving in order to read your correct weight. As long as the pointer is moving equally in the trig-loop opening, the pointer would eventually come to rest in the center of the opening. Even when the pointer is not perfectly centered, as long as it is moving, you are within one quarter pound of your correct weight.

3 Add the indicated reading of small poise to the indicated reading of the large poise. The total is your weight.

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