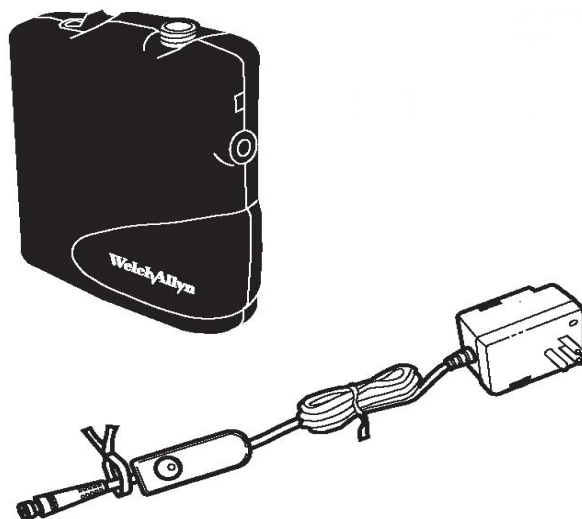
Power Sources (Portable and Direct Plug-In) Operating Instructions.....	4
---	---

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Thank you for purchasing the Welch Allyn **LumiView®** Power Source. To ensure accurate and reliable performance of your new instrument, please follow the operating and maintenance instructions found in this manual.

IMPORTANT: The material outlined in this manual should be reviewed and understood prior to operation of the equipment.

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Symbols

CAUTION: Consult accompanying documents.

Internally Powered

Recycle

NOTE: Provides supplemental information to the text and indicates a potentially hazardous situation, which, if not avoided, may result in property damage. Additionally, it highlights important information on the use of this equipment.

Standards Compliance

Listed Product ETL Laboratories

Listed Standards UL 60601-1, CAN/CSA C22.2
No. 601.1-M90, IEC 601-1

Power Sources (Portable and Direct Plug-In)

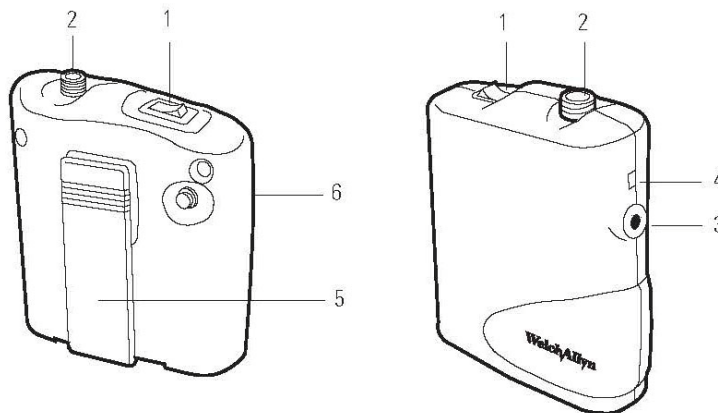
Portable Power Source

(Welch Allyn Model No. 75200)

This power source is designed for use in both the office and the hospital and is ideal for carrying to satellite offices. Power On time is 90 minutes after a full charge of 16 hours.

Features:

- | | |
|------------------------------|--------------------------|
| 1. On/Off Switch | 4. Charging Indicator |
| 2. Instrument Connector | 5. Belt Clip |
| 3. Charging Transformer Port | 6. Spare Lamp Receptacle |

Specifications:

Battery Type

Welch Allyn #72250 (nickel-cadmium)

Voltage/Capacity

3.6 VDC/1.8 Ahr

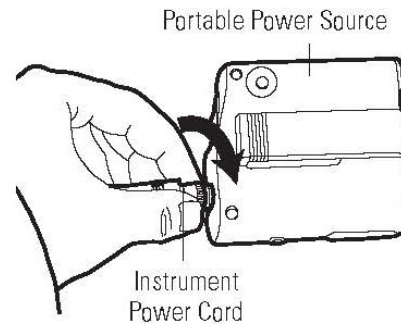
Charger

Welch Allyn #74180

(for international orders, specify country)


Connecting Instrument to Portable Power Source:

- Plug instrument power cord into the power cord connector port on top of the power source.
- Push in until firmly in place.
- Gently screw the external connector sleeve clockwise to secure (optional).


Recharging the Portable Power Source:

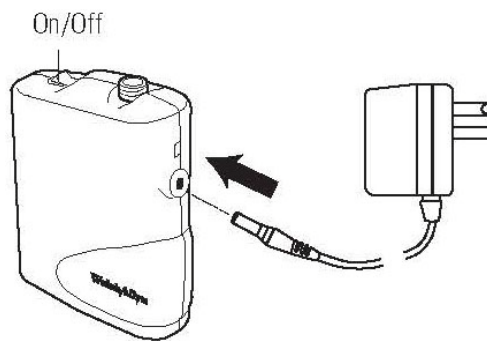- Turn portable power source off.
- Disconnect instrument from power source.
- Plug charging transformer into portable power source.
- Plug charging transformer into wall socket. LED light indicates unit is charging.
- Disconnect charging transformer from power source when fully charged, after 14-16 hours.


Accessing Spare Lamp

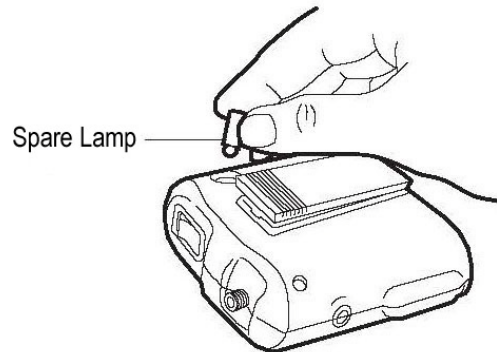- Lift the spare lamp out of its receptacle and use.

Battery Replacement

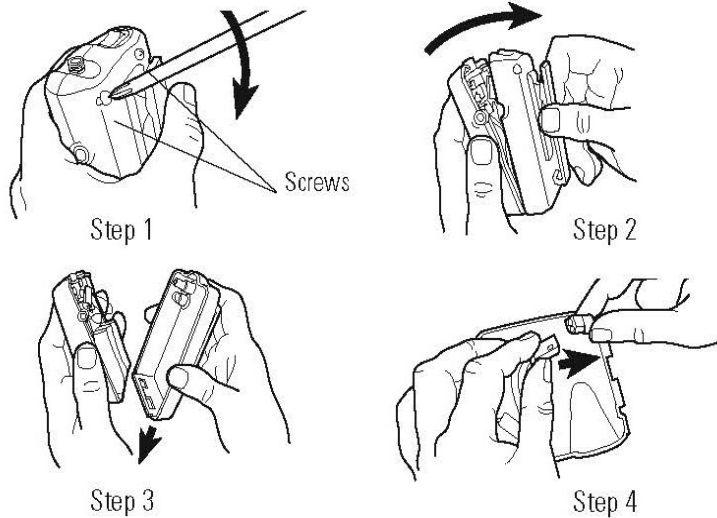- Be sure power is off.
- Remove both screws from back of power source with Phillips head screwdriver.
- Grasp the belt clip, gently pull back and down on the housing. (Steps 2 and 3)
- Lift battery out and unplug from unit.
- Replace with new battery and reverse above steps.

NOTE: Power source housing screws must be securely tightened in order to properly hold electrical components in place.

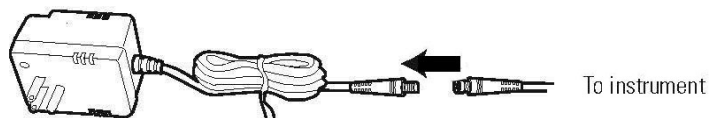
Battery Disposal

Nickel-cadmium battery. Battery must be recycled or disposed of properly.

Direct Plug-In Transformer

- Plug Power cord from the binocular otoscope into the Direct Plug-In Power cord.
- Push in until firmly in place.
- Gently screw the external connector sleeve clockwise to secure (optional).
- The instrument is now ready for use.
- Plug direct power source into a nearby receptacle.

Product and Accessory Ordering Information

Product No.	Instrument Description
	Power Sources
No. 75260	Portable Power Source REF 75200 w/Charger REF 74180
No. 75265	Portable Power Source REF 75200 w/International Charger (specify country and voltage)
No. 74180	Charger REF 74180 only for Portable Power Source REF 75200
No. 74185	International Charger only for Portable Power Source REF 75200 (specify country and voltage)
No. 75210	Direct Plug-In Power Source REF 75210
No. 75215	International Direct Plug-In Power Source (specify country and voltage)

Note: The power source options listed above operate with all instrument configurations.

	Accessories
No. 08500-U	Replacement Lamp for Device (all models)

Note: Only accessories and power sources indicated in this manual are to be used with the Portable Binocular Microscope and Flat Surface Magnifier.

Approvals

UL 60601-1, CAN/CSA NO. 601.1-M90

IEC 601-1

Service and Warranty Information

Service

For customers in North America, please return instruments requiring service to a Welch Allyn Technical Service Department listed below or to an authorized Welch Allyn distributor.

Technical Service Department

Welch Allyn, Inc.

4341 State Street Road

Skaneateles Falls, NY 13153-0220 U.S.A.

Telephone: 800-535-6663 (in U.S.A. only) or (315) 685-4560

Fax: 315-685-4653

For customers outside of North America, return your unit to a local, authorized Welch Allyn distributor or to your nearest Welch Allyn service center.

Warranty

Welch Allyn, Inc. warrants the Portable Binocular Microscope and Flat Surface Magnifier to be free of original defects in material and workmanship and to perform in accordance with manufacturer's specifications for a period of one year from the date of purchase. If this instrument or any component thereof is found to be defective or at variance with the manufacturer's specifications during the warranty period, Welch Allyn will repair or replace the instrument or component(s) at no cost to the purchaser.

This warranty only applies to instruments purchased new from Welch Allyn or its authorized distributors or representatives. The purchaser must return the instrument directly to Welch Allyn or an authorized distributor or representative and bear the costs of shipping.

This warranty does not cover breakage or failure due to tampering, misuse, neglect, accidents, modification or shipping, and is void if the instrument is not used in accordance with manufacturer's recommendations or if repaired or serviced by other than Welch Allyn or a Welch Allyn authorized representative.

No other express or implied warranty is given.

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Printed in the U.S.A.