# NUK®

## *Expressive®*

**Single Electric Breast Pump**

Operating instructions

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## Dear mother,

For babies, breastfeeding is the most natural and important nourishment there is. The NUK® Breastfeeding Program provides you with practical and convenient aids for breastfeeding. This easy to use electric breast pump is designed to help you and your baby whenever natural breastfeeding is not possible. Please take the time to read these operating instructions carefully before using your NUK® Expressive® Single Electric Breast Pump for the first time and keep for future reference. We wish you and your baby every happiness.

## 1. Warnings & Precautions

READ ALL INSTRUCTIONS BEFORE USING THIS ELECTRIC BREAST PUMP.

### IMPORTANT SAFEGUARDS FOR THE ELECTRIC BREAST PUMP.

When using electrical products, especially when children are present, basic safety precautions should always be followed. See details as follows:

**DANGER:** To reduce the risk of electrocution:

- Always unplug this electric breast pump immediately after using.
- Do not use while bathing or showering.
- Do not place or store product where it can fall or be pulled into a tub or a sink.
- Do not place in or drop pump motor unit into water or other liquid.
- Do not reach for this electric breast pump if it has fallen into water or other liquid. Unplug immediately.

**WARNING:** To reduce the risk of burns, electrocution, fire or injury to persons:

- This electric breast pump should never be left unattended when plugged into an electric outlet.
- Supervision is necessary when this product is in the vicinity of children. Electric medical devices like the NUK® Expressive® Single Electric Breast Pump may only be used by adults.

- Use this electric breast pump only for its intended use as described in this manual. Do not use accessory attachments not recommended by the manufacturer.
- Never operate this electric breast pump if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
- Keep the cord away from heated surfaces.
- Never use this electric breast pump while sleeping or drowsy.
- Do not use the device where products related to aerosol spray are being used or where oxygen is being administered.
- Never drop or insert any object into any opening or tubing.
- Only use the voltage adapter that comes with the NUK® Expressive® Single Electric Breast Pump or use batteries.
- Make sure the voltage of the device is compatible with the power source.
- Breast Pumps are personal care items and are not to be shared between mothers.
- When pump is not in use, be sure all parts of the electric breast pump and accessories are stored safely away from children.

## 2. Hygiene Tips

Hygiene is particularly important when using a breast pump. This applies to all the individual pump components as well as to your own personal hygiene. We recommend a daily shower or to wash your nipples in clean water. When possible, avoid using soap on your nipples, as this may cause irritation.

### 3. About this Pump

#### 3.1 Intended Use

The NUK® Expressive® Single Electric Breast Pump is intended to express milk from the breast by a single user.

#### 3.2 Features

The NUK® Expressive® Single Electric Breast Pump has a natural pulsing suction rhythm and offers the following benefits:

- Pulsing rhythm imitates your baby's natural suckling action while breastfeeding
- Easily adjustable suction strength
- Easy cleaning and assembly due to few parts
- Soft lined universal fit breast shield
- Convenient compact handheld design with voltage adapter or battery operation




#### 3.3 Contraindications

In the event of certain nipple problems (inflammation, open cuts) the NUK® Expressive® Single Electric Breast Pump should not be used or only used with special care. If you are unsure at any time whether to use the NUK® Expressive® Single Electric Breast Pump, please consult your medical professional.

#### 3.4 Symbols on the Pump

IN THE PUMP MOTOR UNIT (Battery Compartment)	
Symbol	Meaning
	When inserting batteries, check for correct pole alignment
	Do not dispose of batteries or the pump with household waste
	Refer to the operating instructions
	Refer to accompanying documents
	Lot number
	Serial number
ON THE PUMP MOTOR UNIT	
	Electrically protected device, Protection Class II
	Electromedical Device, Type BF
	Refer to accompanying documents
	Direct current 4.5V, max. 0.5A

## 4. Overview of Parts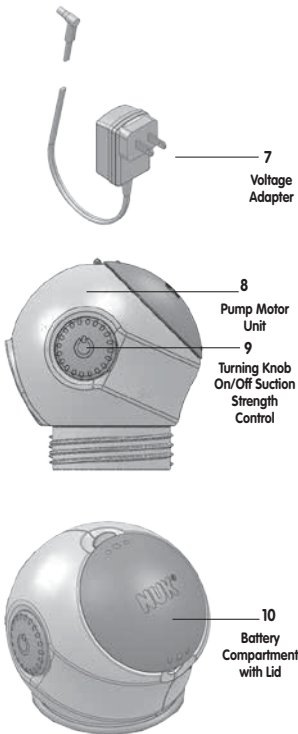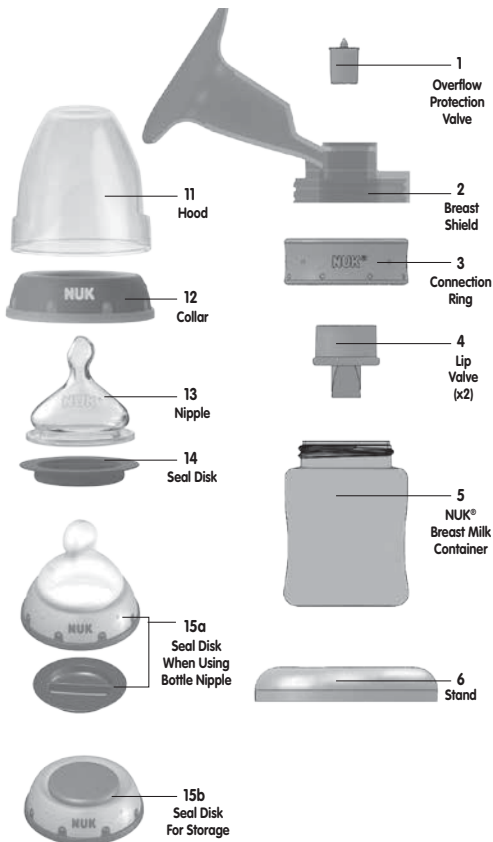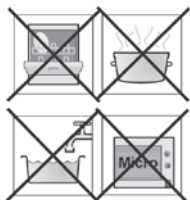

## 5. Disassembly of the Pump

**CAUTION:** Always hold the container (5) upright when it contains milk or the milk may spill out.

For the following please refer to "Overview of Parts".

1. Unplug the voltage adapter (7) from the pump motor unit (8).
2. Unscrew the milk storage container (5) from the connection ring (3).
3. Remove the lip valve (4) from the breast shield (2).

4. Unscrew the connection ring (3) from the breast shield (2) and pump motor unit (8).

5. Separate the breast shield (2) from the pump motor unit (8).

6. Take the overflow protection valve (1) out of the breast shield (2) by turning it upside down.

**NOTE:** Remove batteries if product is to be left unused for an extended period of time.

## 6. Cleaning

Hygiene is of priority when handling breast milk and all parts that come into contact with it. The pump should therefore only be used in a clean and sanitized condition. All accessories and parts, apart from the pump motor unit (8), battery compartment lid (10), and voltage adaptor (7) must be thoroughly cleaned.

**NOTE:** Prior to first use, clean the NUK® Breast Milk Container (5) and Nipple (13) by boiling in water for 5 minutes.

**CAUTION:** Unplug the voltage adapter (7) from the power outlet before cleaning.

1. Disassemble the pump according to Section 5.
2. After each use, wash all parts except for the pump motor unit (8), battery compartment lid (10) and voltage adapter (7) either by hand or using a dishwasher.

- 2a. To wash by hand, wash parts using plenty of hot water and liquid dishwashing soap, then rinse with hot water for 10 to 15 seconds. Clean the lip valve carefully to avoid damaging the material. Gently squeeze the valve to open it slightly during cleaning. Rinse thoroughly.

- 2b. To wash in a dishwasher, place the parts in the top rack of the dishwasher and start the cleaning cycle. Note that slight discoloration of the material may occur in rare cases.


3. Wipe the pump motor unit (8) and battery compartment lid (10) with a damp cloth. This cleaning is sufficient because the pump motor unit (8) and battery compartment lid (10) do not come into immediate contact with the milk.

**CAUTION:** Never immerse the pump motor unit (8) or voltage adapter (7) in water as this could damage the electric systems.

## 7. Pump Assembly

**Before using the pump, disassemble and clean according to the instructions in sections 5 and 6.**

1. Insert the overflow protection valve (1) into the breast shield (2) with the knob pointing upwards.

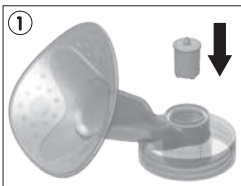
2. Place the pump motor unit (8) on the breast shield (2).


3. Screw the connection ring (3) on the pump motor unit (8) from below.



4. Press the lip valve (4) completely onto the bottom ring of the assembled breast shield (2).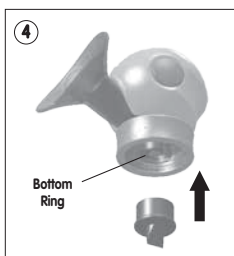

5. Screw the container (5) on the assembled pump motor unit (8) from below.

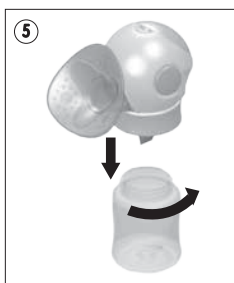
6. Open the battery compartment (10) by the catch lock.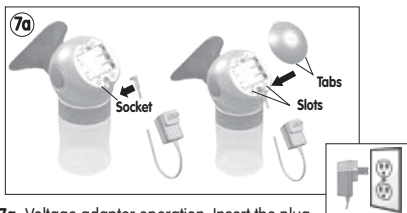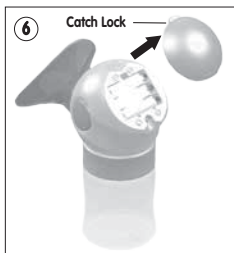

- 7a. Voltage adapter operation: Insert the plug of the voltage adapter (7) into the socket in the battery compartment. Close the battery compartment (10) by inserting the tabs of the lid into the matching slots on the pump motor unit (8). Close the lid (10). Be sure the catch lock engages with an audible click. Plug the adapter plug into a power outlet.


**NOTE:** The device should not be operated without the battery compartment lid in place.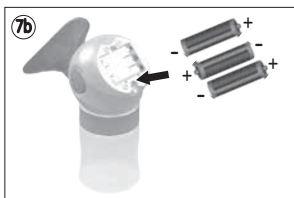

- 7b. Battery operation: Requires 3 Type AAA, 1.5 V batteries. Insert the batteries, ensuring that they are positioned correctly according to the + and - symbols in the battery compartment.

**NOTE:** The voltage adapter will not charge the batteries.

**CAUTION:** Do not mix old and new batteries. Do not mix alkaline, standard (carbon-zinc) or rechargeable (ni-cad, ni-mh, etc.) batteries.

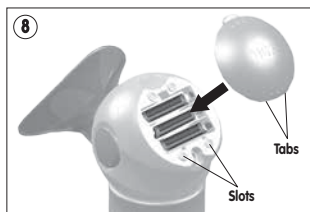8. Close the battery compartment (10) by inserting the tabs of the lid into the matching slots on the pump motor unit (8) and close the lid (10). Be sure the catch lock engages with an audible click.

**NOTE:** The device should not be operated without the battery compartment lid (10) in place.

## 8. Instructions for Use

Before using the pump, disassemble and clean according to the instructions in sections 5 and 6.

### 8.1 Operation

1. Position the breast shield (2) on your breast so that the nipple is positioned in the center of the shield. Gently press the breast shield (2) in order to ensure a good seal when expressing the milk. Hold the pump in a vertical position.

**NOTE:** If the pump tips over, it will turn off automatically.

2. Turn the breast pump on. To do so, gently turn the knob (9) clockwise. The pump is on once you hear a click.

3. Now you can adjust the suction strength by turning the knob (9) even more clockwise. You can turn the knob counterclockwise at any time in order to reduce the suction strength. If you feel pain for any reason, stop pumping, using either the controls on the pump or by inserting a finger between the shield and breast to break the suction.

**CAUTION:** In order not to damage the pump, you should be sure that the expressed milk collected in the container (5) does not contact the lip valve (4).

4. To switch off the pump, turn the knob (9) counterclockwise to the very end. The pump turns off once you hear a click.

5. After expressing, place the breast pump in the stand (6) for stability. The pump must not be allowed to topple over as milk might leak from the container (5).

6. Unscrew the NUK® Breast Milk Container (5) from the connection ring (3) and pump motor unit (8). Be sure to keep the container vertical.

Correct expressing needs practice. You should first become accustomed to your new pump. If you continue to experience difficulty nursing, contact your medical professional.

### 8.2 Frequency of Expressing

The frequency of expressing depends on your individual situation. If you will be apart from your baby and/or want to sustain milk production, the natural breastfeeding rhythm should be maintained. You should therefore express 6 to 8 times daily, including once during the night. In all other cases you should express entirely to suit your own individual needs.

### 8.3 Procedure

Express milk from each breast in turn. This electric breast pump has a pulsing suction rhythm that imitates your baby's natural sucking action while breastfeeding.

- Alternate between breasts when the milk flow slows down or ceases.

- The amount of time needed to express milk will vary considerably from one person to another.

The following intervals have proven effective, but times may vary according to the individual:

- right 5 – 7 minutes, left 5 – 7 minutes;
- right 3 – 5 minutes, left 3 – 5 minutes;
- right 2 – 3 minutes, left 2 – 3 minutes.

### 8.4 Milk Quantity

Do not be surprised if it takes a little longer at first before the milk really starts to flow. This is not unusual, in fact it is perfectly normal, because in general, milk production adapts to daily requirements. This means that the quantity of milk produced may vary considerably.

- Be sure to drink enough fluids to stimulate milk production.
- Do not set the suction strength too high. Higher suction strength does not cause a larger amount of milk.

### 8.5 Let-Down Reflex

During natural breastfeeding, placing the baby on the breast stimulates the let-down reflex and the milk begins to flow. You can further stimulate if you:

- Find a quiet place where you can be fully relaxed as you express the milk.
- Adopt a comfortable and relaxed sitting position.
- Place a warm compress on your breast for a few minutes to relax the tissue before expressing.
- It may be helpful to have eye contact with your baby. If that is not possible, try to look at a photo of your baby.
- The let-down reflex releases the flow of milk to both breasts simultaneously.

## 9. Storing & Feeding Milk

If the breast milk is not to be used immediately, you can seal the filled container (5) with the collar (12) and seal disk (14) as shown in section 4 (15b) or pour from the container (5) into a NUK® Seal 'n Go® Breast Milk Bag. Store it in the refrigerator. Use an erasable marker to write the date and time of expressing on the container (5).

**CAUTION:** Only the NUK® Breast Milk Container with the collar (12) and seal disk (14) should be stored in the refrigerator, not the entire NUK® Expressive® Single Electric Breast Pump.

### 9.1 NUK® Breast Milk Container

The breast milk may be chilled or frozen and thawed out in a sealed breast milk container (5). The breast milk container (5) may also be used for re-heating and feeding. Please always observe the information below on handling breast milk.

**NOTE:** When freezing breast milk, do not fill the container (5) with more than 5 fl. oz as breast milk expands as it freezes and the container (5) may burst.

### 9.2 Tips for Handling Breast Milk

NUK® Seal 'n Go® Milk Bags and Breast Milk Containers (sold with any NUK® breast pump) are safe for storage in the refrigerator or freezer.

- Once the container has been opened, the maximum storage time is 12 hours at 39°F (4°C).
- The milk may be thawed out in the container either in the refrigerator, or, if required immediately, by standing in warm water.

- If desired, warm the milk to body temperature by placing it in a NUK® Bottle Warmer, or in a container of heated water. Always check temperature of the milk on your wrist before feeding baby.

**Caution:** Breast milk must not be thawed out in a microwave as this will destroy valuable nutrients as well as cause possible damage to the container material, which may cause the risk of explosion. Uneven heating may also result in scalding!

- The milk should therefore be shaken gently before feeding to ensure that milk and milk fats are well mixed and that the warm breast milk is evenly distributed.
- Defrosted, re-heated or unused milk must be disposed of and not refrozen.
- If you express breast milk more than once within a period of 24 hours, the freshly expressed milk may be added to the previously cooled breast milk under the following conditions: The temperatures must be the same (cold on cold). Freshly expressed milk must therefore be cooled first before being added to milk that is already cool.
- Consult your medical professional for more detailed information about storage.

Follow the below guidelines for safe storage of breast milk to ensure freshness.

GUIDELINES FOR STORING BREAST MILK <sup>1</sup>			
Location	Temperature	Duration	Comments
Countertop, table	Room temperature (up to 77°F or 25°C)	6–8 hours	Containers should be covered and kept as cool as possible; covering the container with a cool towel may keep milk cooler.
Insulated cooler bag	5 - 39°F or -15 - 4°C	24 hours	Keep ice packs in contact with milk containers at all times, limit opening cooler bag.
Refrigerator	39°F or 4°C	5 days	Store milk in the back of the main body of the refrigerator.
Freezer			
Freezer compartment of a refrigerator	5°F or -15°C	2 weeks	Store milk toward the back of the freezer, where temperature is most constant. Milk stored for longer durations in the ranges listed is safe, but some of the lipids in the milk undergo degradation resulting in lower quality.
Freezer compartment of refrigerator with separate doors	0°F or -18°C	3–6 months	
Chest or upright deep freezer	-4°F or -20°C	6–12 months	
STORAGE GUIDELINES FOR THAWED BREAST MILK <sup>1</sup>			
Room Temperature (60°F to 85°F)		Refrigerator (39°F or colder)	Freezer
Up to 1-2 hours is best. Up to 3-4 hours is ok.		24 hours	Do not refreeze thawed breast milk.

<http://www.womenshealth.gov>

### 9.3 Feeding Expressed Milk with NUK® Breast Milk Container

#### IMPORTANT SAFEGUARDS FOR THE BREAST MILK CONTAINER & NIPPLE

**WARNING:** For safety and health reasons please note the following:

- Prolonged periods of time drinking liquids, other than water, may cause tooth decay.
- To prevent possible choking hazard, test strength before each use by pulling the nipple in all directions. Any tear can lead to the separation of pieces. Discontinue use if any tear or crack appears. Do not attempt to enlarge the hole as this may lead to tearing.

#### CAUTION:

- Do not overheat. Always test temperature of liquid on wrist before feeding baby.
- Do not microwave, since this can cause uneven heating.
- Always supervise baby when feeding. To avoid injury, do not allow a child to walk or run with a bottle.

#### CARE:

- Disassemble before cleaning. Prior to first use, sanitize by boiling in water for 5 minutes.
  - For day to day cleaning, wash with hot water and mild detergent then rinse clean. Dishwasher-safe (top rack). Allow to dry before storage.
  - To heat, use a NUK® Bottle Warmer or place the bottle under hot running water or in a container of heated water.
  - For hygiene and safety, replace with appropriate NUK® Nipples after about 2 months of use.
1. Apply the supplied nipple (13) onto the breast milk container (5) with the supplied container collar (12). The NUK® Breast Milk Container can be used with all NUK® Wide-Neck Nipples.
  2. The NUK® Air System vents the bottle, preventing vacuum buildup and allows baby to drink continuously, swallowing liquid instead of air. Make sure the NUK® Air System symbol on the nipple (13) is lined up under baby's nose for the best results.

## 10. Troubleshooting

Problem	Reason	Solution
Pump does not work	There is no power	For battery operation check that the batteries are correctly inserted.
		For voltage adapter (7) operation check that the plug is correctly inserted or try using a different power outlet.
	Lip valve (4) is missing or is not properly attached	Check that lip valve (4) is attached correctly.
	Pump is being held at too great an angle	Hold the pump upright. If the pump is held at too great an angle it will switch off automatically.
	Connection ring is not tight enough	Tighten the connection ring.
Pump has only little suction	There is no vacuum	Ensure that the breast shield is sealed against the breast so no air escapes.
	Low battery power	Replace batteries.
	Suction strength is set too low	Turn the knob (9) clockwise until the suction strength meets your needs.
	Lip valve (4) has a tear	Check the lip wall for tears. Replace it with spare lip valve (4), if damaged.
Pump suction too powerful	Missing overflow protection valve (1)	Disassemble pump and place overflow protection valve (1) in proper location.
	Suction strength is set too high	Turn the knob (9) counterclockwise until the suction strength meets your needs.

## 11. Technical Data

<b>Medical device</b>	<b>Class II</b>
<b>NUK® Vented Nipple</b>	<b>Complies with EN 14350</b>
<b>Temperature Range</b> — Operation — Storage	47°F to 104°F (8°C to + 40°C) 47°F to 104°F (8°C to + 40°C)
<b>Power Supply:</b> — Operation via adapter — Battery Operation	Input 100 ~ 240V ~ /50 Hz, output 4.5 VDC, 800 mA, hollow plug: polarity inner +, outer - 3 type AAA batteries, 1.5V

## 12. Disposal

Products labeled with a crossed-through wheeled bin must not be disposed of with normal household waste, but should be taken to a collection point for recycling electric and electronic equipment.

## 13. ONE YEAR LIMITED WARRANTY

NUK USA LLC warrants the NUK® Expressive® Single Electric Breast Pump product sold in the USA for a period of one year from the date of purchase against defective material and workmanship. If defective, NUK USA LLC will replace the product with the same or comparable model (at its option) during the warranty period. If the product is no longer available, replacement may be made with a similar product of equal or greater value. This is your exclusive warranty. This warranty is valid for the original retail purchaser from the date of initial retail purchase and is not transferable. Keep the original sales receipt. Proof of purchase (dated sales receipt) is required. NUK® dealers, service centers, or retail stores selling NUK® products do not have the right to alter, modify or any way change the terms and conditions of this warranty. This warranty does not cover normal wear of parts or damage resulting from any of the following: Negligent use or misuse of the product, commercial use, use on improper voltage or current, use contrary to the operating instructions, disassembly, repair or alteration by anyone other than NUK USA LLC.

### What are the limits on NUK USA LLC's Liability?

NUK USA LLC shall not be liable for any incidental or consequential damages caused by the breach of any express, implied or statutory warranty or condition. Except to the extent prohibited by applicable law, any implied warranty or condition of merchantability or fitness for a particular purpose is limited in duration to the duration of the above warranty. NUK USA LLC disclaims all other warranties, conditions or representations, express, implied, statutory or otherwise. NUK USA LLC shall not be liable for any damages of any kind resulting from the purchase, use or misuse of, or inability to use the product including incidental, special, consequential or similar damages or loss of profits, or for any breach of contract, fundamental or otherwise, or for any claim brought against purchaser by any other party. Some provinces, states or jurisdictions do not allow the exclusion or limitation of incidental or consequential damages or limitations on how long an implied warranty lasts, so the above limitations or exclusion may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or jurisdiction to jurisdiction.

### How to claim the Warranty?

To make a claim or troubleshoot a problem, please contact the Customer Service Center at 1-888-NUK-1238. Customer Service will provide the assistance in resolving the issue through troubleshooting and/or provide an address for returning the product.

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### ¿Cuáles son los límites de responsabilidad de NUK USA LLC?