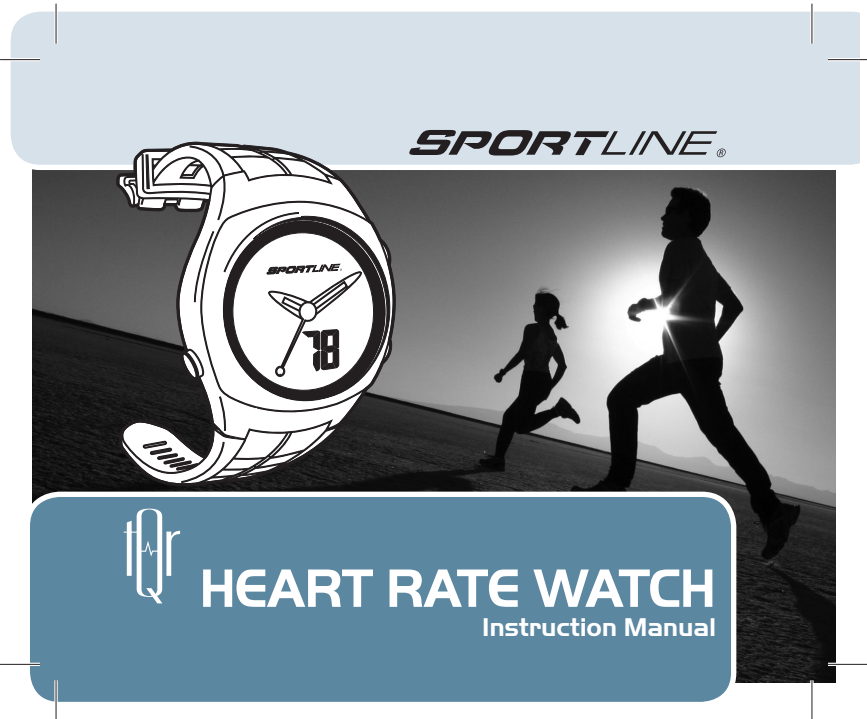
TABLE OF CONTENTS

HEART RATE WATCH

INTRODUCTION	3
WARNING & CAUTIONS	4
CARE AND MAINTENANCE	5
BUTTON OPERATION OVERVIEW	6
HOW TO MEASURE YOUR HEART RATE	7
HELPFUL HINTS IN MEASURING HEART RATE	8
SETTING THE WATCH FUNCTIONS	9
SPECIFICATIONS	10
REPLACING THE BATTERY	11
WARRANTY	13

INTRODUCTION

This sleek, stylish LED Heart Rate Watch measures ECG accurate heart rate with the simple touch of your fingers—no chest strap is required! It operates as a basic analog watch but is also coupled with heart rate monitor functions. Its bright LED output vividly showcases the digital display so you will never miss a beat.

SP4418/ 710 M SP4420/ 710 W SP4422/ 725 W SP4424/ 750 W SP4468/ 775 W

3

WARNING & CAUTIONS

The device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the product.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the transmitting equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio technician for help.

4

SPORTLINE®

CARE AND MAINTENANCE


To ensure proper function of your LED Heart Rate Watch:

Avoid rough usage or severe impacts to the watch.

- Keep the side sensor keys (**SC** and **HR**) and back metal plate free from dirt, oils and other contaminants.
- Periodically clean the watch with a soft cloth, mild soap and water (or similar cleaning solution).
- Keep the watch out of extreme heat or cold.
- Do not expose the watch to intense direct sunlight for long periods of time.
- Do not expose the watch to chemicals such as gasoline, alcohol or solvents.

BUTTON OPERATION OVERVIEW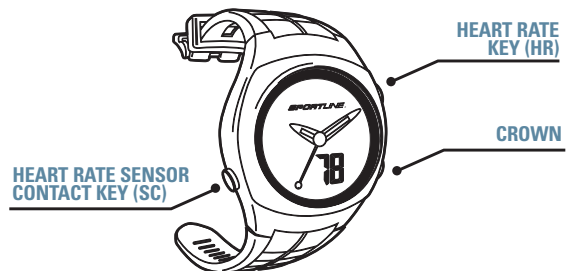

- **Heart Rate Key (HR)**

- Press & Hold: activate Heart Rate function; must be pressed at the same time with Heart Rate Sensor Contact Key (SC)

- **Crown**

- Adjust analog time display

- **Back Cover (Sensor Contact)**

- Primary heart rate sensor

- **Heart Rate Sensor Contact Key (SC)**

- Press & Hold: activate Heart Rate function; must be pressed at the same time with Heart Rate Key (HR)

6

SPORTLINE®

HOW TO MEASURE YOUR HEART RATE

To get your heart rate reading, follow these three simple steps:

1. Place the watch on your wrist, adjusting for a snug, comfortable fit.
2. Place your fingers on the watch's two sensor keys (**SC** and **HR**) and press simultaneously for 3-8 seconds. The  will flash for about 5 seconds.
3. A 'beep' will sound and your heart rate will be displayed. It's that easy!

The LED Heart Rate Watch is a highly responsive monitoring device that reads your ECG signals; reading times may vary. If you don't get a reading in 3-8 seconds, wait a couple of minutes and try again. This gives the watch a chance to settle in and pick up your electrical signals.

Your heart rate will be displayed once measured, and will remain displayed for 5 seconds after you release the sensor keys.

IMPORTANT:

The Heart Rate measure function is not intended for use in or under water.

It is not intended to be used in emergency situations.

7

HELPFUL HINTS IN MEASURING HEART RATE

If you are having difficulty measuring your heart rate, you may want to try the following steps. Try each of these in succession until you are able to measure your heart rate:

1. Make sure the watch is snugly strapped to your wrist. A loose fit makes it more difficult to properly measure your heart rate. Stop if *OK* or ▼
2. When pressing on the two sensor keys (**SC** and **HR**), make sure you are using the soft, flat pads of your fingers—not the tips of your fingers. Stop if *OK* or ▼
3. Press gently with enough pressure to activate the heart rate measure function. Muscle 'noise' from pressing too hard can make it difficult to measure your heart rate. Stop if *OK* or ▼
4. Make sure the metal back of the watch is laying flat on your skin. Stop if *OK* or ▼
5. Make sure your wrist and fingers are clean and free from dirt, oils, or lotions. Stop if *OK* or ▼
6. Stay still and relax your arms on a stable surface while taking your heart rate. Stop if *OK* or ▼
7. Clean the two sensor keys and back metal plate with mild soap and water, or similar cleaning solution. Stop if *OK* or ▼
8. Moisten, with water, the pads of the fingers you are using to contact the two sensor keys. Stop if *OK* or ▼
9. Moisten, with water, the surface between the back metal plate of the watch and the skin on your wrist.

8

SPORTLINE®

SETTING THE WATCH FUNCTIONS

SETTING ANALOG TIME

1. Gently pull the Crown outward until it clicks into place. The 'seconds hand' will remain stationary.
2. Rotate the Crown clockwise or counterclockwise to adjust the hands.
3. When the time is set, push the Crown back into the locked position.
(You should hear it click into locked position.)

9

SPECIFICATIONS

Analog Time

- Time System: Quartz movement
- Hands: hour, minute and seconds

Heart Rate Monitor

- Measuring range: 43 to 199 beats per minute

Other

- Water resistant: 30 meters

IMPORTANT:

The Heart Rate measure function is not intended for use in or under water.

The Heart Rate measure function is not intended to be used in emergency situations.

10

SPORTLINE®

REPLACING THE BATTERY

The watch uses one (1) standard lithium replacement battery: **CR2032, 3 volts.**

Battery replacement will be required from time to time. Battery life will vary depending on usage of the Heart Rate feature which requires high power consumption. To replace the battery of your LED Heart Rate Watch, we recommend this be done by a watch repair service center. This ensures the water resistance will be retained after battery replacement.

The battery should be changed when:

1. The display fades in part or dims completely.
2. The heart rate function will not activate,

||

REPLACING THE BATTERY

CAUTION - Battery Safety Information

- Non-rechargeable batteries are not to be recharged
- Rechargeable batteries must be removed before recharging
- Rechargeable batteries should only be recharged under adult supervision
- Do not mix alkaline, standard (carbon-zinc), or rechargeable (nickel-cadmium) batteries
- Exhausted batteries are to be removed
- Do not mix old and new batteries
- Do not dispose of batteries in fire; batteries may explode or leak
- Do not dispose of product in fire; batteries may explode or leak
- If this product will not be used for an extended period of time, remove the batteries from the product
- The supply terminals are not to be short-circuited
- Batteries are to be inserted with correct polarity
- Clean the battery contacts and also those of the product prior to battery installation
- As always, dispose of batteries in an eco-friendly manner

CUSTOMER SUPPORT

If you have questions, comments or need assistance with your TQR watch, you may contact Customer Support by calling **1-866-694-4575**.

12

SPORTLINE®

WARRANTY

GUARANTEE CERTIFICATE

For purchases made in the U.S.

If at any time within one (1) year from the purchase date of this product, it fails to perform properly because of defects in material or manufacturing, return it prepaid to: EB SPORT GROUP, 585 Oak Ridge Road, Hazleton, PA 18202

Complete this Guarantee Certificate and enclose it (or a copy of it) with the product.

(PLEASE PRINT CLEARLY)

NAME _____ TEL _____

ADDRESS _____

CITY _____ STATE _____ ZIP _____

STORE PURCHASED FROM _____ DATE OF PURCHASE _____

PURCHASE PRICE _____ PROBLEM WITH PRODUCT _____

REPLACEMENT

This product has been carefully designed, manufactured and packaged. If any parts are missing or damaged, please contact us at the address above and we will replace the unit at no charge. (Guarantee does not cover crystal or battery.)

FOR PURCHASES MADE IN THE UK:

Should you experience problems with this item, please return it to the point of purchase. This will not affect your statutory rights. Alternatively return it to the address shown below for a repair/replacement: EB Brands UK, Berkeley Business Park, Wainwright Road, Worcester WR4 9FA

13

IMPORTANT:

This LED Heart Rate Watch is not a medical device, nor intended for use in any medical or patient monitoring applications.

This watch is not intended for use in any commercial application.

Always consult a physician before starting any physical activity.

The features/images of your watch may vary slightly from those described in this User Guide. The information in this User Guide is intended to be for information purposes only, and is subject to change without notice.

S-Pulse™ is a trademark of Salutron, Inc.


PATENTS

This LED Heart Rate Watch and S-Pulse™ technology are a result of, and protected by, the following Salutron, Inc. patents:

US: 5,738,104 & 5,876,350

Europe: EPO 0861045B1

Distributed under license by EB Brands, EB Sport Group

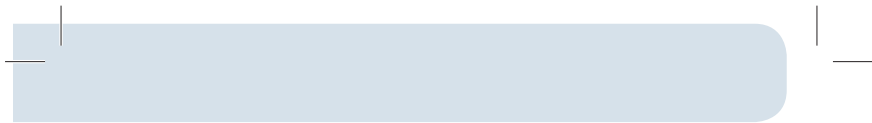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Sportline[®] is a trademark of SPORTLINE,
All Rights Reserved. ©2011
S-Pulse™ ist ein Markenzeichen der Salutron, Inc.