



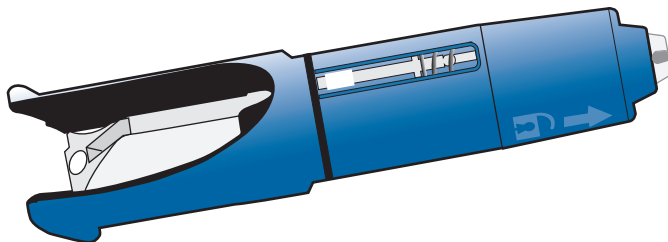
Medtronic


MINIMED

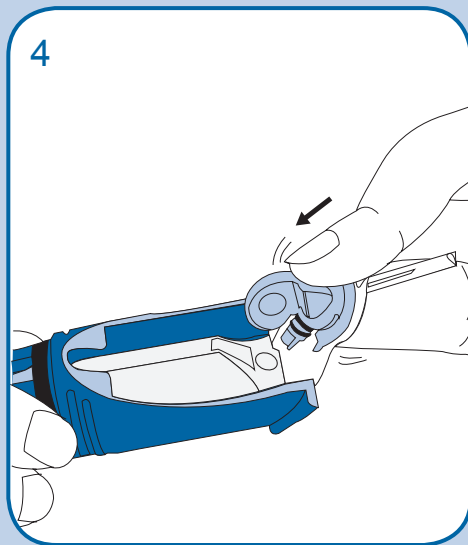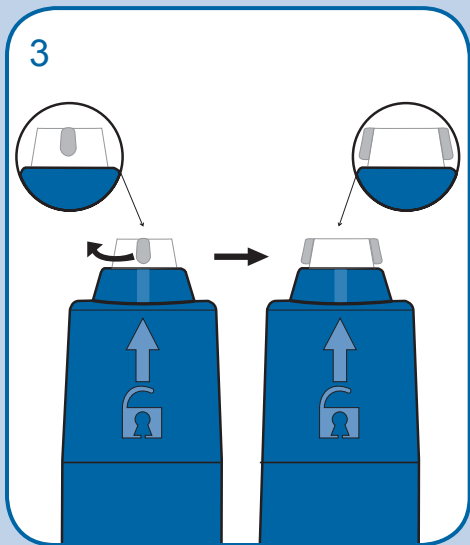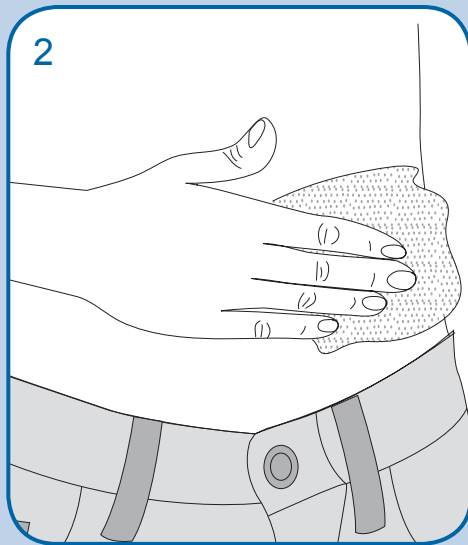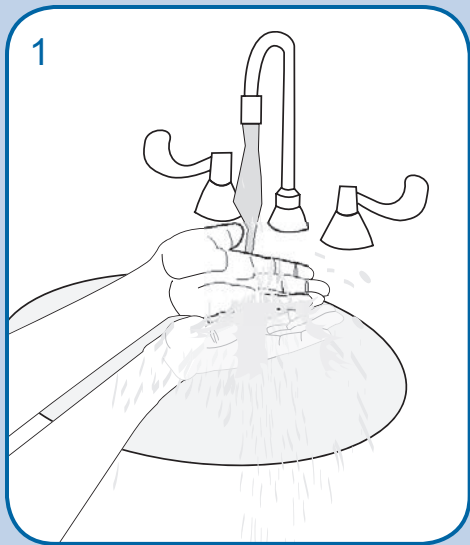- Sen-Serter[®] User Guide
- Manuel d'utilisation du Sen-Serter[®]
- Sen-Serter[®] Gebrauchsanweisung
- Sen-Serter[®] Guía del usuario
- Sen-Serter[®] gebruiksaanwijzing
- Guida per l'utente del Sen-Serter[®]
- Brukanvisning for Sen-Serter[®]
- Sen-Serter[®]-käyttöopas
- Sen-Serter[®] Bruksanvisning
- Sen-Serter[®] Brugsanvisning
- Sen-Serter[®] - Guia do utilizador
- Sen-Serter[®] Navodilo za uporabo
- Sen-Serter[®] uživatelská příručka
- Sen-Serter[®] használati útmutató
- Sen-Serter[®] - Podręcznik użytkownika
- Sen-Serter[®] Kullanım Kılavuzu
- Sen-Serter[®] – příručka uživateľa
- Sen-Serter[®] Οδηγός χρήσης
- Sen-Serter[®] Руководство пользователя
- Sen-Serter[®] دليل المستخدم
- Sen-Serter[®] המדריך למשתמש של


5

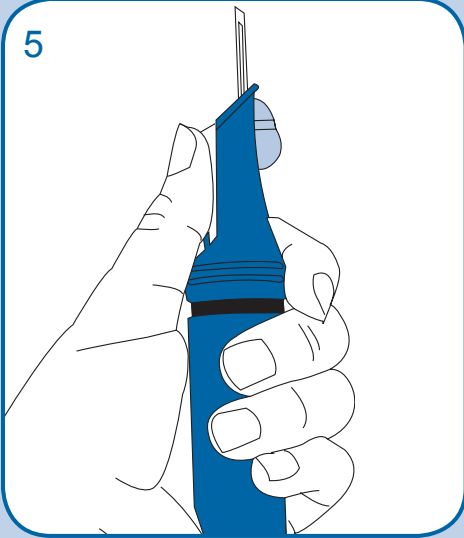
6





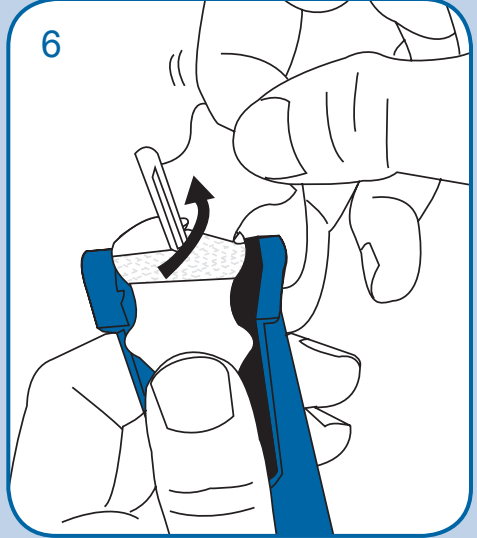
7

"click"	"klik"	"kattanás"
"clíc"	"klik"	"klik"
"klick"	"clique"	"tik"
"clíc"	"klik"	"κλικ"
"klik"	"kiknëte"	«щелчок»
"clíc"		"klik"
"klick"		"نقر"
"klik"		"קליק"

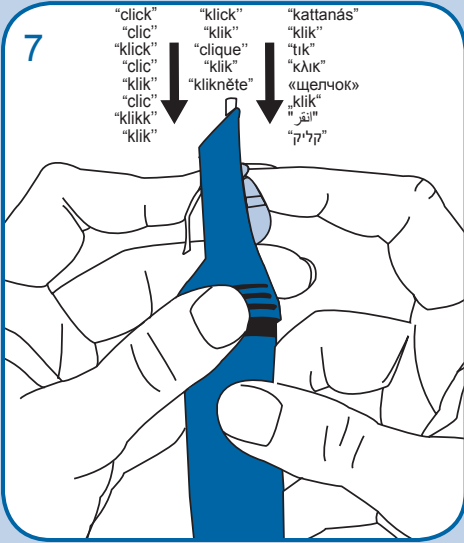
8

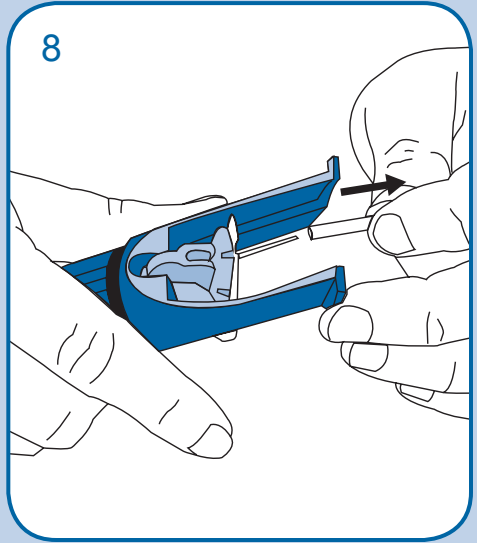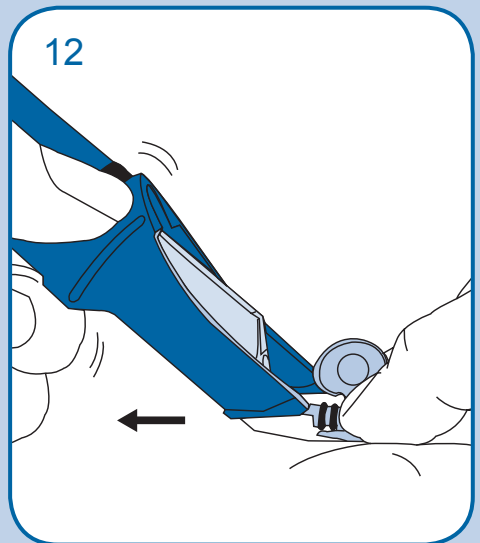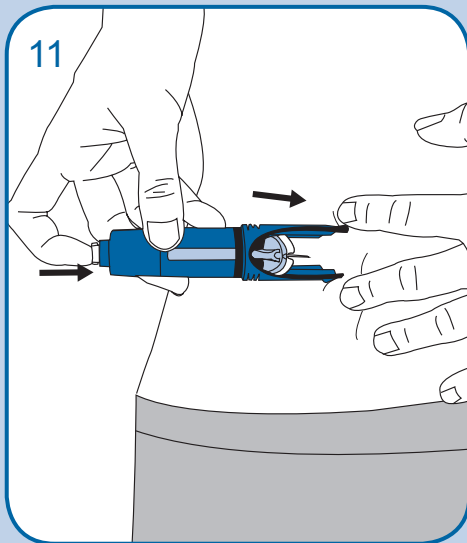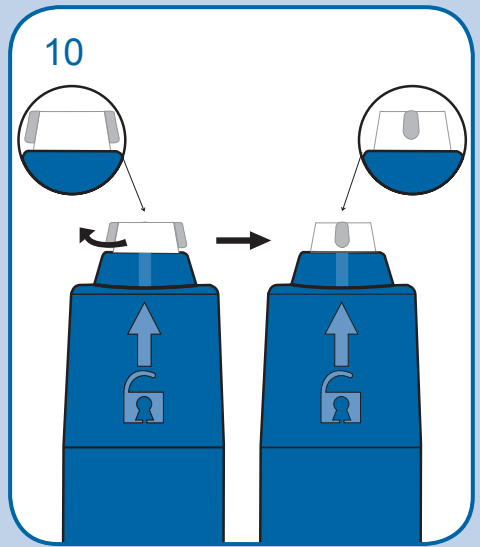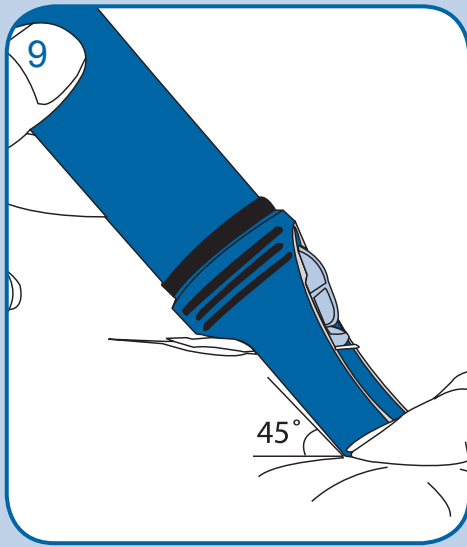
13

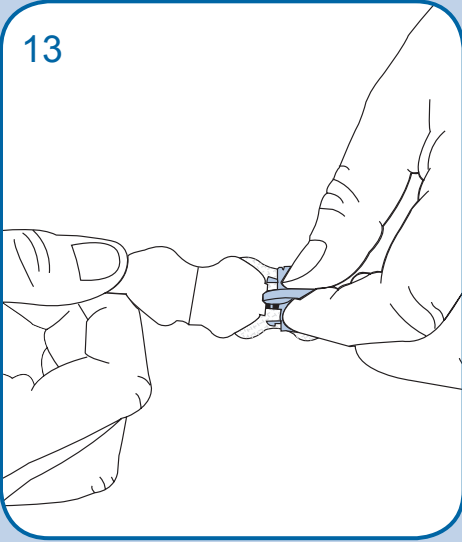14

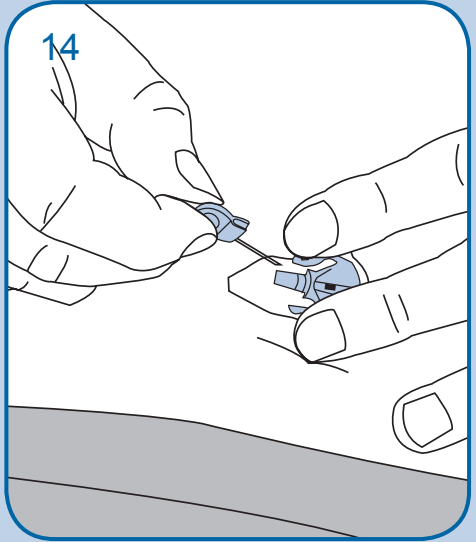Indications For Use

The Sen-serter is indicated as an aid for insertion of the Medtronic MiniMed Glucose Sensor.

The Glucose Sensor is intended for use with Medtronic MiniMed glucose sensing systems to continuously monitor glucose levels in persons with diabetes.

Contraindications

The Sen-serter is not indicated for use with product other than the Glucose Sensor. Medtronic MiniMed cannot guarantee this product's safety and/or efficacy if used on other products.

Warnings

Failure to follow instructions may result in pain or injury.

Bleeding, swelling, irritation and/or infection at the insertion site are possible risks associated with inserting the Sensor and may result from improper insertion and maintenance of insertion site.

After insertion, wait 5 minutes before attaching a transmitter to the sensor. Watch for bleeding at the insertion site. If bleeding occurs, apply steady pressure using a sterile gauze or clean cloth for up to 3 minutes.

If bleeding stops:

1. Attach the transmitter to the sensor.

If bleeding does NOT stop:

Caution: DO NOT attach the transmitter to the sensor.

1. Remove the sensor and discard.
2. Check the site for redness, bleeding, irritation, pain, tenderness or inflammation and treat accordingly.
3. Insert a new sensor in a different location

If Sensor is not securely placed in Sen-serter prior to insertion, pain or minor injury may occur.

Keep the Sen-serter locked until ready for insertion. Never point loaded Sen-serter toward any body part where insertion is not desired.

If you develop inflammation, redness, soreness or tenderness at insertion site, or if you experience unexplained fevers, remove Sensor. Check site often for these conditions and to ensure Sensor is still in place, especially before going to bed and upon waking.

Precautions

Always wash hands with soap and water before opening the Sensor package and handling Sensor. After opening the package, avoid touching any Sensor surfaces that will come in contact with the body, i.e., Sensor, needle, connector adhesive surfaces and tape.

Before inserting Sensor, clean insertion site with a topical antimicrobial solution, such as isopropyl alcohol. Allow area to dry.

Do not insert Sensor through tape, and do not use sticky preps before insertion.

Establish a rotation schedule for choosing each new Sensor site. Choose a site with adequate subcutaneous fat for Sensor insertion. The lower abdomen and upper, outer quadrant of the buttocks are most frequently used. Avoid sites that are constrained by clothing, scar tissue, accessories or subjected to rigorous movement during exercise. Also, avoid sites around belt or waistline or 2 inches (5.08 cm) around navel.

Make sure Sensor insertion site is at least 3 inches (7.62 cm) from insulin pump infusion site or manual injection site. When replacing Sensor, select new insertion site at least 2 inches (5.08 cm) from previous site.

Sen-serter Maintenance

For single-patient use

1. Dampen a clean cloth with mild liquid soap and warm water, and wipe the Sen-serter.
2. Rinse with warm tap water.
3. Using an antibacterial hand-sanitizer (readily available at your local drugstore), wipe down the Sen-Serter.
4. Place the Sen-serter on a clean dry cloth and allow to air dry.
5. Store Sen-serter in released position to maintain optimum product performance and life.

For multiple-patient use

1. Dampen a clean cloth with mild liquid soap and warm water, and wipe the Sen-serter.
2. Rinse with warm tap water.
3. Apply 3-4 drops of quaternary ammonium compound disinfectant, for example Cavicide®, on a clean, dry cloth. Wipe down the Sen-serter.
4. Using 70 percent isopropyl alcohol, or isopropyl alcohol wipes, wipe down the Sen-serter.
5. Place the Sen-serter on a clean dry cloth and allow to air dry.
6. Once the Sen-serter is dry, place it in a sealed bag, labeled with the cleaning date.

Store Sen-serter in released position to maintain optimum product performance and life.

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EN The Sensor and Sen-serter are covered by the following U.S. Patents: 5,390,671; 5,391,250; 5,568,806; 5,586,553; 5,777,060; 5,779,665; 5,786,439; 5,851,197; 5,882,494; 5,954,643; 6,093,172; 6,293,925; 6,462,162; 6,520,326; 6,607,509; DES 433,755. The Glucose Sensor and Sen-serter are covered by other/pending patents. ©2006, Medtronic Minimed, all rights reserved. CGMS[®], MiniMed[®] and Sen-serter[®] are registered trademarks of Medtronic MiniMed. Guardian™ is a trademark of Medtronic MiniMed. IV3000[®] is a registered trademark of Smith&Nephew, Inc.
Medtronic B.V.