SUPERfeet[®]
THE PREMIUM INSOLE™

Congratulations.

You've just made a very comfortable decision.

Enjoy Superfeet Premium Insoles in
EVERY SHOE. EVERY DAY.

Superfeet Worldwide Inc
1419 Whitehorn Street
Ferndale WA 98248
USA

T 800 634 6618
F 800 320 2724
www.superfeet.com
customer@superfeet.com

**SUPPORT THE FOOT.
ALIGN THE BODY.**

Superfeet Easy-Fit help support and stabilize the feet, making your dress shoes fit better so you can avoid unnecessary discomfort or pain. The slim, lightweight design helps reduce pronation and allows the heel's fat pad to naturally cushion impact.

THE EASY-FIT™ DIFFERENCE
Patented biomechanical shape and design

EASY-FIT 3/4 HIGH HEEL

- Slim support for women's high heels **1" and higher.**
- Redistributes body weight off the ball of the foot.
- Improves stability and reduces fatigue, blisters, pump bumps and knee pain.

EASY-FIT WOMEN'S 3/4

- Improves fit for women's dress shoes **1" and lower.**
- Lightweight design provides biomechanical support.
- Improves alignment, shock absorption and comfort.

EASY-FIT MEN'S 3/4

- Improves fit for men's dress and casual shoes.
- Lightweight design provides biomechanical support.
- Improves alignment, shock absorption and comfort.

THE PROPER FIT
The key to a great fit is proper sizing

(TOO NARROW)

(CORRECT)

Check heel width. If the insole is too narrow, go up one size.

It's quick and easy to move your Easy-Fit insoles from shoe to shoe. Use only Easy-Fit High Heel in your heels!

Insole fit and comfort is influenced by model and size of shoe and insole, sock thickness, and lacing method.

Superfeet is not responsible for discomfort or pain caused by improper insole or shoe fit, latent pre-existing medical conditions, or non-insole related issues. If foot ailments develop or persist after the break-in period, please consult a healthcare professional.

EASY-FIT™

Quick fitting instructions

STEP 1: DON'T TRIM

Easy-Fit insoles are designed to fit in tighter-fitting, low-profile shoes and require no trimming.

STEP 2: INSERT

Look for the L (left) and R (right) on the bottom of each insole. Place the Easy-Fit insoles in your shoes, over the original factory insole or shoe liner.

For an ideal fit, remove any heel wedges or foam shapes in the shoe that will prevent the Easy-Fit from lying flat.

STEP 3: SECURE IN PLACE (optional)

To avoid excess movement, use enclosed Velcro® dots to hold insoles in place.

Attach a black dot to a white dot (A) and stick white dot to the bottom of insole (B). Remove backing paper from black dot and place insole into shoe, making sure the insole is positioned as far back as possible. Press firmly into shoe.

IF YOUR SHOE HAS A REMOVABLE INSOLE
If removing your shoe's original insole creates too much room in the shoe, check out our Trim-to-Fit™ Premium Insoles at www.superfeet.com.

HOW EASY-FIT™ SHOULD FIT

The insole should rest directly on the bottom of your shoe to give you the best fit and the most support. To help ensure a proper fit, make sure to use the correct Easy-Fit insole with the correct footwear! **The Women's High Heel insole needs to be worn in heels 1" or higher, and the Women's 3/4 (Low Heel) must be worn in shoes with a heel height of 1" or less.**

BREAK-IN PERIOD

The tendons and muscles of your feet may need time to adjust to their properly supported position. We suggest taking a conservative break-in approach; try wearing your Superfeet for just 1-2 hours the first day, and gradually add more time each day.

HOW TO CLEAN

Hand-wash Superfeet with mild soap, lukewarm water, and an old toothbrush with soft bristles. Rinse well. Remove excess water with a soft towel.

DO NOT MACHINE WASH OR DRY. DO NOT AIR DRY NEAR A HEAT SOURCE!

60

60 DAY COMFORT GUARANTEE

You have nothing to lose but your sore feet. If you are not satisfied with your new insoles, return them within 60 days to your place of purchase for a credit or refund.

Only valid with proof of purchase. Not valid for misused, abused or improperly cleaned Superfeet products. **THIS GUARANTEE IS SUBJECT TO THE RETAILER'S RETURN POLICY.**

WATERPROOF LINER DISCLAIMER

Most orthotics will conform to the heel in your footwear, and over time can wear through the lining. Superfeet offers no guarantee against this resulting side effect in footwear with waterproof liners such as Gore-tex®.

SUPERFEET SUSTAINABILITY

Our core values challenge us to "be socially and environmentally responsible at work and at home." We print our promotional materials on post-consumer paper with vegetable-based inks, seek out energy-efficient options whenever making business decisions, and have reduced the size and weight of our 100% recyclable packaging. [Find out more at www.superfeet.com.](http://www.superfeet.com)

View our complete User's Guide at www.superfeet.com or contact Customer Care at 800 538 5471 with any questions or concerns.

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