Pavlik

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User Manual ♦ Mode D'Emploi

Modo De Empleo ♦ Manuale D'Istruzioni

Pavlik

REF 23 400 ...

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◆ Description

With the Pavlik luxation bandage, the hip-knee bone flexion can be set to an angle of between 90 ° to 120 °. The bandage consists of a shoulder strap and a chest strap, and medial and lateral running leg straps with foot sections. Adjustments to the straps can be adapted according to the therapy course.

◆ Indication

- Hip-dysplasia
- Functional treatment of hip luxation

◆ Function

- Dynamic functional method

◆ Use

- For newborn and infants

◆ Safety

 Individual adjustment / adaptation of this aid by a doctor, therapist or an experienced orthopaedic technician according to the medical instruction given is absolutely necessary before use.

 Incorrect selection, adaptation, use, and/or lack of control of this aid can lead to health risks!

 Do not use this aid on damaged skin.

 Users of this aid (e.g. therapists, relatives, nursing staff etc.) must check this aid for damage before use. (e.g. loose connections, buckles, belts etc.).

 Users of this aid (e.g. relatives, nursing staff, etc.) must have received adequate training in the use and be instructed about possible risks involved in its use.

 Consult a doctor immediately regarding performance disruption (e.g. ineffective treatment).

 Never allow children to play with this aid. Danger of injury!

◆ Dimensions

| Pavlik REF | Size | Chest circumference | Foot size max. | Scope of delivery |
|------------|------|---------------------|----------------|---|
| 23 400 011 | S | 30 - 40 cm | 6 cm | Chest section size S / Leg section size S |
| 23 400 012 | S+ | 30 - 40 cm | 7,5 cm | Chest section size S / Leg section size M |
| 23 400 022 | M | 40 - 50 cm | 7,5 cm | Chest section size M / Leg section size M |

◆ Application

1. Fitting the Chest Section

1.1 Before initial use, the five buckles on the chest, shoulder and front leg belts should be opened. Lay the Pavlik Bandage open and stretched out on a suitable base (a nappy-changing table). The child should be positioned on its back, lying on the Pavlik Bandage so that the chest strap can be closed in a circular manner, a finger's width under the armpits whilst still above the nipples.

1.2 The shoulder straps should be crossed across the back so that the bandage does not slip over the shoulders.

2. Fitting the Leg Section

2.1 The leg straps are placed so that the longitudinal straps run parallel down both sides of the calves and over the inner and outer ankle. The belt part running underneath the foot must be positioned some centimetres in front of the rear edge of the heel, in the direction of the centre of the foot.

2.2. The leg straps running down the inner side are attached to the chest strap, so that a spreading of the upper thigh within the hip joints is possible to a degree of between 50° and 55° per leg.

 Important: The attachment points of the leg straps must lie below an imaginary line drawn between the armpits.

Following this, the outer leg straps are fastened to the rear buckles on the chest strap.

2.3 Should an increase in movement of the upper thighs be undesirable during the therapy course, the adjustable abduction inhibitor belt can be used. It has

an insert sleeve for exchangeable stages, with the help of which an undesired adduction of the hips can be limited. A shorter stage is already enclosed within the belt, a longer stage is enclosed with the Pavlik bandage. The stages can also be shortened when required. To guarantee a correct fit of the foot parts, the three fasteners for the instep, above the heel and around the calf should not be applied too loosely.

3. Final fixing of the straps and padded straps

3.1 All belts can be shortened using a pair of scissors. However, make sure to leave enough excess end. Keep in mind that the belts might need to be changed during the course of the therapy. After the final length of the belts is set, the ends with the hook and loop fasteners of the padded flaps must also be fixed.

3.2 Threading the belt during fitting.

3.3

3.3 Threading the end of the belt.

3.4

3.4 The belt end is positioned as a deposit of the buckle and is shortened. With the hook and loop fastener the padded flap on the belt end can be finally fixed.

3.5

3.5 Correctly threaded fasteners with underlying padded straps. The tube-shaped pads of the shoulder belts can be pulled as far forward and downward as possible over the buckles.

When required, the pads on the shoulder and leg straps with the REF 23.402.000.00 can be reordered.

◆ Material

Straps: PA velour

Pads: Velour foam

Foot sections: Subortholen, PA velour

◆ Product care Cleaning Maintenance

- Wash bandage straps by hand with mild detergent at 30 °C
- Do not iron
- Not suitable for tumble dryers
- Do not bleach
- Do not dry-clean

This aid is not intended for reuse by other patients!

Storage/Disposal: This product does not contain any environmentally damaging materials. It can be disposed of as part of the normal household refuse.

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Wilhelm Julius Teufel GmbH