D *Cedis SanDry Premium*

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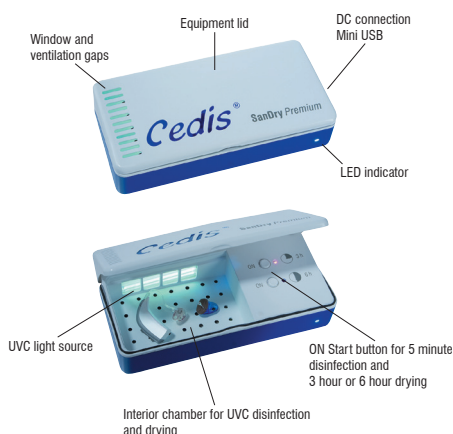
Product description:

Scope of delivery:

Cedis SanDry Premium
Power supply with USB connection cable
Operating instructions

Operating instructions:

Use the *Cedis SanDry Premium* after the daily cleaning of your hearing system with *Cedis* cleansing wipes or cleansing spray.

1. **Plug the Mini USB plug of the supplied 12 V power supply into the right-hand socket on the side of the equipment.** Now plug the power supply into a suitable power socket and your *Cedis SanDry Premium* is operational. With the application of the USB cable the drying case can also be operated with suitable equipment (PC/Laptop) with USB connections.

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2. Open the lid of the equipment.
3. Remove the batteries from your hearing aid prior to the drying. Place the hearing system in the interior chamber. **Hint: Leave the battery chamber of the hearing aid open for effective drying.**
4. By pressing the “ON” button 3 h or 6 h please choose the drying time for your hearing system. The chosen time will be confirmed with the lighting of the red LED diode. The lighting of the blue LED diode at the exterior of the equipment shows the operational readiness. **Hint: In case of high humidity or if you were perspiring extremely we recommend the 6 hour intensive drying.**
5. Close the lid of the equipment. The UVC disinfection commences for 5 minutes. Upon conclusion of the UVC disinfection the *Cedis SanDry Premium* automatically changes to the drying modus, which is indicated by the red LED diode. **Upon conclusion of the drying the equipment switches off automatically**, the red LED diode switches off. After opening the lid you may now remove your hearing system.
6. Please allow your hearing system to cool down to body temperature for approx. 3–5 minutes after drying in the *Cedis SanDry Premium*.

Hint: After every use please clean the interior chamber with a *Cedis* cleansing wipe.

The equipment has a safety switch-off during the UVC disinfection if the lid of the equipment is opened during operation. After re-closing of the lid the selected operation period commences anew.

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Important hints:

- Do not use solvents or other aggressive cleansing aids for the cleaning of your equipment.
- Do not clean the equipment under running water.
- Do not remove the power supply from the power plug by pulling on the cable.
- In case of damaged cable the immediate exchange is necessary.
- At any sign of damage or after inappropriate use please take the *Cedis SanDry Premium* to your dealer for a check-up.
- Use only for the drying of hearing aids and earmolds.
- The equipment emits heat downward.
- Do not operate on heat-sensitive surfaces.
- Do not cover the equipment.
- This equipment is exclusively designed for personal use and not for medical purposes.

Warranty: 1 year from date of purchase with appropriate operation. Please submit purchasing docket.

Attention: In case of inappropriate usage of the *Cedis SanDry Premium* all warranty and liability claims are excluded.

Customer service: In case of damages or default please contact your dealer.

 **SanDry Premium**

Bedienungsanleitung
Operating instructions
Mode d'emploi
Modo de empleo

GB *Cedis SanDry Premium*

The professional equipment for the gentle disinfection and drying of the entire hearing system, preserving the integrity of the hearing system.

Drying, protecting and disinfecting in one. Your hearing system is in good hands in a *Cedis SanDry Premium*. It keeps your hearing system functional over time and ensures a long operating life.

The regular exposure of your hearing system to UVC light offers optimal protection against viruses and bacteria – without liquid and additives. Even multi-resistant germs are deactivated by the UVC technology and thus become harmless. Moisture in the hearing aid, for example due to perspiration, will lead to premature deterioration of the high-quality electronics. The *Cedis SanDry Premium* initially disinfects and subsequently gently and safely dries the hearing aid and the earmold. This significantly increases the life span of your hearing aid.

Performance characteristics:

UVC light tube: 1.5 W
UVC wavelength: 253.7 nm (ozone free)
Power supply: Input 100–240 V – 50/60 Hz
Output 5 V – 0.5 A
LED display: LED-diode with color-change function (blue-red)
Outer dimensions: W 150 x D 72 x H 44.5 mm
Inner dimensions of interior chamber:
W 85 x D 53 x H 19 mm

egger Otoplastik + Labortechnik GmbH
Aybühlweg 59
87439 Kempten/Germany
Tel.: +49 (0)831 58113-20
Fax: +49 (0)831 58113-13
E-Mail: labortechnik@egger-labor.de
Internet: www.egger-labor.com

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