CONFIQUICK® Self-etch Adhesive

Light-cure 5 ml

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MS7174

- Dispense one or two drops of adhesive into a mixing well immediately prior to application or open the unit dose container by twisting the cap and dipping the brush into the bottom of the capsule immediately prior to application.
- Mix the adhesive with a brush and saturate the brush tip, then apply to the slightly wet enamel and dentin for 30 seconds, with agitation, thoroughly coating all surfaces to be bonded.
- Using oil-free, moisture-free air for about 10 seconds, allow volatile components to gently evaporate **but do not thin or blow away the layer of adhesive**.
- Light-cure the adhesive for 10 seconds before placing the composite (recommendation: quartz-tungsten-halogen light, minimum 500 mW/cm²). Do not allow any contamination of the cured adhesive surface.
- Place the composite in layers of about 2.5 mm or less. Refer to manufacturer's instructions for its placement, curing and finishing.

WARNINGS:

- Dentists and assistants should wear gloves, masks and protective eye wear. Patients also should wear eye protection.
- If accidental contact occurs with skin, flush immediately with copious amounts of water. For eye contact, flush immediately and for a prolonged period with copious amounts of water and consult a physician.
- Uncured monomers in this resin material may cause an allergic reaction or skin irritation in some individuals.
- Do not use in individuals with allergies to these materials.
- Any non-sterilizable item that is handled in the delivery of the dental service should be disinfected by standard dental office hygiene procedures.
- Do not allow saliva or water contamination of unset or set material or of any step in the subsequent restorative procedure after applying adhesive.
- Test the dental curing light periodically to assure sufficient intensity of the light output.
- For additional information consult the Safety Data Sheet.

STORAGE:

- Store between 59°F – 75°F (15°C – 24°C).
- Allow adhesive to come to room temperature before use.
- Do not freeze.
- Do not store in proximity to eugenol-containing products.
- Do not expose the bonding agent to elevated temperatures.
- Do not expose to intense light prior to application.

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Instructions for use

INDICATIONS:

CONFIQUICK Self-etch Adhesive is indicated for the bonding of direct composite, resin modified glass ionomers or compomers to dentin/enamel.

DESCRIPTION:

CONFIQUICK Self-etch Adhesive is a light-curing, one component adhesive resin system designed to bond composites to enamel and dentin. It is a quick and easy way to bond composites to the tooth. This self-etch adhesive provides “all in one”: etching of, priming of and adhesion to enamel and dentin. This system is responsive to any dental curing light; curing times can vary with the radiance of the light source.

FORMULATION:

Dimethacrylate based bonding system with photosensitizers.

DIRECTIONS FOR USE:

- Isolate the tooth or teeth to be prepared, preferably with a rubber dam.
- The tooth should be cleaned by scaling and prophylaxis with flour of pumice. Prepare cavity form according to normal dental procedures.
- Base as necessary. Preferably use a thin layer calcium hydroxide material on pulp exposures. The calcium hydroxide can be covered with a thin layer of glass ionomer base cement. Avoid use of any eugenol containing cements.

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