E

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**GB****User instructions:**

When attaching the brace, care has to be taken that the shells are laterally positioned exactly centrally at the shank.

Put the wide lower strap around the brace and fix Velcro closure. Pull forward the narrow strap below and fix Velcro closure.

Put the upper strap around the brace and fix Velcro closure. If required readjust lower strap to provide tight fit of the brace.

*If he tumefaction decreases, the Malleo-Stabil®-SOFT Brace might loosen. Readjust if required.*

**Use of foam component:**

Pull off foam component from below upwards. When reattaching the foamed component take care that it will be located exactly at the original place. The correct positioning assures that the plastic edges are covered.

**Indications:**

For stabilisation of ankle joint. For reducing the time spent in plaster. Ligament injuries, upper ankle joint sprains, supination and/or external rotation traumas. Fibular ligament rupture.

**Contraindications:**

Large-area skin injuries, new fractures in the upper and lower ankle area, disorders of the lymphatic system, large-area edemas, oversensitivity against one of the materials used.

**Composition:**

80% Polypropylene, 20% polyamide velour

**Important instructions:**

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove it if necessary. If complaints continue, please contact the doctor.

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