ORBIS Correction Mass® GB

Rev. 4/2005

PRODUCT	ISO 4823	COLOUR	DOSAGE			
			BASE PASTE		AKTIVATOR	
			PASTE	LIQUID	PASTE	LIQUID
green	Typ 1	grün	1 Teilstrich 0,61 ml = (0,85 g)	1/2 Teilstrich 0,09 ml = (0,095 g)	10 Tropfen 0,19 ml = (0,2 g)	
blau	Typ 2	blau	1 Teilstrich 0,58 ml = (0,75 g)	1/2 Teilstrich 0,09 ml = (0,095 g)	10 Tropfen 0,19 ml = (0,2 g)	
violett	Typ 3	violett	1 Teilstrich 0,49 ml = (0,59 g)	1/2 Teilstrich 0,09 ml = (0,095 g)	10 Tropfen 0,19 ml = (0,2 g)	

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Times given in the table above are for 23 °C and relative moisture of 50+/- 10% 1 Graduation Mark = (refers to mixing block)

CONDENSATION CURING PRECISION IMPRESSION MATERIALS ON SILICONE ELASTOMER BASE

Type 1 = heavy bodied – impressions for partial prostheses and mucous membrane impressions
Type 2 = medium bodied – syringe material for putty wash technique (2-step) and simultaneous Putty wash, material for the relining of prostheses.
Type 3 = light bodied – syringe material for putty wash technique (2-step) and simultaneous putty wash, material for the relining of prostheses.

Indications for Use:
The contrasting colouring of the base and catalyst masses ensures good control during mixing. Mix the correction material with the aid of a spatula.

Carefully close tubes after use.

Disinfection:
The impressions may be disinfected in a 2% solution of glutardialdehyde. Please refer to the manufacturer's instructions for use.

Galvanizing:
The impressions may be galvanized with copper or silver coatings.

Model Production:
Filling of impressions possible after 60 minutes. No further time constraints.

Storage: Do not store above 25 °C.

Shelf Life: See information on package

Working and Safety Instructions:
Avoid contact with eyes, if contact should occur, rinse immediately and copiously with water and consult an ophthalmic specialist without delay. Avoid staining of clothing, as the condensation curing impression materials are chemically resistant. We recommend wearing gloves when using kneadable impression materials. Responsibility for the processing, use and application of the products supplied by us lies entirely with you, in as far as there exists for us no possibility of controlling same. In cases where we have provided technical advice re application, either verbally, in writing or by means of trials, this advice has been without obligation both on our part and as regards any third party patent rights.

You are therefore not absolved from the obligation to test the products supplied for suitability and appropriateness for the intended purposes and procedures.

Our liability is limited in all instances and for all damages to the value of the goods supplied by us and used by you.

The impeccable quality of our products is guaranteed within the framework of our general Terms of Business.

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ORBIS°DENTAL Handels GmbH · D-48153 Münster