tempofit® N colours

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tempofit® N colours

GB
Instructions for use

Material for making temporary crowns and bridges, self curing, fast setting, bis-acrylic-composite 4:1

Fig. 1

1. Preparation of the impression

Before preparation or extraction, make a study or partial impression using alginate or silicone. For ease of repositioning, cut out excess interdental flash. Ensure adequate dimensions and if necessary expand relevant areas (Fig. 1). In the case of pontics a bar-shaped connection between the prepared teeth is produced by cutting a groove between the abutment teeth. Alginate impressions should be kept at 100% air humidity until the introduction of **tempofit® N colours**.

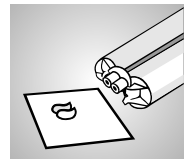
Fig. 2

2. Use

The cartridges are expressed using the mini-mix system. Place the plunger in the body of the cartridge. Remove the cartridge cap by twisting off. **Extrude a small amount of material until material is dispensed evenly from both openings (Fig. 2).** To attach the mixing cannula note the guides on the cannula and cartridge. Lock by turning them in opposite directions (Fig. 3). Proportioning can now be performed individually. Dispense the material with steady pressure. After use leave the mixing cannula on the cartridge until the next use. Use only green or brown 4:1 mixing cannulas. Use brown mixing cannulas for single and small temporaries and use green mixing cannulas for multi-units.

Fig. 3

3. Mixing and application

Mixing and portioning take place automatically when the two components are dispensed by pressing the plunger into the cartridge. First express a pea-sized amount onto a mixing block (Fig. 4). Discard this material. This applies for each new mixing. **tempofit® N colours** is now inserted directly in the deepest part of the preliminary impression and is then filled as far as the gingival regions (Fig. 5).

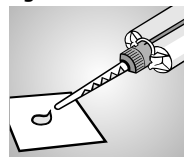
Fig. 4

4. Modelling of the temporary

Place the filled impression onto the prepared dies and remove excess material with a plastic implement. Approx. 1 min. 30 sec. – 2 min. after the mixing process **tempofit® N colours** has a hard elastic consistency and can be taken out of the mouth easily together with the study impression. The hardening process must be observed exactly as taking the impression of the temporary is possible only during the elastic phase. (Fig. 6).

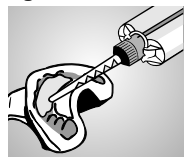
Fig. 5

5. Hardening and finishing

After removing the resin temporary from the impression, excess material and proximal undercuts are removed. After the appropriate strength is reached after approx. 5 – 6 min, the resin temporary can be processed with rotating instruments. The inhibition layer on the surface produced by the oxygen in the air, including the inner surface, must be removed with a solvent (e.g. ethyl alcohol or isopropyl alcohol).

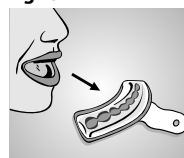
Fig. 6

6. Fastening the temporary


The temporaries can be inserted with the usual temporary cements, e.g. **tempolink®** (eugenol-free).

7. Repairing and making minor corrections

Repair is carried out with **smartrepair®** using light curing. Rinse and dry the temporary thoroughly. Roughen the surfaces to be treated and clean them and remove all grinding dust. Apply **smartbond®** outside the mouth with **smartbrush** and rub in, blow with compressed air and allow to dry for approx. 1 min. Apply **smartrepair®** and light cure for at least 20 sec. from all sides. Then complete the light curing outside the mouth by exposing to light on all sides for 30 sec. Finish the repaired areas and polish the temporary.

Important working hints

- To ensure a homogeneous mixing: Prior to insert the mixing cannula, **always** eject a small amount of material to ensure extruding **equal amounts** of base and catalyst **from both orifices** (Fig. 2).
- Store in a cool place at 2° C to 12° C / 35° F to 54° F!

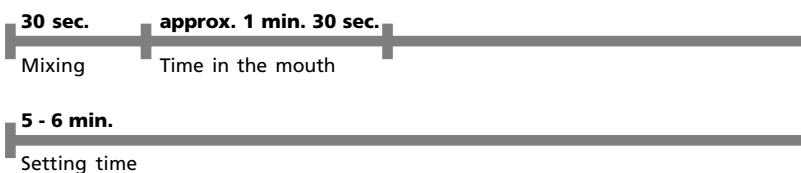Indications for use:

- Construction of crown and bridge temporaries
- Short- and long-term temporaries

Technical Data:

- **Mixed volume:** 16 g cartridge (10 ml)
- **Mixing ratio:** 4:1
- **Product colour:** BL, B1, A2, B3 (as indicated on package)
- **Mixing & application:** approx. 30 sec.
- **Time in the mouth:** approx. 1 min. 30 sec. (hard-elastic phase, from beginning of mixing)
- **Setting time:** approx. 5 - 6 min. (until setting is completed)
- **Application:** At room temperature (23°C ± 2°C / 73°F ± 4°F)

Characteristics:

- **Compressive strength:** 160 MPa
- **Flexural strength:** > 70 MPa
- **Transverse strength:** > 32 MPa
- **Barcol hardness:** > 28
- **Water uptake:** 10 µg/mm³
- **Maximal temperature reached during setting:** < 40 °C

Ordering information: tempofit® N Colours

Single packing BL 02576
1 x 10 ml mini-mix 4:1 mixing cannulas

Single packing B1 02530
1 x 10 ml mini-mix 4:1 mixing cannulas

Single packing A2 02574
1 x 10 ml mini-mix 4:1 mixing cannulas

Single packing B3 02569
1 x 10 ml mini-mix 4:1 mixing cannulas

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