Directions

1. Tear open packet and empty contents into beaker.
2. Fill beaker with at least 400 ml of warm tap water.
3. Place denture in beaker. Insert beaker into positioning cover in an ultrasonic machine.
4. Operate ultrasonic machine as usual for 3 - 5 min. or longer if necessary.
5. Remove denture and rinse with running water.
6. Dispose of solution in beaker. Sterilize beaker.

- Designed For One Cleaning Only.
- Discard Solution Promptly After Use.
- Do Not Add Any Ingredients To This Product Except Water.
- Do Not Use on Aluminum.

Store in a cool, dry place.

EMERGENCY and FIRST AID Procedures

Eye Contact: If this solution comes in contact with the eyes, flush with water for at least 15 minutes. Seek medical attention if irritation persists.

Skin Contact: If this solution comes in contact with the skin, wash thoroughly with soap and large quantities of water. Seek immediate medical attention if irritation persists.

Ingestion: If this solution is swallowed, DO NOT INDUCE VOMITING. Give large quantities of water. Seek immediate medical attention.

Inhalation: If breathing difficulties occur, move to fresh air. If symptoms persist, seek medical attention.

Protective Gloves: The use of impermeable gloves is advised to prevent skin irritation.

Eye Protection: Safety glasses are recommended to safeguard against potential eye contact, irritation or injury.

Spills: Flush with water. Remove contaminated clothing. Wash clothing before reuse.

Warning! For Professional Use Only. Not For Internal Use. Strong Acid. Keep Away From Children. For Ingredients and Additional Information, Refer to Material Safety Data Sheet.

Contains: Citric Acid (CAS#77-92-9), Fumaric Acid (#110-17-8) Sodium Dodecylbenzene Sulfonate (#25155-30-0)

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Hergestellt in den USA/Geproduceerd in de U.S.A.