Patella-Sehnenband Sport

Patella Tendon Strap Sport | Bande de fixation pour tendon rotulien Sport |
Soporte para tendón rotuliano Sport | Supporto per tendine rotuleo Sport

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With the BORT patellar tendon support pressure is placed on the tendon connected to the popliteal fossa (kneecap). This helps the kneecap to be channelled correctly during movement of the knee. At the same time, this pressure leads to increased tension in the muscles responsible for the movements of the popliteal fossa.

Instructions for use:

Open the strap completely and place it directly under the kneecap. Fasten the Velcro on the patellar tendon support so that there is sufficient pressure on the patellar tendon, but without creating stasis in the lower part of the leg.

Indications:

Patellar chondropathy, femoropatellar arthrosis, patellar tendonitis.

Contraindications:

Peripheral arterial disease (PAD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

84 % polyamide, 16 % spandex

Important instructions:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

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BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com