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Additional elements less than 1 %: Mn, N, Nb

Contains no nickel, beryllium or iron

Technical data

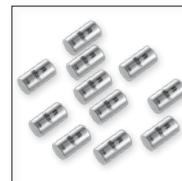
Yield strength*	R _{p0,2}	620 MPa
Tensile strength*	R _m	845 MPa
Hardness*	HV10	280
Fracture elongation*	A ₅	10.2 %
Modulus of elasticity*	E	190 GPa
Density		8.6 g/cm ³
Solidus		1320 °C / 2408 °F
Liquidus		1420 °C / 2588 °F
CTE (25 – 500 °C / 77 – 932 °F)		14.1 x 10 ⁻⁶ K ⁻¹
Color		white
Weight per ingot		approx. 6 g

* Variations of +/- 10 % are possible depending on the batch.

Use	CoCrW bonding alloy compatible with conventional ceramic materials with appropriate CTE range.
Melting	Torch, Induction (HF), Light arc Yes
Mirror like melt surface (HF)	
Casting time point (HF)	As soon as the molten mass begins to split
Pressure casting	250 – 450 mbar Observe instructions for use REF 989-448-00!
Welding	Possible using methods such as laser welding and TIG impulse welding
Additive material	CoCr welding wire, REF 528-215-00 (0.25 mm) REF 528-210-00 (0.35 mm) REF 528-200-50 (0.5 mm)
Presoldering	CoCrMo-Sold 1, REF 102-306-00
Flux	rema®-Flux 1, REF 102-304-00
Recommended ceramic	Ceramics with suitable CTE range such as Carmen® and CCS, from Dentaureum
Cooling of ceramics in the furnace	Slow cooling, unless otherwise recommended by the ceramic manufacturer

For further information see:**remanium®**, non-precious metal alloys for crown and bridge-work, instructions for use **REF 989-448-00**Available free of charge from Dentaureum or in the internet under www.dentaureum.de**Technical Hotline:****+49 72 31/803-410****remanium®**
star

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