Septoscell

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Septoscell

Zinc oxyphosphate cement for sealing

Composition

- . Zinc oxide-based powder.
- . Phosphoric acid solution.

Indications

Cementing : SEPTOSCELL can be used in a creamy consistency, without impairing its physical qualities.

Cavity lining : it is preferable, in this case, for an equal quantity of liquid to add powder until a putty like consistency is achieved.

Properties

Excellent adhesive qualities, volumetric stability, resistance to compression, microfine form, as well as a low level of acidity and solubility, make SEPTOSCELL an ideal cement for accurately sealing as well as for cavity lining.

The powder comes in a single shade : light yellow n°3 and should only be mixed with SEPTOSCELL liquid : **the use of other liquids, whatever the source, is strictly to be avoided.**

Instructions for use

It is advisable to use a thick, polished glass slab. Mix a level measuring-spoonful of powder with 5 drops of liquid. Incorporate the powder in the liquid, little by little, using a stainless steel spatula. **Mixing should be carried out in one minute thirty seconds.** Sufficient time is then available to place the **cleaned and dried** prosthesis on the **also dried** preparations. The setting time in mouth is 6 to 7 minutes.

As to cavity lining, mix a level measuring-spoonful of powder with 3 drops of liquid. Setting occurs in 5 to 6 minutes.

Warnings and precautions for use

SEPTOSCELL liquid is corrosive. Causes burns. In case of contact with eyes, wash immediately with plenty of water and consult a specialist. In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible).

The setting time is shortened when the quantity of powder to liquid is increased, when the mixing time is shortened or when the room temperature is raised.

Inversely, a reduction of the quantity of powder to liquid, a longer mixing time or a low surrounding temperature lengthen the setting time.

It is possible to control the effect of the surrounding temperature on the setting time by warming slightly the glass slab or by cooling it whilst avoiding misting.

Never replace in the bottle any excess powder remaining on the slab after mixing.

As the liquid is hygroscopic, take care to close tightly the bottle immediately after use ; shake the bottle prior to use before removing any powder.

Presentation

Kit containing :

- 1 vial of powder of 90 g,
- 1 vial of liquid of 42 g,
- 1 measuring-spoon,
- 1 mixing slab.

For professional use only in the practice of dentistry.