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GB

Back Supports support injured tissue structures (bones, ligaments and cartilages). Hereby the compression effect supports the holding function of the local ligament and muscular function, and thus relieves the vertebral column. Integrated stabilising elements provide additional protection against wrong movements.

Instructions for use:

Close the back support for the back at abdomen area by means of Velcro closure. The support should tightly fit, nowever not cramp. Use the finger loops for comfortable fit.

Indications:

Treatment of pain in the lumbar spine, low back pain, recurrent low back pain, osteochondrosis, spondylarthrosis, spondylolysis without spondylolisthesis, mild lumbar deformity (degenerative spondylolisthesis).

Contraindications: Peripheral arterial disease (PAD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

36 % Nylon, 24 % helanca, 20 % spun rayon, 20 % elasthan

Important instructions:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

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