Quickset

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Quickset

Quick setting temporary restoration material

CE 0459

Formulas

Powder :
Zinc oxide 80.00 g
Excipient q.s. ad 100.00 g

Liquid :
Eugenol 98.50 g
Excipient q.s. ad 100.00 g

Indications

- Temporary restorations.

Directions for use

Dosage : 1 spoonful of powder per 2 drops of liquid.

- Shake the bottle of powder before use so as to make sure that the density is homogeneous.
- Fill the gauged spoon without tamping the powder and level flush using a spatula.
- Put the powder on a glass slab.
- Pour two drops of liquid per level spoonful of powder.
- After depositing the liquid, recap the bottles of powder and liquid immediately.
- Using the spatula, mix the powder progressively with the liquid until an even consistency is obtained.
- Mixing should last about one minute.

Placing

- When preparing the cavity, take heed of the mechanical characteristics of the product.
- Isolate the operating field as usual.
- Dry the cavities with cotton swabs ; avoid long lasting air drying.
- Varnishes for cavities are not recommended.
- Place QUICKSET straight into the cavity.
- Tell the patient to close his/her mouth and to eliminate the excess of product.
- **The span of time elapsing between the onset of mixing and that of setting is about 5 minutes.**
- If the restoration needs to be trimmed, use a tungsten-carbide bur.
- A high temperature and/or humidity reduce the available working time.

Warnings and precautions for use

Liquid:

Xi – Irritant (contains eugenol).
Irritating to skin. May cause sensitization by skin contact.
Avoid contact with skin. Wear suitable gloves.

Powder :

N – Dangerous for the environment. Contains zinc oxide.
Very toxic to aquatic organisms, may cause long-term adverse effects in the aquatic environment.
Avoid release to the environment. Refer to special instructions/safety data sheets.

Presentations

Box containing :

- 1 x 42 g bottle of powder
- 1 x 13 ml bottle of liquid
- 1 spoon.

Storage

Store away from light and damp places.

For professional use only in the practice of dentistry.