CoxaFlex®

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User manual ◆ Modo de empleo

Manuale d'istruzioni

◆ Inhalt

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◆ Contents

- 8 Description
- 9 Indication
- 9 Contraindication
- 9 Side Effects
- 9 Function
- 9 Reimbursement Code
- 9 Important Notices
- 10 General Information
- 10 Safety
- 11 Measurements
- 12 Applying the Orthosis
- 13 Care Instructions
- 26 Notes

◆ Description

The advanced design of our flexion abduction splint CoxaFlex® is based on a treatment concept which has led to successful results for more than 25 years (Gekeler, 1986).

Consisting of an adjustable belt system, a continuously variable abduction splint and integrated leg braces, the CoxaFlex® centers the hip joints in a natural position of 90° to 100° of flexion and an abduction of 45° to 50° (see images A&B). By positioning the patient's thighs this way, any extension or adduction that might provoke luxation or dysplasia is avoided. Additionally, the orthosis prevents the extreme spreading of the thighs into the unnatural and harmful "frog position" (also known as the Lorenz position). Evasive movements leading to a wider flexion and rotating motions, however, are deliberately promoted.

Therefore, the flexion abduction splint CoxaFlex® allows for a precisely controllable, unconstrained adjustment of the lower extremities in flexion and abduction. It serves the centering of the femoral head with an instable or dysplastic hip joint structure (i.e. developmental hip disorder) under circumstances as natural as possible. CoxaFlex® helps maintain the foetal "seated squat position" recommended for newborns and infants and also complies with the natural squatting position of children huddling against their mother's body.

◆ Indication

Hip dysplasia, hip joints prone to luxation and reduced hips (i.e. set hip dislocations) in newborns and infants.

◆ Contraindication

Non-reduced hip luxation.

◆ Side Effects

Where applied correctly, no unwanted side effects have been reported for CoxaFlex® in more than 25 years.

◆ Function

CoxaFlex® reliably centers the hip joints in an adjustable, natural position. It maintains the muscles of the lower extremities relaxed while in resting position, and avoids extreme positions and circulatory problems of the joint. Additionally, the orthosis allows for widely unimpaired hygienic care and unproblematic radiographic depiction of the joint even while worn.

◆ Reimbursement Code

DE: HMV-Nr.: 23.05.02.0012

AT: Tarif Nr.: 35031 012

◆ Important Notice

The CoxaFlex® flexion abduction splint is not capable of reducing a luxated hip, but of centering and stabilizing the joint in a physiological position. To prevent relaxation, special attention should hence be paid to the hip joint's correct and reduced position when applying the orthosis.

In case of dysplasia (without the risk of luxation) the orthosis can shortly be taken off during the day (e.g. for hygienic care). This, however, should only be done with the explicit permission of the attending physician.

If worn over an opening romper, the orthosis does not have to be taken off when changing diapers. The splint's convex shaping in the belly area allows for the diaper to be easily removed.

◆ General Information

CoxaFlex® was developed in order to treat hip dysplasia and luxation more effectively and more safely. It helps control the growth of the cartilaginous and bony articulation partners, and specifically supports the ideal shaping of the hip joint. CoxaFlex® resulted from the close cooperation of physicians and technicians and will foster the subsequent development of the hip joint when applied correctly. As with many disorders, the prospects of a complete recovery of the dysplastic hip will improve if the defect is detected and treated early on.

When treated within the first days and weeks of life, regular dysplasia can usually be healed within a few weeks. Dysplasia stemming from a preceding luxation, however, usually takes several months of orthotic treatment to heal. Constant and regular control examinations (at first every 2 to 3 days) are as much part of a good treatment plan as a professionally fitted and correctly applied orthosis. Therefore, the close cooperation of physicians, technicians and parents is indispensable for the patient's healthy future.

Newborns and infants show a high degree of strong, reflexive, pedalling leg movements. Thus, children subconsciously reject any form of restriction to their mobility, and might show some resistance against the splint until fully accustomed to the new orthosis.

Should a patient seem to cry out of pain for an unusually long period of time, remove the orthosis and go to see a physician so the proper fit of the splint and the stability of the joints can be checked.

Children wearing the splint should preferably sleep on their backs.

◆ Safety

- Before using this device an individual adaptation of the orthosis and a familiarisation in its use by an experienced orthopaedic technician according to the medical instructions is essential.
- Incorrect selection, adaptation, application and/or failure to monitor the correct fit of this device can cause health damages.
- Never wear the orthosis on bruised skin.


Image A / B

- Check orthosis for damage (e.g. loose connections, fasteners, belts, etc) each time it is put on.
 - Should any performance malfunctions (e.g. apparent reduction in or lack of corrective action) be suspected, immediately call at an orthopaedist or an orthopaedic technician.
 - The orthosis must be checked for function, correct fit and possible damage by an orthopaedist or orthopaedic technician at regular intervals. The intervals between checks are to be individually determined for each patient.
- Should the product have been subject to disproportionately strong forces, it must be checked for possible damages by an orthopaedic technician before further use.
 - Avoid excessive heat accumulation. Wear cotton clothes under the orthosis if necessary.

The orthosis is not suitable for reuse, i.e. use by another patient.

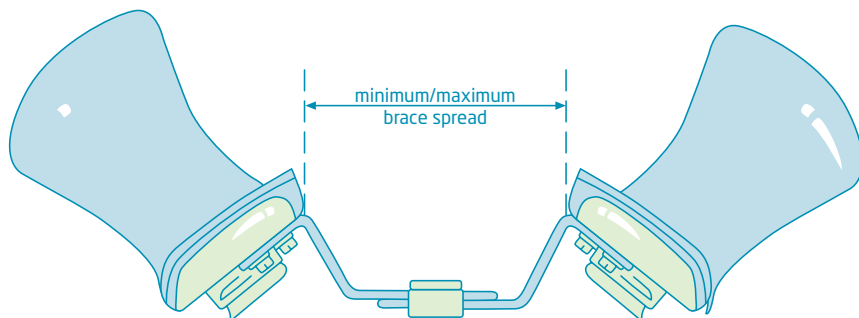◆ Measurements

CoxaFlex®			
REF	Size	brace spread min.	brace spread max.
23.300.000.00	XS	8,0 cm	13,5 cm
23.300.001.00	S	9,5 cm	16 cm
23.300.002.00	M	11 cm	17,5 cm
23.300.003.00	L	12,5 cm	19 cm
23.300.004.00	XL	17,5 cm	24 cm

◆ Applying the Orthosis

1. Put the orthosis on a flat surface with the blue neck part facing you.

Open the white buckle in the chest area and open the two front buckles on the leg braces.

Removable, washable leg cushions

2. Completely unfold the orthosis in front of you.

Removable, washable neck cushion

3. Lay the patient onto the belt system and put the straps over the infant's shoulders.

Place the child's thighs into the leg braces.

4. First, close the belt buckles on the leg braces, then the white buckle on the chest.

The flexion abduction splint is usually applied over the infant's romper. (Exception: With instable hips the splint should be worn under the romper, with tubular cotton stockinette placed underneath the orthosis to protect the skin.)

During the fitting, have a second person hold the child's legs so the hip joints are bent at 100° and are abducted at an angle of 50°. Now adjust the distance between the knees.

5. Position the closure of the chest belt as low as possible.

6. Adjusting and securing the belts.

Pull the belts out of the three-bar-slides and adjust them to the desired length before putting them back into the slides. The belts should be suspended in such a way that a flexion of 90° to 100° can be ensured even during strong pedalling motions.

7. Important Notice:

The knurled screw generally allows for a completely variable adjustment of the degree of abduction without using any tools.

To ultimately secure the desired adjustment, the knurled screw can be fastened using a coin. The widest brace spread can be achieved by extending the perforated rails to the maximum.

Please Note: Always verify that the buckles on the leg braces are securely locked.

◆ Care Instructions

- Neck and brace cushions are removable and should be washed with a detergent for delicate fabrics at 30°C.
- A spare neck cushion is included free of charge with every CoxaFlex®.
- Neck part, belts and spreading-unit should only be wiped down with a damp cloth.

**Do not reuse -
use for one patient only.**

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Wilhelm Julius Teufel GmbH