addition mini Junior

GB

Instructions for use

Precision ear impression silicone for children and adolescents

Specifically designed for the special needs of paediatric fittings. Soft, smooth consistency, completely non-stick, highly thixotropic (stable). No pressure build-up, no displacement of soft tissue, no unwanted run-off, easy to demould. Extreme tear-resistant, highly elastic recoverability, dimensionally stable. Skin-friendly & biocompatible. Easy to dispense from simple to guide minimix manual cartridges. Child-friendly: extremely short time in the ear and lemon scent.

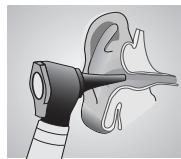
Fig. 1

Fig. 2

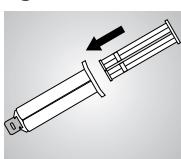
Fig. 3

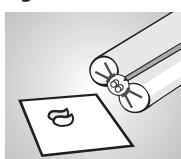
Fig. 4

Fig. 5

Fig. 6

Fig. 7

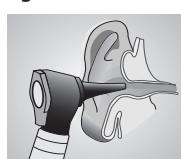
Fig. 8

1. Ear preparation

Before taking an impression, the auditory canal and the eardrum have to be examined (Fig. 1). If distinctive features are found (e.g. inflammations or perforation of the eardrum, tubes) impression must not be taken. The auditory canal has to be cleaned. In case of insufficient cleaning, remaining cerumen can cause retarded setting resp. inhibition of setting process.

It is essential to protect the eardrum prior to impression taking with an impression plug/cotton wool. Place the impression plug at the end of the outer auditory canal in front of the eardrum (Fig. 2) and check its correct position. The use of **impression plugs vent** (aired) is recommended.

Now **addition mini Junior** is injected directly into the prepared ear of the patient as described under point 2.

2. Mixing and applying

Dispensing is performed with the minimix system. Insert the plunger in the body of the cartridge (Abb. 3). If necessary, the mini mixing gun can be used. Remove the cap from the cartridge by turning. Carefully extrude a small amount of material to ensure proper flow from both orifices (Fig. 4). Insert mixing cannula into guide grooves on cartridge and turn clockwise to lock in position (Fig. 5). If required, a piece of silicone tubing (Flexi tube) can also be pushed onto the mixing cannula. Again, bleed small amount to ensure uniform and even material flow. Leave mixing cannula on cartridge after use, thus sealing the cartridge. Press out a small amount and discard.

3. Dispensing

addition mini Junior flows easily, thus allowing pressureless, constant filling. Slowly apply the impression material into the auditory canal until the concha is filled completely (Fig. 6). Keep the end of the cannula/tube immersed in the material during dispensing. Leave mixing cannula on cartridge after use, thus sealing the cartridge. After curing remove the impression carefully from the ear (Fig. 7). Finally, the auditory canal/eardrum must be examined for integrity (Fig. 8). The level of the minimix cartridge can be checked according to how far the plunger is protruding.

Important working hints

- Cured impression materials are chemically inert, pots on clothing should be avoided.
- Be sure to remove all traces of impression material from the patient's ear.
- Latex gloves and latex contaminated surfaces as well as cerumen, cremes and resins may inhibit the setting reaction of **addition mini Junior** (we recommend standard gloves made of nitrile rubber or polyethylene).
- Careful pre-examination and cleaning of the auditory canal are necessary prior to any impression taking (point 1).
- Careful post-examination of the auditory canal is necessary after every impression taking.
- For processing soft, high-elasticity silicone materials with a low final hardness (< 30 Shore A), special grinding sleeves are generally better than conventional metal milling machines. We recommend **Detax grinding sleeves**: Round, Ø 5 mm (Art. No. 03006, 10 pcs.) or Pointed, Ø 5 mm (Art.No. 03011, 10 pcs.).

Safety advice

The impression taking of the external auditory canal has to be executed by trained specialist staff only. The working instructions and precautionary measures have to be strictly observed. DETAX is not liable for any damage caused by improper application of the impression material.

Further information: Silicone based materials are proven a million times. On condition of a proper application, undesired effects are not to be expected. However, reactions of the immune system like allergies, irritations, cannot be absolutely excluded. In case of doubt, we recommend to make an allergy test before the application of the material.

For use by trained specialists.

Indications for use:

Ear impression with:
■ Infants
■ Adolescents

Technical data:

- **Mixing volume:** 10 ml (minimax cartridge)
- **Mixing ratio:** 1:1
- **Colour code:** Base: yellow catalyst: white
- **Application time:** approx. 30 sec.
- **Time in the ear:** approx. 2:00 min. at body temperature
- **Total setting time:** approx. 2:30 min.
- **Final hardness:** approx. 25 Shore A
- **Strain in compression:** approx. 10 % ± 1 %
- **Recovery from deformation:** 99,8 %
- **Linear dimensional change:** < 0,2 %
- **Application:** At 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5% rel. humidity
- **Storage:**

Ordering information:

Standard packing **03439**
4 x 10 ml minimix cartridges
8 mixing cannulas minimix, green
1 plunger, flowchart

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