MM-EDTA Cream

FR

EDTA en crème 19 %

NOTICE

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19% EDTA Cream

INSTRUCTION FOR USE

Easier Filing with Lubricating Action

APPLICATIONS

- Root canal preparation with files and reamers.
- Presence of calcification in the canal.
- Too narrow canals in posterior teeth.
- Smear layer removal.

MODE OF ACTION

- Removal of inorganic substances by chelation (EDTA-Ca 2+).
- Easier filling due its lubricating action.
- Produces the cleanest canal wall after smear layer removal.

COMPONENTS

- **MM-EDTA Cream** 7.0g (5.4ml) x 2.
- Disposable tips.
- Net weight : 14g.

DIRECTIONS FOR USE

1. Deposit a small amount of **MM-EDTA Cream** in the canal orifice.
2. Carefully pump into the canal with a fine file or reamer.
3. Re-apply **MM-EDTA Cream** when necessary or until cleaning and shaping is complete.

HOW TO USE ROTARY RING

- If in contact with air, content could be hardened, so it is necessary to cover a cap on the needle after use.
- Putting a syringe into a rotary ring is more convenient to handle & Use the rotary ring as a wrench to tighten or loosen a tip.

CAUTIONS

Use **MM-EDTA Cream** with discretion to prevent perforation.

Avoid extrusion of the solution into the apex.

Keep the syringe capped all the time.

STORAGE

Store at temperatures18°C~24°C (64°F~75°F).

SHELF LIFE

- 2 years from the manufactured date.

	Use by
	Batch Code
	CE-Mark and identification number of notified body.
	Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC
	Authorized representative in the EC
	Consult operating instructions
	Manufacturer

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METABIOMED CO., LTD.