# green eco

**High precision ear impression material, addition curing, very high consistency, with „snap set effect“**

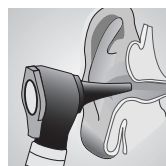
Fig. 1

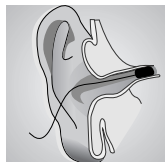
Fig. 2

Fig. 3

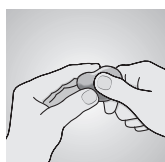
Fig. 4

Fig. 5

Fig. 6

Fig. 7

## 1. Ear preparation

Before taking an impression, the auditory canal and the eardrum have to be examined (Fig. 1). If distinctive features are found (e.g. inflammations or perforation of the eardrum, tubes) impression must not be taken. The auditory canal has to be cleaned and depilated. In case of insufficient cleaning, remaining cerumen can cause retarded setting resp. inhibition of setting process.

Before taking the impression, the eardrum must always be protected with an impression plug placed at the end of the external auditory canal in front of the eardrum (Fig. 2). Now **green eco** is injected directly into the prepared ear of the patient as described under point 2.

## 2. Mixing and Dosing

Remove impression material A (yellow) and B (green) using measuring scoops (Fig. 3). Mix together (mixing ratio 1:1) until a uniform colouring is achieved (Fig. 4). Fill the homogeneous mixture into an impression syringe and apply it slowly and bubble-free into the ear canal until the concha is filled completely (Fig. 5). Keep the end of the cannula immersed in the material during dispensing. After curing remove the impression carefully from the ear (Fig. 6). Re-examine the ear (Fig. 7).

## Important working hints

- Avoid mutual contact of components A + B.
- Do not interchange measuring scoops and lids.
- Do not combine with condensation curing silicones.
- Cured impression materials are chemically inert – spots on clothing should be avoided.
- Be sure to remove all traces of impression material from the patient's ear.
- Latex gloves and latex contaminated surfaces as well as cerumen, cremes and resins may inhibit the setting reaction of **green eco** (we recommend standard gloves made of nitrile rubber or polyethylene).
- A careful preparation (point 1) as well as a thorough cleaning of the auditory canal is required before taking an impression.

## Safety advice

The impression taking of the external auditory canal has to be executed by trained specialist staff only. The working instructions and precautionary measures have to be strictly observed. Non-observance might lead to irreparable damage of the ear or ear drum.

DETA is not liable for any damage caused by improper application of the impression material.

Further information:

Silicone based materials are proven a million times. On condition of a proper application, undesired effects are not to be expected. However, reactions of the immune system like allergies, irritations, cannot be absolutely excluded. In case of doubt, we recommend to get tested the allergenic potential before the application of the material.

## Indications for use:

- Ear impression taking

## Technical Data:

- **Mixed volume:**  
2 x 800 g (jar)
- **Mixing ratio:** 1:1
- **Colour code:**  
base: green  
catalyst: yellow
- **Mixing time:** 30 sec.
- **Setting time:**  
approx. 4 ½ - 5 min.  
at body temperature
- **Final hardness:**  
40 Shore A
- **Recovery from deformation:** >99.7 %
- **Application:**  
at 23 °C ± 2 °C / 73 °F ± 4 °F,  
50 ± 5 % rel. humidity
- **Storage:**

## Ordering information:

**green eco**  
Standard packing **02487**  
2 x 800 g  
base + catalyst  
2 measuring scoops  
green / yellow

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