smarthylocare®

GB

Instruction for use

Contains 0.1% low-molecular sodium hyaluronate from biotechnological production (not animal origin). The skin-friendly cream forms a thin film. Facilitates insertion and removal of earmoulds, increases wearing comfort and improves sealing. Skin regeneration (contact points) is promoted by the favourable properties of hyaluronic acid. Also suitable as preventive care for skin irritations caused by wearing earmoulds or spectacles.

Application:

apply a thin film of **smarthylocare®** on the otoplastic/ earmould. A very thin film of the cream can also be applied directly on thoroughly cleaned and dry skin.

Notes

- Facilitates cleaning of the earmould, possible cerumen residue remains soft.
- No discolouration of the earmould.
- Only apply **smarthylocare®** in a thin film.
- Do not close (block) openings of the earmould with cream.
- Only for external application. Do not bring into contact with eyes.
- Only apply to the outer skin areas of the ear.
- Can be used for 6 months after opening.
- Store dry.

General properties of hyaluronic acid:

Hyaluronic acid is a biopolymer produced by the body and is present in the human body and thus also the connective tissue. The special properties of hyaluronic acid, such as biocompatibility, hydrophilicity, viscoelasticity and target specificity (i.e. supportive tissue regeneration), make this molecule a multi-faceted substance with cosmetic as well as medical applications.

DETAX

GmbH & Co. KG
Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany
Telefon 07243/510-0 · Fax 07243/510-100
www.detax.de · post@detax.de

CE

10/2013

Fields of application:

- facilitates insertion/ removal of earmoulds
- improves sealing
- increases wearing comfort
- preventive care for skin irritations

Storage:

Contraindications:

Contains paraben. Do not use in known hypersensitivity to PHB-Esters (paragroup allergies).

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