discovery[®]
delight

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Instructions for use

Mode d'emploi

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D
DENTAURUM

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Instructions for use discovery[®] delight lingual brackets

Dear Customer

Thank you for having chosen a quality product from Dentaurem.

In order to use this product at its best for you and your patients, it is important to study and follow these directions for use carefully.

The written instructions cannot cover all eventualities during operation. For this reason our Hotline is available to answer any other questions and ideas that may arise.

Due to a constant development we recommend, even when you use the same products frequently, to study the enclosed latest issue of the instructions for use or refer to our website at www.dentaurum.de.

1. Manufacturer

Dentaurem GmbH & Co. KG · Turnstraße 31 · 75228 Ispringen · Germany

2. Working steps

2.1. Conditioning teeth

Carefully examine each tooth for enamel damages to avoid damaging the teeth during debonding. Follow the manufacturer's instructions of the bracket adhesive for conditioning the teeth.

2.2. Selecting bracket adhesive

discovery[®] delight lingual brackets can be bonded using all commercially available bracket adhesives. We recommend our light-curing ConTec adhesives or our MIX adhesives. Please follow the manufacturer's instructions for the bracket adhesive being used.

2.3. Removing and handling brackets

Grasp discovery[®] delight lingual brackets mesio-distally using clamping tweezers (e.g. REF 025-280-00) to remove them from the packaging and handle them according to the following steps. The patented laser-structured surface of discovery[®] delight lingual brackets should not be touched or contaminated in any way, as this could otherwise detrimentally affect the bond.

2.4. Applying bracket adhesive

Prepare adhesive according to the manufacturer's instructions. Apply a small amount of adhesive to the center of the bracket base, but enough to cover the entire bracket base. This guarantees optimal bond.

2.5. Placing brackets – Direct bonding

After applying the adhesive, place the discovery[®] delight lingual brackets in the usual way on the tooth. Align discovery[®] delight lingual brackets using the positioning tool (REF 023-278-00) and press the bracket in the usual way onto the tooth until the adhesive exudes from the sides of the bracket base.

Avoid contact with the antagonist due to the risk of abrasion and/or premature bracket loss. If necessary, use a palatal bite raise.

Completely remove the exuded excess adhesive before it has cured. This is particularly important to ensure easy debonding at the end of treatment.

2.6. Placing brackets – Indirect bonding

After applying the adhesive, place the transfer tray with the discovery[®] delight lingual brackets as usual in the mandible and/or maxilla. Completely remove the exuded excess adhesive before it has cured. This is particularly important to ensure easy debonding at the end of treatment.

2.7. Ligating arch wires

Once the adhesive has cured (observe adhesive hardening times in the manufacturer's instructions for use), ligate as usual with wire or plastic ligatures.

2.8. Debonding brackets

The excellent bonding properties of discovery[®] delight lingual brackets require special attention during debonding:

For the debonding of discovery[®] delight lingual brackets, we recommend using Weingart pliers (REF 003-120-00). Grasp the bracket mesio-distally by the wings and remove the bracket from the tooth with a short turn to the right or left side without pulling.

3. Information about the composition

Information about the composition is included in the materials list, see www.dentaurum.de.

4. Safety information

The product should not be used, if there is a known allergy to one or more of the ingredients.

5. Storage and shelf-life

Store discovery[®] delight lingual brackets in closed packaging.

6. Information for single-use products

discovery[®] delight lingual brackets are intended for single use only. Due to excessive mechanical stress during treatment and debonding, the bracket's dimensional exactitude and bonding properties can no longer be guaranteed when reused.

Reconditioning of discovery[®] delight lingual brackets (recycling) and reuse on patients are not permitted.

7. Additional tips

- Please review the technical information with reference to torque, angulation, size, etc. on the product label or in our catalog.
- There are no particular precautions to be taken when disposing of discovery[®] delight lingual brackets.
- Our orthodontic hotline is available should you have any questions regarding the fixed appliance technique: +49 72 31 / 803 - 550.

8. Delivery program

For the complete range of discovery[®] delight lingual brackets, please see our Orthodontics Catalog.

9. With reference to our quality standards

Dentaurem ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

10. Explanation of symbols used on the label

Please refer to the label. Additional information can be found at www.dentaurum.de (Explanation of label symbols REF 989-313-00).

Date of information: 06/14
Subject to modifications