E**Modo de empleo:**

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GB

User instructions

When attaching the brace, care has to be taken that the shells are laterally positioned exactly centrally at the shank. Put the wide lower strap around the brace and fix Velcro closure. Pull forward the narrow strap below and fix Velcro closure.

Put the upper strap around the brace and fix Velcro closure. If required readjust lower strap to provide tight fit of the brace.

If the tumefaction decreases, the MalleoStabil® Brace might loosen. Readjust if required.

Use of gel component

Pull off gel component from below upwards. When reattaching the gel component take care that it will be located exactly at the original place. The correct positioning assures that the plastic edges are covered.

For cryotherapy

- Remove gel component and put it for one hour into the deep-freeze. Note: gel will not become solid.
- Reattach gel component at the original place.

- Get refrigerated gel component not directly in touch with skin.
- For maximum wearing comfort always wear socks.

Foam component

The gel cushions may be replaced by the attached foam cushions if necessary – concerning this please absolutely take care of the doctor's instructions.

Use of foam component

Pull off foam component from below upwards. When reattaching the foamed component take care that it will be located exactly at the original place. The correct positioning assures that the plastic edges are covered.

Indications:

For stabilisation of the ankle joint. For reducing the time spent in plaster. Ligament injuries, upper ankle joint sprains, supination and/or external rotation traumas. Fibular ligament rupture.

Contraindications:

Large-area skin injuries, new fractures in the upper and lower ankle area, disorders of the lymphatic system, large-area edemas, oversensitivity against one of the materials used.

Composition:

80% Polypropylene, 20% polyamide velour

Important instructions:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove it if necessary. If complaints continue, please contact the doctor. CAUTION: If the gel components should leak and the gel comes into contact with the skin, wash the area immediately with soap and water and rinse well. In case of eye contact, rinse the eye under running water for several minutes holding the eyelid wide open. After swallowing, rinse the mouth and drink plenty of water. Consult a doctor if feeling unwell.

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