ASPIJECT® SAFE

Fingergreb til sikkerhedssprøjte
 Handtag för säker injektionsspruta
 Handle for Safe Injection Syringe
 Handgriff für Sicherheitsspritze
 Handgreep voor veiligheidsinjectiespuit
 Poignée pour seringue protectrice
 Empuñadura para jeringuilla de seguridad
 Impugnatura per siringhe di sicurezza

(DK) Vejledning
 (S) Bruksanvisning
 (GB) Instructions for use
 (D) Gebrauchsanleitung
 (NL) Gebruiksaanwijzing
 (F) Mode d'emploi
 (E) Modo de empleo
 (I) Modalità d'uso

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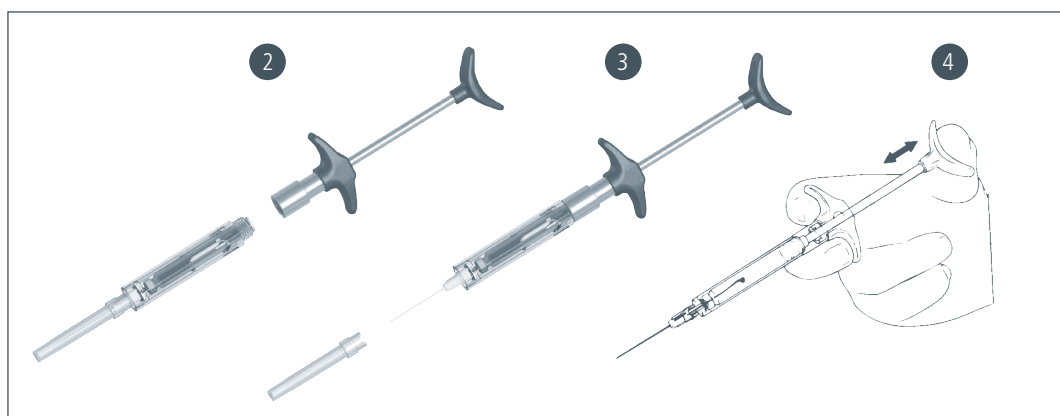
(GB) This product is intended for professional use in dental practice.

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DESCRIPTION

Item no. 2087 ASPIJECT® SAFE
 Handle is to be used with Protective Barrels, item no. 2098 or 2099.

- Item no. 2098: Disposable Protective Barrels for imperial needle thread (7/32") (USA, UK & Commonwealth)
- Item no. 2099: Disposable Protective Barrels for metric needle thread (M6) (Europe) The Protective Barrels come in boxes of 100 pcs. and take both 1.8ml and 2.2ml cartridges.

INSTRUCTIONS

1. Follow the instructions of the protective barrels concerning assembly, loading and function of the barrels
2. Screw the loaded barrel onto the autoclaved ASPIJECT®SAFE Handle.
3. Remove the needle sheath and ASPIJECT® SAFE is ready for injection.
4. Aspiration is accomplished by easing the pressure on the piston (self-aspirating).

MATERIALS

The ASPIJECT®SAFE handle is made from high grade (acid resistant) stainless steel and heat resistant engineering polymer.

CLEANING AND DISINFECTION

Use water and detergent or disinfect in dental dishwasher (= thermo-disinfect).

STERILIZATION

The Handle can be sterilized by any method in autoclave by max 200°C/392°F.

GUARANTEE

The ASPIJECT®SAFE Handle is guaranteed for 5 years. Damages due to inadequate care or incorrect handling are not covered by the guarantee.

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