180 mm

130 mm

INSTRUMENTS DENTO-VIRACTIS 55

DÉTERGENT PRÉ-DÉSINFECTANT DE L'INSTRUMENTATION
PRE-DISINFECTANT DETERGENT FOR INSTRUMENTS
DETERGENTE PREDESINFECTANTE DEL INSTRUMENTAL
DETERGENTE PRÉ-DESINFECTANTE DOS INSTRUMENTOS

10-01R/DV/2007-03

2 KG

175 mm

DENTO-VIRACTIS 55 INSTRUMENTS

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[GB] Pre-disinfecting detergent for the decontamination of dental and medical instruments prior to sterilisation. Eliminates rapidly blood, pus and proteins prior to its enzymatic action.

DIRECTIONS :

- Dilute Dento-Viractis 55 in hot tap water (30-60°C) (5g per litre of water : 0.5% dilution). The graduated measuring device supplied with each bucket allows to prepare bath of 3 L, 5 L or 10 L of solution.
- Completely immerse the soiled instruments for 15 minutes.
- Rinse thoroughly with tap water.

The instruments don't have to be cleaned before immersion in the Dento-Viractis 55 bath. The solution should be renewed after each use.

ACTIVE INGREDIENTS :
Didecylidimethylammonium chloride – Surfactants – Proteolytic enzyme.

STORAGE :
Store in a dry place. Keep away from cold.

PRECAUTIONS :
Contains Sodium carbonate, Didecylidimethylammonium chloride, Non ionic surfactants. • R36/38 : Irritates the eyes and the skin. • S24/25 : Avoid all contact with the skin and eyes. • S26 : In case of contact with the eyes, rinse thoroughly with water and consult a specialist. • S37 : Wear appropriate gloves. • S60 : Eliminate the product and its container like an hazardous component.

Tests for the determination of antimicrobial spectrum

Bactericidal activity	NF EN 1040 NF EN 13727 Dirty conditions AFNOR XPT 72-301/171 Dirty conditions 4 strains
Fungicidal activity	Active on Candida albicans according to European Norm NF EN 1275 (partial test)
Virucidal activity	Tested on HIV 1 (Lille Pasteur Institute)

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60, rue Boursault 75017 PARIS Tél.: 01 47 42 81 81 • Fax: 01 47 42 82 02	
REF. ENJOY DESIGN:	DENV0407_DENTOVIRACTIS_55_2KG
TITLE	DENTOVIRACTIS
SUBTITLE	55
DIMENSIONS	175 x 130 x 180 mm
ml / g	2 KG
Contact	Samy
MARKETING APPROVAL	
<p>*Do not match printed material to the colors & the simulated background patterns of this electronic document, they are just for layout purpose. USE INSTRUCTIONS BELOWS. *Do not match printed material to the colors & the simulated background patterns of this electronic document, they are just for layout purpose. USE INSTRUCTIONS BELOWS.</p>	
ARTWORK	Vincent M. 28/03/07
COLORS QUADRI 	
PANTONE 	

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