N NETHERLANDS - Rentwood badzit

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Manufactured for Patterson Medical Ltd.
Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK: Tel: 08444 124 330 Fax: 08448 730 100
www.pattersonmedical.co.uk
International: Tel: +44 1623 448 706
Fax: +44 1623 448 784

Patterson Medical Products, Inc.
28100 Torch Parkway Suite 700,
Warrenville, IL 60555-3938
Tel: 1-800-323-5547 Telefax: 1-800-547-4333

Patterson Medical Australia
PO Box 1006, North Ryde, NSW Australia 2113.
Tel: 1300 473 422 Fax: 1300 766 473

Patterson Medical France
Zone Industrielle de Montjoly,
11 Avenue de l'Industrie - CS 40712
08013 Charleville-Mézières Cedex, France.
Tel: 03 24 52 61 15 Fax: 03 24 52 90 34

PATTERSON
MEDICAL

Rentwood Bath Seat

09 107 2971 / AA1200

These user instructions are available in large print, please contact Patterson Medical Customer Services for a copy.
Please retain these instructions for future reference.

Intended uses

- compensate for a disability by providing a stable sitting surface.

Instructions for use

- The seat is shipped assembled to provide a high sitting position, with the longer leg portion (white feet) down, as shown in A.
- If a lower seating position is desired, detach the leg assemblies as shown in B. Then turn the legs so the shorter ends (dark feet) are down and reattach them as shown in C.
- The weight of the user causes the seat to wedge between the sides of the bathtub. If necessary, the width can be adjusted by removing the wing nuts (B) and moving the attaching bolts to a different width-adjusting hole (labelled a in B).
- If the seat is used in a plastic bathtub, the Rentwood Universal Paddle Caps (09 107 3063 / AA1230, shown in D) are recommended. They should be used on the lower legs to spread the weight load. To attach them, simply pull the feet off of the legs and replace them with the paddle caps.

Care and cleaning

The seat can be cleaned with warm water and mild detergent or household disinfectant.

Cautions

The bath seat is not to be used to support more than 95kg / 210 lb. / 15st.

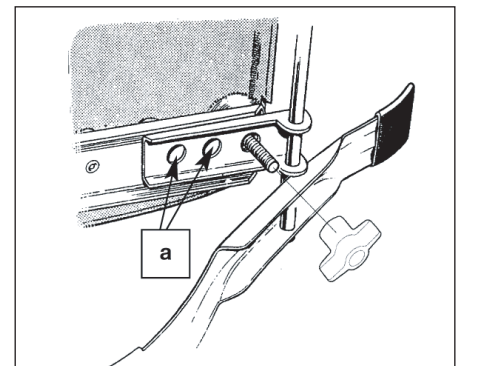
Be sure the wing nuts are tight before each use.

Be sure the seat is securely in place before putting weight on it.

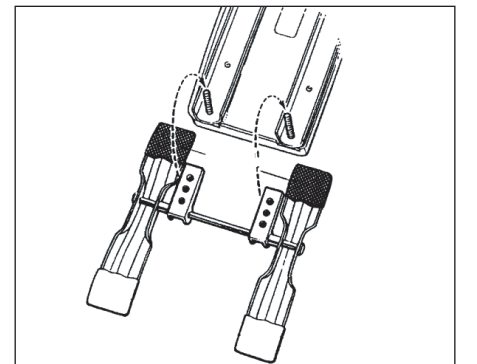
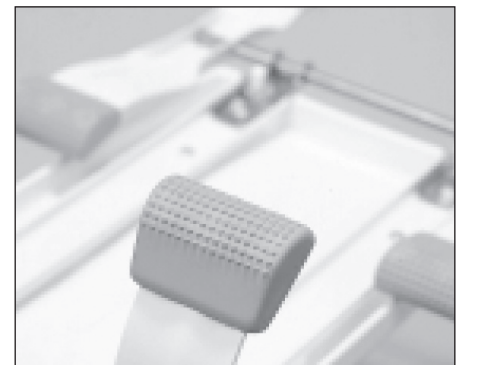
Cover the seat with a towel to help reduce slipperiness.

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