EuroSept® Max Impression Liquid

Quick-acting, ready-to-use, immersion disinfection solution for use on impression materials (silicone alginate and polyether-based materials)

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Distributed by (in US only)
HENRY SCHEIN Inc.
(manufacturer)
135 DURYEA ROAD
Melville, NY 11747 USA

HENRY SCHEIN U.K. HOLDINGS LTD.
MEDCARE HOUSE, CENTURION CLOSE
GILLINGHAM BUSINESS PARK
GILLINGHAM, ME8 0SB U.K.
TEL: +44 870 849 0872
www.henryscheinbrand.com
Rev. 2015/01

5L e

CE 0473

Instrucciones de dilución y modo de uso: Pour alginate materials, immerse briefly and allow to dry for 5 minutes. For other materials, immerse and leave in the solution for 5 minutes. HS Impression Liquid disinfects without affecting dimensional accuracy or surface quality. **Range of action:** bactericidal, levurocicide, effective against enveloped viruses (including HIV, HBV and HCV) – 5 minutes. **Composition:** 100g contains 0.98g glyoxal, 0.98g glutaral. **Hazard notes and safety advice: WARNING:** Causes skin irritation. May cause an allergic skin reaction. Causes serious eye irritation. Avoid breathing vapours. Wear protective gloves/ eye protection/ face protection. **IF ON SKIN:** Wash with plenty of soap and water. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/ attention.

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