D Cedis SanDry Eco

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Operating instructions:

Use the Cedis SanDry Eco after the daily cleaning of your hearing system with Cedis cleansing wipes or cleansing spray.

1. Place the supplied 3 batteries into the battery compartment at the underside and ensure the correct poling of the batteries.
2. Open the lid of the equipment.
3. Remove the batteries from your hearing aid prior to the drying. Place the hearing system in the interior chamber with a drying capsule. **Hint: Leave the battery chamber of the hearing aid open for effective drying.**
4. Press the “ON” button.
5. Close the lid of the equipment. The UVC disinfection commences automatically for 5 minutes, the disinfection modus is signaled by the blue LED diode. Upon conclusion of the UVC disinfection the blue LED diode switches off and the disinfection is completed. The manual drying should occur for a further 8 hours. The capsule can be re-used until the gold-yellow color indicator has turned whitish.

Attention: Do not open capsule! Do not swallow capsule!

After opening the lid you may remove your hearing system.

Hint: After every use please clean the interior chamber with a Cedis cleansing wipe.

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Bedienungsanleitung
Operating instructions
Mode d'emploi
Modo de empleo

GB Cedis SanDry Eco

The equipment for the gentle disinfection and drying of the entire hearing system, preserving the integrity of the hearing system.

Drying, protecting and disinfecting in one. Your hearing system is in good hands in a Cedis SanDry Eco. It keeps your hearing system functional over time and ensures a long operating life.

The regular exposure of your hearing system with UVC light offers the utmost protection against viruses and bacteria – without liquids and additives. Even multi-resistant germs are deactivated through the UVC technology and thus become harmless. Moisture in the hearing aid, for example due to perspiration, will lead to premature deterioration of the high-quality electronics. The Cedis SanDry Eco initially disinfects and subsequently gently and safely dries the hearing aid and the earmold. This significantly increases the life span of your hearing aid.

Performance characteristics:

UVC light tube: 1.5 W
UVC wavelength: 253.7 nm (ozone free)
Power supply: 3 x 1.5 V LR03 batteries (AAA/Micro)
LED display: blue LED-diode
Outer dimensions: W 150 x D 72 x H 44.5 mm
Inner dimensions of interior chamber:
W 85 x D 53 x H 21 mm

The equipment has a safety switch-off during the UVC disinfection if the lid of the equipment is opened during operation. After re-closing of the lid the UVC disinfection commences anew.

Important hints:

- Do not use solvents or other aggressive cleansing aids for the cleaning of your equipment.
- Do not clean the equipment under running water.
- At any sign of damage or after inappropriate use please take the Cedis SanDry Eco to your dealer for a check-up.
- Use only for the drying of hearing aids and earmolds.
- Please dispose of batteries appropriately.
- This equipment is exclusively designed for personal use and not for medical purposes.

Warranty: 1 year from date of purchase with appropriate operation. Please submit purchasing docket.

Attention: In case of inappropriate usage of the Cedis SanDry Eco all warranty and liability claims are excluded.

Customer service: In case of damages or default please contact your dealer.

egger Otoplastik + Labortechnik GmbH
Aybühlweg 59
87439 Kempten/Germany
Tel.: +49 (0)831 58113-20
Fax: +49 (0)831 58113-13
E-Mail: labortechnik@egger-labor.de
Internet: www.egger-labor.com

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