## FR

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## GB

Provide strong stabilization for ankles with the rubber plate inserted in the malleolus area.

### MATERIAL

Neopren group: 3 mm (neoprene 80 %, nylon jersey 20 %).  
Woven group: Lycra woven textile (55 % polyamide, 45 % elastan).

### IMPORTANT ADVISES AND WARNINGS

- This product is not burn protected. It will lose its characteristics by high temperature. Therefore do not use above 248°F.
- If you want to make changes for your comfort, please ask to your doctor or where you purchased the product.
- this product can not be used out of its own utility and cannot be combined with other products, only use with its original utilities.
- Usage of neoprene products is not allowed for more than 3-4 hours.
- In any case, do not use more than 8 hours (unless otherwise specified by your doctor). Please only use with doctor advice.

### WASHING INSTRUCTIONS

- Please close the velcro bands if there are and take out the removable support (like joint parts...).
- Do not put in washing-machine. Only handwash it at 104°C.
- Do not use bleach.
- No dry-cleaning.
- Do not use dryer.
- No ironing.
- After the wash please be sure that it is totally dry before the use.

### UTILITY OF THE PRODUCT

- Please be sure the size of the product is exactly your size.
- Open the velcro and put the product on your ankle, making sure the heel is well placed in the space provided.

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