Federbolzenschrauben

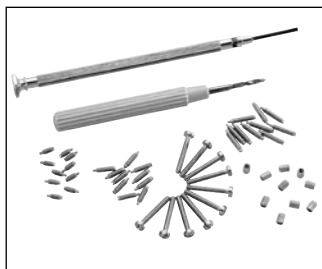
Piston spring screws

Vis-poussoir

Microtornillos con resorte

Viti a bullone

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Instructions for use

Mode d'emploi

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Federbolzenschrauben

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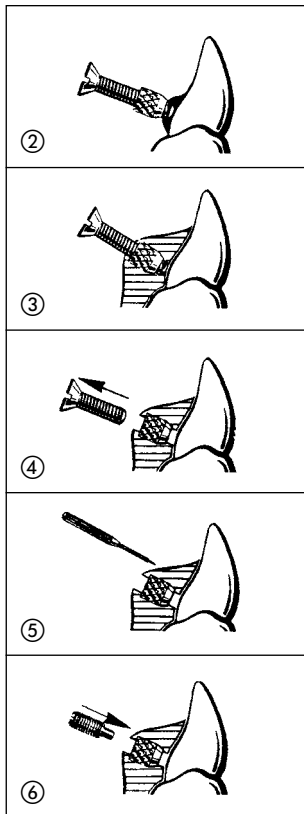
Instructions for use

Dear Customer

Thank you for having chosen a quality product from Dentaureum.

In order to use this product at its best for you and your patients, it is important to study and follow these directions for use carefully.

The written instructions cannot cover all eventualities during operation. For this reason our Hotline is available to answer any other questions and ideas that may arise.

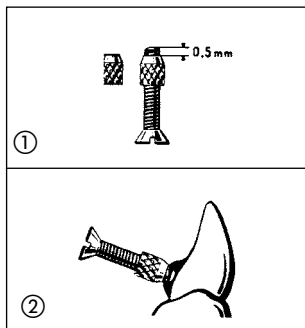
Due to a constant development we recommend, even when you use the same products frequently, to study the enclosed latest issue of the instructions for use or refer to our website at www.dentaureum.de.

Manufacturer

Dentaureum GmbH & Co. KG · Turnstraße 31 · 75228 Ispringen · Germany

With reference to our quality standards

Dentaureum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

Orthocryl® salt-and-pepper technique

The mounting screw is inserted into the serrated housing, with 0.5 mm projecting beyond the conical end (Fig. 1). The mounting screw must be positioned correctly against the tooth and held with sticky wax (keep the model dry to ensure adhesion of wax), Fig. 2.

Alternatively, a 1.5 mm bur can be used to drill a 2 mm deep hole in the tooth on the model and insert the mounting screw into this. The housing should be approx. 0.5 mm away from the surface of the tooth.

Soak the model for 10-15 min in luke warm water to release all entrapped air, then apply separating medium and use salt-and-pepper method. After processing is completed, the mounting screws are removed, the plate lifted from the model and polished, and then the drill holes are cleaned out with the reamer (REF 609-300-00).

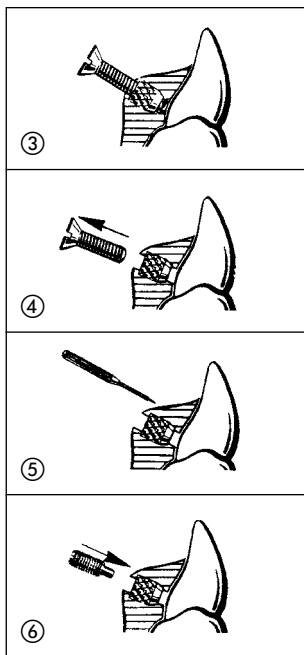
Finally the selected piston spring screw is inserted. Starting with the 4 mm screw, this is exchanged later for the 6 mm and finally the 8 mm screw if the tooth under treatment has to be moved further.

Use with heat curing process

The housing with the mounting screw is fixed in the correct position with sticky wax and the housing is covered with wax. The mounting screw keeps the housing in the correct position. Pack with acrylic and process. Deflask and remove the mounting screw (which can be used repeatedly). Remove the plate and polish.

Note:

The screw-driver REF 609-400-00 is not a medical device and therefore not suitable for the use in the mouth.

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Dentaaurum GmbH & Co. KG · Turnstraße 31 · 75228 Ispringen · Germania