ATHENA CHAMPION®

Athena Champion High Speed Turbine (Canister Style)

Item Number: 61900

- Replacement Turbine for Athena Handpiece
- Standard or Fiber Optic
- High-quality and Made in USA
- Replaces Pana-air type cartridges

INSTALLATION INSTRUCTIONS

Read these instructions before proceeding to replace the turbine.

Before replacing the turbine, carefully examine handpiece head for dents. If there are any dents in the head, return the entire handpiece to your dealer for repairs. **Do not install** replacement turbine into a dented handpiece. Installation will permanently damage turbine.

1. Remove cap by turning counterclockwise, using the cap wrench.
2. Remove worn turbine by pressing the blunt end of an inserted shipping post or bur against a hard surface to expel turbine.
3. Remove all foreign material and debris from interior of handpiece head.
Lubricate interior of head with Champ-Lube™ 20+.
4. Insert new turbine cartridge into handpiece head. When replacing with canister style cartridge, the index ball on the canister must be lined up with concave groove inside handpiece head to ensure proper fit.
5. Replace cap using fingers to prevent cross threading. Once seated, tighten cap with cap wrench.
6. Install bur securely into handpiece and operate per handpiece instructions.
Do not operate without a bur!

INSTRUCTIONS FOR USE /
MODO DE EMPLEO / MODE D'EMPLOI /
GEBRAUCHSANWEISUNG / BRUKSANVISNING:
www.athenachampion.com

continued

Sterilization Procedures

Remove the bur and any attachments from the handpiece. Manually clean and remove all visible soil, such as saliva from the handpiece prior to sterilization. Insert the handpiece into a sterilization pouch. Place into a steam autoclave. Sterilize at 135°C (275°F) per autoclave manufacturer's parameters. *Do not let the handpiece remain in pouch under wet or damp conditions. Such conditions can result in corrosion of internal parts and shorten the life of the handpiece. Do not use cold sterilization solutions containing acids or chlorine. Do not immerse the handpiece in water or disinfection solutions. Do not place the handpiece in an ultrasonic bath.*

Lubricate Before Use

After sterilization and cool down time, insert bur and tighten with bur tool. Lubricate one more time with 1 or 2 drops of Champ-Lube 20+ and briefly operate to expel excess lubricant.

Warranty: This part is warranted to the original purchaser against defective workmanship and materials under normal installation, use and service for six months. For warranty repair, send the product with copy of the original invoice to your dealer.

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YI Europe, Ltd., Box 7037