KIT EASYFEEL by Adip’Sculpt (FR)

DESCRIPTION

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KIT EASYFEEL by Adip’Sculpt (EN)

DESCRIPTION

The kit is sterile and double packaged, and contains:

- 4 cap closures, 1 aspiration cannula, 2 injection cannula, 1 infiltration cannula, 2 Luer-Lock transfer connectors, 4 Luer-Lock stoppers.

INDICATIONS

The kit is indicated for use in the fields of reconstructive or cosmetic surgery in order to increase the volume of intra and sub-dermal space deficiencies, in all parts of the body via the addition of adipose tissue.

The use of the kit is more particularly indicated in facial reconstruction, in particular in the case of light to moderate lipodystrophies or for small scars

The principal indications are:

- treatment of light to moderate facial lipodystrophies,
- correction of small scars anywhere on the body, except for the breasts.

CONTRA-INDICATIONS

The use of the EASYFEEL kit is contra-indicated:

- in pregnant women,
- in individuals with a known allergy to one of the components of the kit,
- in individuals whom a surgical act under anaesthesia is contra-indicated.

It is the responsibility of the practitioner to ensure that the patient is a suitable candidate for the use of the kit. The device is single use and as such it is forbidden to reuse or re-sterilize the kit. The possible principal risks associated with the use of this kit are: infection, various complications related to potential infection, death of the patient.

UNDESIRABLE SIDE EFFECTS

Oedema and haematoma, over-correction of the zone to be treated, under-correction of the zone to be treated, cytoateatonecrosis, allergy.

DIRECTIONS FOR USE

Carefully read these instructions before using this kit

If one of the steps in the protocol described in these directions for use does not proceed according to the recommended instructions, the procedure must be stopped.

In the case of an infiltration procedure, use the infiltration cannula provided in the kit and carry out the infiltration in accordance with current practices.

- Lipoaspiration should be carried out according to your usual practice by using the lipoaspiration cannula provided in the kit and a 10 ml Luer-Lock syringe (not provided).
- Seal the end of the syringe with the provided Luer-Lock stopper.
- If centrifugation is to be carried out, withdraw the piston from the syringe and re-close the syringe with the provided cap closure.
- After centrifugation, remove the cap closure from the top of the syringe, replace the piston then remove the blanking cover from the end of the syringe.
- Remove the subnatant.
- If the adipose tissue must be transferred to a specific syringe for injection, use the Luer-Lock transfer connector provided with the kit.
- Connect the injection syringe filled with adipose tissue to the injection cannula.
- Pierce the skin at the injection site with a needle of the appropriate diameter (not provided).
- Inject the preparation into the zone to be filled while simultaneously withdrawing the annula.

Dispose of the kit components in an appropriate manner, respecting the legislation and/or precautionary principals in force for infectious risk management in the country of use.

PRECAUTIONS

The device is provided in double sterile packaging. Check to ensure that the packaging is intact before using the device. ***DO NOT USE THE DEVICE IF THE PACKAGING IS OPEN OR DAMAGED.*** Another kit must always be available in the event of failure of the first. The kit is single use. Do not re-use any element of the device. Do not use if the expiry date is exceeded. It is imperative that the recommendations contained within the directions for use are respected at all times. The kit can only be used by a surgeon qualified in the techniques of cosmetic and reconstructive medicine. Open the sachet and lay its contents on to a sterile operative field. During the procedure aseptic techniques must be respected. The wearing of safety goggles is compulsory whilst using the kit. The components of the kit must be handled with precaution. The kit should not be used if one of its components is missing or damaged. The elements of the kit are not modifiable and/or interchangeable with other external elements. The kit should no longer be used if one of its components is dropped, or there is any degradation to one of the 14 components. The kit may require the use of a centrifuge. Before beginning the procedure, the medical team must ensure that the centrifuge is set up in the operating theatre and that it is working correctly. The principal materials used in the kit are: stainless steel, polypropylene and isoprene.

STORAGE CONDITIONS

The kit must be stored at room temperature, away from direct light, and in a clean and dry place.

LIMITS OF THE GUARANTEE

ALCIS products undergo rigorous controls at various stages of the manufacturing process in order to guarantee their conformity with the reference standards. ALCIS will, under no circumstances be held directly or indirectly responsible for any loss, damage or expense:

- for the purchase, use or replacement of the kit,
- for the non-observance of the instructions, recommendations and warnings contained within the directions for use,
- for the use of unsuitable or defective accessories or tools.

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