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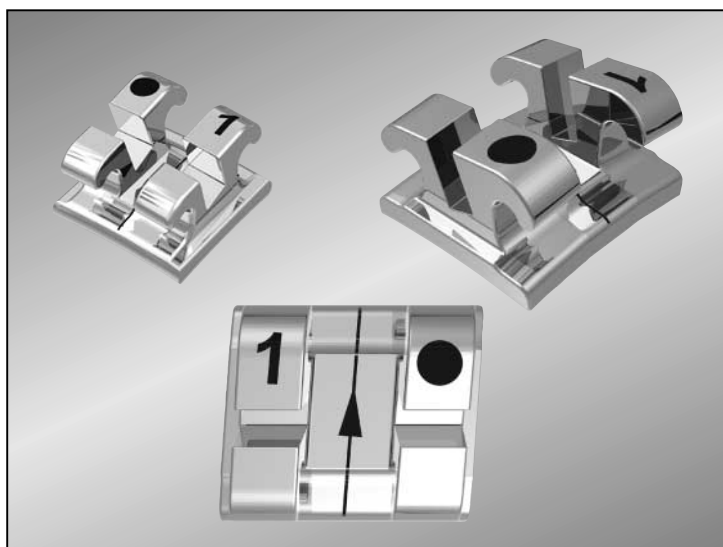
CE 0483

equilibrium[®] 2

equilibrium[®] ti

Gebrauchsanweisung	S. 3
Instructions for use	P. 6
Mode d'emploi	P. 9
Modo de empleo	P. 12
Modalità d'uso	P. 15

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EN Warranty

Dentaurum guarantees a faultless Quality of the products manufactured by us. These non-obligatory recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence in the processing on site. Eventually occurring claims for compensation may only be made up to the value of our goods.

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Dear Customer

Thank you for having chosen a quality product from Dentaureum.

In order to use this product at its best for you and your patients, it is important to study and follow these directions for use carefully.

The written instructions cannot cover all eventualities during operation. For this reason our Hotline is available to answer any other questions and ideas that may arise.

Our products are regularly modernized to the latest technical development. For this reason we advise examining the most up-to-date operating instructions supplied for any possible changes, even if the product is frequently used.

Working stages

1. Selecting the bracket adhesive

equilibrium[®] brackets can be bonded with all standard bracket adhesives. We recommend our ConTec, NO-MIX and MIX adhesives as well as Smart-Bond[®]. Adhere to the instructions for use for the particular bracket adhesive.

2. Removal and handling

Grip the equilibrium[®] brackets with haemostatic forceps (e.g. REF 025-277-00) at the mesiodistal surfaces or at the distal and mesial bracket wings and remove from the packaging. The patented laser-contoured base of the equilibrium[®] brackets should not be handled or contaminated in any way, as this would have a detrimental effect on the bond.

3. Preparing the teeth

Check each tooth carefully for any existing damage to the enamel to ensure there are no “surprises” with tooth damage during debonding.

Adhere to the bracket adhesive manufacturer’s instructions for conditioning the teeth.

4. Applying the bracket adhesive

Prepare the bracket adhesive according to the manufacturer’s instructions. Apply only a small amount to the centre of the bracket base, but enough to cover the adhesive surface. This produces an optimum bond.

5. Placing the brackets

After applying the adhesive, place the bracket on the tooth in the usual way, if necessary using the angulation aid on the haemostatic forceps; align the bracket and press it on.

Before the adhesive hardens, completely remove any excess exuding from the side of the bracket base. This is particularly important for easy debonding at the end of treatment.

When placing equilibrium® brackets, ensure that there is not any contact with the opposing dentition (risk of abrasion and premature bracket loss). If necessary, raise the bite with a bite-raising appliance.

6. Placing the arch and ligating

Ligating can be completed in the usual way after the adhesive has hardened (hardening time according to the adhesive manufacturer’s instructions).

rematitan® Lite or Tensic arches are recommended for initial treatment.

7. Debonding the brackets

Allow for the excellent bond properties of equilibrium® brackets when debonding:

equilibrium® brackets can be debonded with Weingart pliers, e.g. REF 003-120-00. Grip the brackets at the mesial and distal and loosen the base of the equilibrium® bracket carefully from the tooth by closing the pliers and bending the bracket. Then, if necessary, remove the bracket from the tooth without pulling by twisting the bracket briefly to the right or left.

General information

- Refer to the relevant product label or catalogue specifications for technical information, e.g. torque, angulation, size etc.
- Store equilibrium® brackets in sealed packs.
- No special precautions have to be taken when disposing of equilibrium® brackets after treatment.
- If you have any questions, contact the orthodontic fixed appliance hotline, tel. +49 72 31/803-550.

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