FR

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EN

COMPOSITION**Powder** : zinc oxide, magnesium oxide.**Liquid** : 42 % polyacrylic acid aqueous solution.**INDICATIONS**

1. The cementation of all fixed prostheses, crowns, onlays, inlays, pinledges, large bridges, orthodontic anchorages.

2. Cavity lining.

PROPERTIES

Polycarboxylate cements offer numerous advantages over classical luting cements, named oxyphosphates :

- adhesion superior to the dentine and the metals,
- considerable hardness associated with an increased flexibility,
- physiological neutrality allowing their use as a cement on living teeth as well as cavity liners,
- dimensional stability after setting,
- setting time varying only slightly according to temperature and humidity.

INSTRUCTIONS FOR USE**Sealing :**

Carefully degrease and dry the abutment and the prosthesis.

Place a level measuring spoonful of powder on the mixing block, add 5 drops of liquid from the dropper-bottle which should be held at an angle of approximately 45° so as to ensure the dispensing of equal size drops.

Mix powder and liquid for 30 seconds.

The working time is then 4 minutes allowing thereby the application of the cement and the positioning of the prosthesis.

Cavity lining :

Mix 3 drops of liquid with one measuring spoonful of powder for 30 seconds in the same way as mentioned above. In that case, the working time will be reduced to 3 minutes. Dry well prior to application.

WARNINGS AND PRECAUTIONS FOR USE

As SELFAST adheres to metals, it is necessary to clean without delay the stainless steel spatula after use. The mixture remains available as long as it has a shining appearance. Tightly close the bottle after use.

In case of blockage by crystals, the dispenser can be unblocked with a suitable metal instrument.

SELFFAST liquid is irritating to eyes.

PRESENTATION

Kit containing :

- 1 bottle of powder 60 g
- 1 bottle of liquid 40 ml
- 1 mixing pad
- 1 measuring spoon

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