**D**

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**GB**

Immobilisation is a basic principle in the treatment of injuries and overstrain since it greatly accelerates the recovery process and prevents secondary complications. Hard-frame knee braces support damaged tissue structures (joints, ligaments, capsules, cartilage), stabilise, guide and hold, thereby contributing to a positive recovery process.

**Contraindications:**

Peripheral arterial disease (PAD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

**Instruction for use:**

Before fitting the brace please set the extension and flexion locks on the medial and lateral tooth segment joint. The extension is preset to 0°.

A different extension or flexion stop can be set by loosening the pre-

inserted mounting screw, removing the limitation wedge and secur-

ing with the mounting screw.

Then open all the fixing straps and fit the brace with the leg slightly flexed. Position the brace such that the rotation point of the brace joints is in the horizontal plane at the height of the middle of the patella and in the vertical plane along the Nierert compromise axis

(ratio 40:60).

In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove if necessary.

If complaints continue, please contact the doctor.

If pressure points, abrasions or similar occur, have your orthopaedic technician or doctor check the fit of the brace immediately.

**Composition:**

60 % Aluminium, 22 % nylon, 18 % polyurethane

**Important instructions:**

No reutilization – this medical device is meant for the care of one patient only.

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First adjust the frontal fastener on the thigh such that the brace can-

not slide backwards. Then close the frontal infrapatellar fastener on

the lower leg. Close the fixing straps – one by one - from bottom to

top.

**Safety information:**

The brace can be adapted to the patient's physiognomy as desired. This is done by bending the metal frame and heating the tibia clasp which is thermoplastic at 130 °C with a hot air gun. **When shaping/bending the aluminium frame, it is essential to ensure that the mechanical joints are parallel in all planes, otherwise the function of the mechanical joints may be impaired.**

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