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**GB**

Back Supports support injured tissue structures (bones, ligaments and cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the vertebrae. Fit and height are adapted to the female anatomy. The pad provides a massage effect which reduces swelling tendencies or which helps to an earlier restrain of swelling.

**Instruction for use:**

Place the support around the waist and – using the hand straps – evenly overlap the two Velcro fastening ends by tensioning slightly. Ensure that the support is centered and that the lower edge of the support is in the groin area (forward bend). This is important for optimal effect of the BORT StabiloBasic. The pad is velcroed to the support and can be positioned freely. Ensure that the wide side of the pad faces upward.

**Indications:** Dorsalgia, lumbalgia, lumbago, ischialgia, pain in the sacroiliac joint, nerve root irritations.

**Contraindications:**

Patients with pronounced deformity of the spine and serious neurological disorders; also patients with intra-abdominal space-occupying processes and risk of thrombotic events.

**Composition:**

68% Elastodies, 32% polyamide

**Important instructions:**

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor’s order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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