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GB

Wrist Supports support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint. The thumb cut-out prevents the support from shifting.

The aluminium splint inlay is formed in a way that the hand is hold in a relaxed, easy position. Additionally the splint offers increased protection against wrong movements.

Instructions for use:

Before applying bend the splints in desired shape. Push the hand into the support, fasten the wrist support Velcro.

Indications:

For carpal tunnel syndrome, tenosynovitis, strains to the wrist and after removal of plaster cast. For immobilisation of the wrist after bone and non-bone injuries, radial nerve palsy, stabilisation of the carpal region.

Contraindications:

Lymphatic drainage disorders and indistinct soft tissue swellings re-mote from the applied aid. Hypesthesia and circulatory disorders of the arms. Skin disorders in the treated part of the body.

Composition:

83% Polyester, 17% microfibre

Important Instructions:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor’s order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

The product contains latex and can trigger allergic reactions.

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