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ES

Montaje

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Assembly

The following is a step-by-step process for correct assembly of the enamel stripper. Following this procedure will insure the proper functioning of the Enamel Stripper.

1. Place abrasive strip into front retention slot as shown in fig. 1. The strip should be placed with at least 3.5 mm exposed above instrument. This will insure a proper mechanical lock in step 3.
2. Wrap abrasive strip around the front of the instrument as shown in fig. 2. Make sure that the exposed 3.5 mm portion is still correctly positioned.
3. Wrap the abrasive strip back over the 1/8 [in] exposed portion of the abrasive strip as shown in fig. 3. The abrasive strip is now locked in position on the front of the enamel stripper.
4. With the abrasive strip locked in the front of the instrument, slide the opposite end of the abrasive strip through the "abrasive strip slot" located on the bottom of the locking lever as shown in fig. 4.
5. Rotate locking lever forward until it snaps into place as shown in fig. 5.

The Enamel stripper is now ready for use.

Application

When contact between teeth is extremely tight and abrasive strip cannot be passed – use a separating material such as Dentalastics® Separators (REF 774-100-01 or 774-200-01), for a few minutes. Then use a safe sided strip or one that had been previously used, follow with a new strip. Once space has been created, the other contacts loosen and additional stripping is easily accomplished.

When working on teeth in the maxillary arch, place patient in horizontal position and work from behind. This places the teeth in a more favorable working position. An upright sitting position is best when working on mandibular teeth. The stripper blade should be rotated around the axial angles of the dental crown to prevent flattening of the proximal surfaces. Washing the strip occasionally in water will prevent clogging and improve cutting. A worn abrasive strip is ideal for polishing the enamel after reduction of tooth material.

Several millimeters of space may be obtained by stripping the mesial and distal surfaces of the six anterior teeth.

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Dentaurum GmbH & Co. KG · Turnstraße 31 · 75228 Ispringen · Allemagne