Estimado cliente

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Dear Customer

Thank you for having chosen a quality product from Dentaureum.

In order to use this product at its best for you and your patients, it is important to study and follow these directions for use carefully.

The written instructions cannot cover all eventualities during operation. For this reason our Hotline is available to answer any other questions and ideas that may arise.

Due to a constant development we recommend, even when you use the same products frequently, to study the enclosed latest issue of the instructions for use or refer to our website at www.dentaureum.de.

1. Manufacturer

Dentaureum GmbH & Co. KG
Turnstr. 31 · 75228 Ispringen · Germany

2. General product description

The hyrax® screw (Hygienic Rapid Expansion Screw) is a special screw for rapid palatal expansion. The retention legs of the palatal expansion screw are welded or soldered onto preformed metal bands or cap splints. Alternatively, they can be polymerized into acrylic splints. The bands or splints are cemented in the usual way.

3. Area of application

Used for rapid palatal expansion with a fixed appliance.

4. Fabricating a hyrax® appliance using the example of preformed bands

Preformed bands are fitted on the maxillary left and right 1st bicusps and left and right 1st molars. These bands are transferred to a dental stone model using an impression or fitted to a dental stone model. The legs of the hyrax® screw are then bent to fit the bands so that they can be soldered or welded into place (Fig. 1).

To prevent the legs from breaking during bending, we recommend using our universal bending tool (REF 611-122-00, Fig. 4). If one is not available, hold the legs against the body of the expansion screw, using strong flat pliers (Fig. 5).

The hyrax® screw should „float“ freely at least 1-2 mm above the palate roof/gingiva (Fig. 2). Its posterior side should form a straight line with the mesio palatal cusps of the 1st molars. To avoid any leverage effect, it should be ensured that the occlusal level is parallel.

5. Information on use of hyrax® screws

To increase strength, the bands can be joined on the buccal and/or lingual side with a piece of 1.0 mm remaloy® wire (REF 528-100-00). Laser weld or solder these wires above the legs. The palatal strengthening wire should project slightly beyond the bands and be supported by the adjacent teeth to allow better distribution of expansion forces.

Easy, problem-free bending of the retention legs of hyrax® screws with the universal bending tool (REF 611-122-00), without the use of additional pliers (leverage principle).

To prevent the legs from breaking during bending, they should be held against the body of the expansion screw using strong flat pliers e.g. Maxi wire-bending pliers (REF 013-522-00).

When using the soldering technique please protect the body of the screw against excessive heat, e.g. with heat screen (REF 380-070-00).

6. Composition

The composition is included in the materials list, refer to www.dentaureum.de.

7. Safety information

Always use the blue safety key supplied to prevent swallowing!

The product should not be used, if there is a known allergic reaction to one or more of the material components.

8. Information on single-use products

The product is intended for single use only. Reconditioning of a product that has already been used (recycling) and its reuse on a patient is not permitted.

9. Availability

Description	straight legs	bent legs	Quantity
hyrax® Mini - 7 (Expansion 7 mm)	602-800-10 602-800-30	602-801-10 602-801-30	1 piece 10 pieces
hyrax® Medium - 10 (Expansion 10 mm)	602-808-10 602-808-30		1 piece 10 pieces
hyrax® Maxi - 12 (Expansion 12 mm)	602-811-10 602-811-30	602-812-10 602-812-30	1 piece 10 pieces

with safety key

Safety key	611-120-01	10 pieces
Universal bending tool	611-122-00	1 piece

10. With reference to our quality standards

Dentaureum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

11. Explanation of symbols used on the label

⚠ Please refer to the label. Additional information can be found at www.dentaureum.de (Explanation of the symbols used on the label REF 989-313-00).

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