TILDATM

the seat tilter

BRUKSANVISNING (S)

USER MANUAL (GB)

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MODO DE EMPLEO (E)

MANUALE D'USO (I)

BRUKSANVISNING (N)

CE

 auto[®]
ADAPT
for your independence

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GB CONTENTS

Declaration of conformity	page 2
Footrest as option	page 3
User instructions	page 5

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GB FOOTREST AS OPTION

The footrest is unfolded by first pulling the knob placed on the side. Not needed when just folding it.

Height adjustment is made by first unscrewing the three screws placed at the front.

Adjustment forwards and backwards is made by first unscrewing the 4 screws that are placed underneath the attachment towards Tilda.

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USER INSTRUCTIONS

Presentation

- Simplifies getting the feet over the sill and provides more space for the head when TILDA is used with the swivelbase TURNOUT.
- TILDA provides three tilt positions:
The maximum tilt position is 15°, but can be limited to 11° or 7°.
- Footrests are available as option and makes it easier to get the feet over the sill when TILDA is used with the TURNOUT swivel base.
- Maximum user weight: 150 kg.
Maximum load on footrests: 70 kg.
Weight incl.footrest: 8,3 kg
Width: 520 mm
- Crash tested and approved in compliance with ECE R-17 and 2005/39/EEC for seat in position 0 and tilt position 1.

How Tilda is used

TILTING THE SEAT:

1. Release the tilt function by pressing both handles towards each other (fig. 1) and pull them backwards.
2. Release the lower handle (lock handle) to allow the lock to fix inside one of the three tilt position catches.

RISING THE SEAT:

1. Press the two handles towards each other (fig. 1) and push them forwards until the seat is risen to an upright position.
2. Let go of the lower handle (lock handle) to allow the seat to be fixed in the desired upright position.
If it feels heavy just put your hand on top of the backrest and push.

IN SNUG CONDITIONS

In some vehicles with a high and wide centre console it might be necessary to slide the seat forward or to limit the tilt angle before swivelling outwards (in Turn-out applications).

⚠ RISK OF INJURIES!
Make sure that no fingers are under the seat when rising it from tilted position.

⚠ During drive the seat may not be tilted to more than pos. 1 (lowest tilt position)

⚠ When combined with Carony the Anti-tip protection must be used.

⚠ Watch your head when passing through the doorway.

Traceability

Tilda has a CE-poster where the serial number is stated.
Please write down this number!
The number is needed when ordering spare parts.

Serial No: _____

Cleaning and maintenance

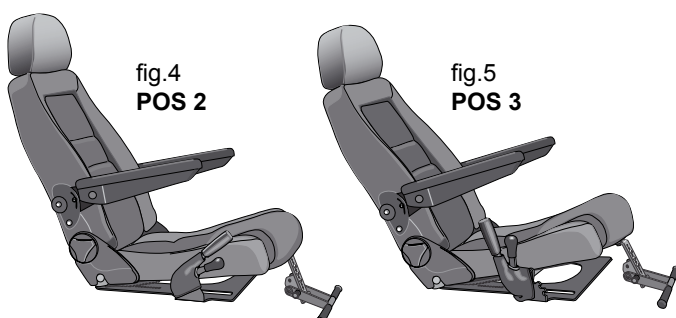
- ! Make sure that there are no foreign objects or gravel in the mechanics or between the plates.**

Clean with a vacuum cleaner. Can also be wiped gently with a moist towel and some mild ceaning detergent, when needed.

Tilda is designed to function maintenance-free.
Check irregularities such as loosening screws ot unusual noise during use.

Reinstatement

To remove Tilda, follow the installation instructions in reversed order. Removal time 1 h.

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