
tomas®-X marker*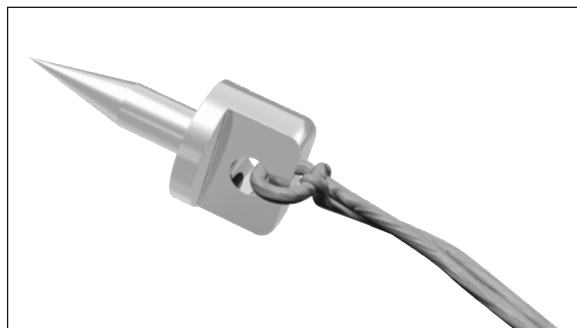

* Idee: Dr. Björn Ludwig (Traben-Trarbach, Germany)

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Instructions for use

Mode d'emploi

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Instructions for use tomas®-X marker

Concept: Dr. Björn Ludwig (Traben-Trarbach, Germany)

Dear Customer

Thank you for choosing this high-quality product from Dentaurum. In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use. The written instructions cannot cover all possible situations that can occur during use. For this reason, our hotline is available to answer any other questions or concerns that may arise. Due to constant developments, we recommend, even when using the same products regularly, that you study the enclosed current instructions for use or refer to our website at www.dentaurum.de.

1. Manufacturer

Dentaurum GmbH & Co. KG · Turnstraße 31 · 75228 Ispringen · Germany

2. With reference to our quality standards

Dentaurum ensures a faultless quality of the products that we manufacture. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

3. General information

The tomas®-X marker is useful during pre-operative diagnostics for locating the correct position for the tomas®-pin. The largest part of the tomas®-X marker is identical to the shaft diameter of the tomas®-pins, thus the space requirements can be accurately assessed. The product is supplied with a packaged safety thread, prepared for sterilization.

4. Procedure

The tomas®-X marker has been cleaned, disinfected and is ready for sterilization. Place the tomas®-X marker in the sterilization bag into the steam sterilizer at 134 °C/273 °F for 5 min.

Administer anesthetic to the planned area of insertion (superficial or local anesthetic). Open the sterile packaging and remove the tomas®-X marker using Weingart pliers (e.g. REF 003-120-00). Hold at the shaft handle and press into the soft tissue at the area of insertion, up to the bone (Fig. 1). Take an x-ray picture of the situation (Fig. 2). When making a dental film, use the right angled technique. The x-ray image enables the final position of the tomas®-pin to be easily determined.

Taking the clinical and paraclinical findings into account, the tomas®-pin is now ready to be inserted at the selected position (Fig. 3-4).

5. Safety instructions

The accuracy of any type of x-ray aid such as x-ray tube, x-ray object, x-ray film and x-ray sensor is dependant upon their position to each other, and is susceptible to optical distortion. The interpretation of such an image can lead to false positive or false negative results. Therefore when positioning a mini screw, please bear the clinical findings in mind as the decisive factor.

The tomas®-X marker is intended for single use only. It is not permissible to recycle the tomas®-X marker and after single use must not be re-used on any other patient.

6. Availability

tomas®-X marker, 1 piece, REF 302-004-19

7. Explanations of symbols

REF	Order number
	Batch description
	Non-sterile
	CE Mark MP Class 1
	Not for reuse
	Instructions for use available in the internet
	Codablock barcode
	Matrix code

Date of Information: 04/10
Subject to change

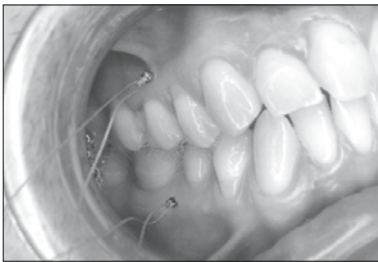
Fig. 1

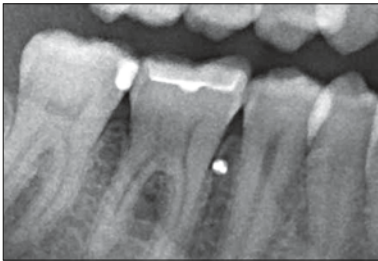
Fig. 2

Fig. 3

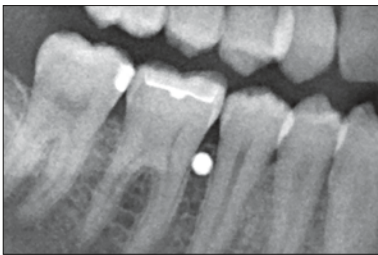
Fig. 4

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Dentaureum GmbH & Co. KG · Turnstraße 31 · 75228 Ispringen · Germania